VERMEIREN

Venus 3 - 4

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: A, 2014-03

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2014

Contents

1	Product description	3
1.1	Intended Use	3
1.2	Safety instructions	4
1.3	Technical specifications	5
1.4	Accessories	6
1.5	Components	7
1.6	Location identification plate	8
1.7	Explanation of symbols	8
2	Use	9
2.1	Remarks on electromagnetic compatibility (EMC)	9
2.2	Carrying the scooter	10
2.3	Assembly and disassembly of the scooter	10
2.4	Operating the brakes	11
2.5	Transfer in and out the scooter	12
2.6	Correct position in the scooter	12
2.7	Driving the scooter	12
2.8	Driving the scooter on ramps	15
2.9	Driving the scooter over steps	15
2.10	Operator control	16
2.11	Neutral	17
2.12	Transport in the car	17
2.13	Automatic Power shut down	18
2.14	Charging the batteries	18
3	Installation and adjustment	20
3.1	Tools	20
3.2	Manner of delivery	20
3.3	Mounting / removing rear frame	20
3.4	Adjusting the steering column	21
3.5	Mounting / removing the shopping basket	21
3.6	Adjusting the seat	21
3.7	Folding the backrest	23
3.8	Adjusting the armrests	24
3.9	Locking steering wheels	24
3.10	Thermal fuses	25
3.11	Anti-tipping	25
3.12	Changing the batteries	26
4	Maintenance	27
4.1	Troubleshooting Fault codes	27
5	Declaration of conformity	28

Preface

First of all we want to thank you for putting your trust in us by selecting one of our scooters.

The expected lifetime of your scooter is strongly influenced by the care and maintenance of the scooter.

This manual will help you get acquainted with the operation of your scooter.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

1 Product description

1.1 *Intended Use*

The scooter is intended to comfortably transfer people. This scooter is designed to transfer 1 person. Do not carry passengers on your scooter (including children).

The scooter is designed to transfer adult people only, no goods, nor any persons under the age of 16.

Neither should it be used by people obviously suffering from physical or mental limitations that render them incapable of safely handling the scooter in traffic.

These limitations could involve:

- hemiplegics or paraplegics
- loss of limbs (arm amputation)
- limb defects or deformations (if resulting in reduced equilibrium and steering abilities)
- contractures or damaged joints (if resulting in reduced equilibrium and steering abilities)
- balance disturbances
- cachexia (decrease in muscle)
- mental disorders
- damage to the cerebral cortex owing to psychological disorders
- visual impairment

The scooter is suited for indoor and limited outdoor use by the occupant.

When providing for individual requirements:

- body size and weight (max. 130 kg)
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

Your scooter should be used on flat surfaces where all wheels are touching the ground and where there is sufficient contact to propel the wheels equally.

You should practice for use on uneven surfaces (cobblestones, etc.), slopes, curves and to get past obstacles (curbs, etc.).

The scooter should not be used as a ladder, nor is it a transport for heavy or hot objects.

When using your scooter on streets or footpaths, local laws and regulations apply.

The scooter can be used on sidewalks, urban road. Under no circumstances the scooter may be used for driving on larger roads or expressways.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the warranty conditions so we recommend that you read the following pages very carefully.

Visually impaired people can contact the dealer for the instructions for use.

No complications are known for the scooter.

1.2 Safety instructions

-  Use only Vermeiren approved accessories.
-  Before getting on or off, taking apart or transporting your scooter, turn the start key to the “off” position.
-  When transporting the scooter, no persons may be transferred along with it.
-  Test the effects on the handling of the scooter when its center of gravity is shifted, for example on up or down gradients, on laterally sloping ground or when driving over uneven terrain.
-  Do not drive on roads with heavy traffic or roads that are muddy, gravelly, bumpy, narrow, snowed over, icy or canal towpaths not guarded by any fence or hedge.
-  Keep away from places where you might get the wheels stuck.
-  Pay attention that your clothes do not tangle in the wheels.
-  When picking up objects lying in front of, next to, or behind the scooter, take care that you do not lean too far sideways: risk of tipping over.
-  Avoid putting your scooter into the free-wheeling position on slopes.
-  Never reverse uphill.
-  Reduce speed when you go round corners, driving indoors.
-  Do not take the scooter onto escalators.
-  Do not make sudden turns when driving on gravel roads or ramps.
-  Do not drive on roads with large drops or potholes.
-  While driving, hold the handgrips with both hands.
-  Put your legs and feet on the surfaces provided for them during the drive.
-  Do not use your scooter in snowing, misty, heavy windy conditions or at night.
-  When storing or parking your scooter outside, protect it with a shrouding cover against humidity.
-  High levels of humidity or very cold conditions can reduce the performance of your scooter.
-  Use your scooter strictly in accordance with regulations. Avoid driving straight over obstacles (e.g. step, edge of the curb) or down from high ledges.
-  Obey traffic regulations when driving on public roads. Please take other road-users into consideration. Ride on the pavement, single carriage roads, or pedestrian areas only. Never ride on motorways or dual carriageways.
-  As is the case with any other vehicle, you must never be under the influence of alcohol or medicine while driving your scooter. This also applies to indoor driving.
-  When traveling outdoors, adapt your driving to weather and traffic conditions.
-  When driving in low light, wear bright clothing or clothing with reflectors to be more visible. Your scooter is not designed for use at night.
-  Never use your scooter as a seat in an automobile or other vehicle.
-  Check that the profile depth of the tyres is adequate.
-  Make sure that the maximum load of the scooter is not exceeded.
-  Do not overload the shopping basket. The maximum load of the shopping basket is 3 kg.
-  If someone else uses the scooter, please make sure that you provide them with the instruction manual for his or her consideration.

1.3 Technical specifications

Technical terms below are valid for the scooter in standard settings and optimum environmental conditions. If other accessories are used, the tabulated values will change. Changes in outdoor temperatures, humidity, uphill, downhill, soils and battery levels can reduce output.

Brand	Vermeiren	
Address	Vermeirenplein 1/15, B-2920 Kalmthout	
Type	Rear wheel drive scooters	
Maximum occupant mass	130 kg	
Model	Venus 3	Venus 4
Description	Minimum dimensions	Maximum dimensions
Max. speed	6 km/h	
Continuous driving distance range*	10,5 km (battery: 12Ah)	
Length	1090 mm	1070 mm
Width	510 mm	
Height	900 mm	
Folded / dismantled length	Front frame: 910 mm Rear frame: 340 mm	Front frame: 940 mm Rear frame: 340 mm
Folded / dismantled width	Front frame: 430 mm Rear frame: 490 mm	Front frame: 440 mm Rear frame: 490 mm
Folded / dismantled height	Front frame: 380 mm Rear frame: 230 mm	Front frame: 380 mm Rear frame: 230 mm
Total mass	43 kg	46 kg
Mass of heaviest part (that can be dismantled or removed)	35,1 kg (without seat, basket) or 25,7 kg (without seat, basket and batteries)	38,1 kg (without seat, basket) or 28,7 kg (without seat, basket and batteries)
Masses of parts that can be dismantled or removed	Seat: 7,4 kg; Battery housing: 9,4 kg; Shopping basket: 0,5 kg; Rear frame: 10,80 kg	
Maximum safe slope	8°	
Obstacle climbing	60 mm	
Ground clearance	30 mm	
Seat plane angle	5°	
Effective seat depth	380 mm	
Seat surface height at front edge	525 mm - 575 mm	525 mm - 575 mm
Backrest angle	8°	
Backrest height	350 mm	
Distance between armpad and seat	200 mm	
Front location of armrest structure	330 mm	
Motor	Nom. 270 Watt, electromagnetic brakes	
Batteries	12 Ah	
Controller	45 A	
Nominal voltage (battery)	2 x 12V ---	
Degree of protection	Not applicable	
Battery charger	1,8 Amp (external)	
USB connection	No	
Minimum turning diameter	1100 mm	1400 mm
Reversing width	1530 mm	1620 mm
Diameter rear wheels (number)	200 x 50 mm PU tire (2)	

Brand	Vermeiren	
Address	Vermeirenplein 1/15, B-2920 Kalmthout	
Type	Rear wheel drive scooters	
Maximum occupant mass	130 kg	
Model	Venus 3	Venus 4
Description	Minimum dimensions	Maximum dimensions
Tyre pressure, rear (driving) wheels	Not applicable	
Diameter steering wheels (number)	200 x 50 mm PU tire (1)	200 x 50 mm PU tire (2)
Tyre pressure, steering wheels	Not applicable	
Handle bar	Speed control lever	
Lights	No	
Indicator light	LED	
Mirrors	No	
Storage and use temperature	+5 °C to +41 °C	
Operating temperature of the electronics	-10°C to +40°C	
Storage and use humidity	30%	
<p>We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / °.</p> <p>* The theoretical driving distance will be reduced if the scooter is used frequently on slopes, rough ground or to climb curbs. Maximum driving distance is based on an ambient temperature of 20°C, a 75 kg driver and a brand new fully recharged battery by a constant driving speed at 6,4 km/h with 70% battery power discharged.</p>		

Table 1: Technical specifications


1.4 Accessories

The following accessories are available for the scooters Venus 3, Venus 4:

- WARNING: Risk of injury - Be sure that crutches are attached very well and cannot fall on the user.**

Crutch holder


1.5 Components

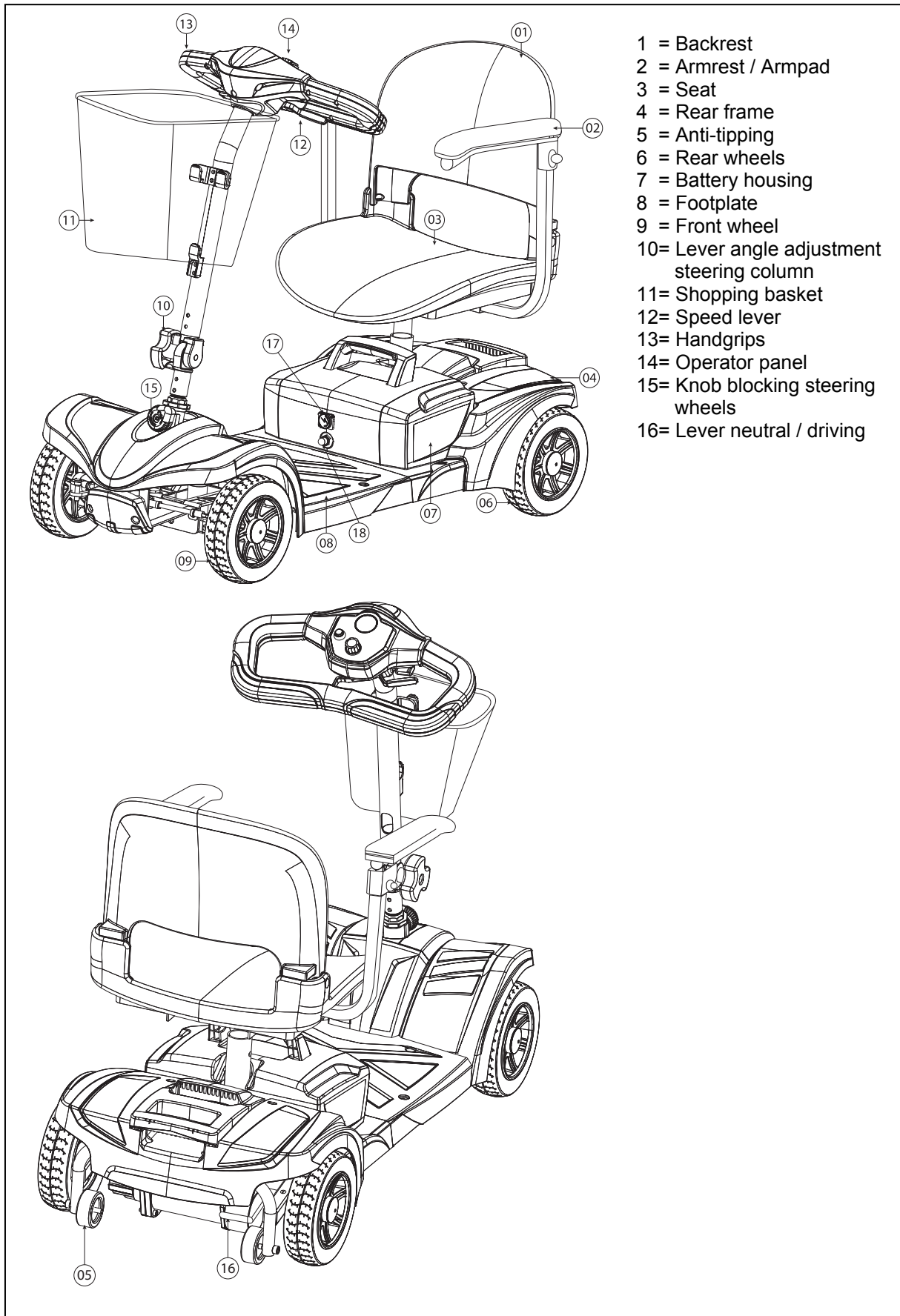1.6 Location identification plate

The location of the identification plate is on the seat tube. Remove the battery housing to see the identification plate.

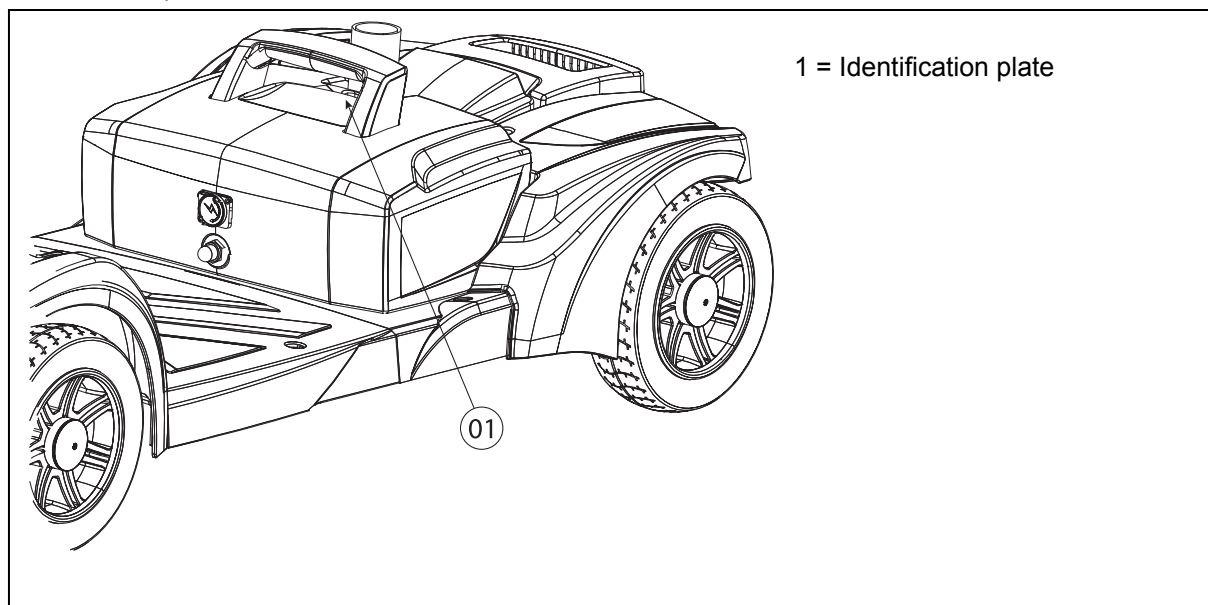1.7 Explanation of symbols

Maximum mass

Indoor and outdoor use

Maximum safe slope

Separate recovery and recycling of electric and electronic devices

CE conformity

Maximum speed

Not intended to be used as a seat in a Motor vehicle

Type designation

Warning: Do not Pinch hands / fingers

Warning EMI label

Please carefully read all labels on the scooter before driving. For your future reference, do not remove them.

2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**

The scooter is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the scooter are given in § 2.3 and 3.

2.1 Remarks on electromagnetic compatibility (EMC)

Your scooter has been tested according EN 12184: 2009 section 9.

We wish to point out that sources of electromagnetic waves (e.g. cell phones) are liable to create interferences. The electronics of the scooter itself could affect other electric appliances too.

To reduce the effect of electromagnetic sources of interference, please read the following warnings:

- ⚠ **WARNING: The scooter might disturb the operation of devices in its environment that emit electromagnetic field.**
- ⚠ **WARNING: The driving performance of the scooter can be influenced by electromagnetic fields (e.g. electricity generators or high power sources).**
- ⚠ **WARNING: Avoid using any portable TV or radio in the immediate environment of your scooter for as long as it is turned on.**
- ⚠ **WARNING: Avoid using any transmitter-receiver or cell phones in the immediate environment of your scooter for as long as it is turned on.**
- ⚠ **WARNING: Check the area for transmitter masts and avoid using the scooter close to them.**
- ⚠ **WARNING: If involuntary movements or braking occur, turn off the scooter as soon as it is safe to do so.**

Interfering electromagnetic fields may have a negative effect on the scooter's electronic systems. These can include:

- Disengagement of the engine brake
- Uncontrollable behavior by the scooter
- Unintentional steering movements

In the presence of very strong or enduring interfering fields, the electronic systems could even break down entirely or suffer permanent damage.

Possible sources of radiation include:

- Portable receiver and transmitter installations (receiver and transmitter with fixed antenna mounted directly on the transmitting unit)
- Transmitting and receiving sets
- Cell phones or "walkie talkie"
- portable TV, radio and navigation devices
- other personal transmitting devices

- Mobile medium-range transmitting and receiving devices (e.g. car antennas, with the antenna mounted on the outside of the vehicle)
 - Fixed transmitting and receiving sets
 - Fixed mobile transmitting and receiving devices
 - Fixed radio, TV and navigation systems
- Long-range transmitting and receiving devices
 - Radio and television towers
 - Amateur radio sets

Other home devices like CD player, notebook, cordless phones, AM/FM radios, electric shavers and hair dryers, so far as we know, will have no influence provided that they function perfectly and their cabling be in an excellent condition. Please conform to the operating instructions accompanying such electric instruments to ensure trouble free operating of your scooter.

1. Do not operate hand-held (transceivers-receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as cellular phones, while the powered vehicle is turned ON;
2. Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them;
3. If unintended movement or brake release occurs, turn the powered vehicle OFF as soon as it is safe;
4. Be aware that adding accessories or components, or modifying the powered vehicle, may make it more susceptible to EMI (Note: There is no easy way to evaluated their effect on the overall immunity of the powered vehicle);

The intensity of the interfering EM energy can be measured in volts per meter (V/m). Each powered vehicle can resist EMI up to a certain intensity. This is called its "immunity level". The higher the immunity level, the greater the protection. At this time, current technology is capable of achieving at least a 20 V/m immunity level, which would provide useful protection from the more common sources of radiated EMI. This powered vehicle model as shipped, with no further modification, has an immunity level of 20 V/m without any accessories.

2.2 Carrying the scooter

Turn the scooter "OFF" before carrying.

The weight of the scooter is for Venus 3: 34,8 kg (with batteries) or 24,4 kg (without batteries) and Venus 4: 38,1 kg (with batteries) or 28,7 kg (without batteries). Carry the scooter with 2 or 3 persons to the desired place. Take the frame on the chassis, and not by the bumpers or the plastic parts. Only on the fixed parts from the frame. On the rear frame is a handgrip that can be usefull to carry the scooter to the desired place.

The best way to carry the scooter is to make use of neutral mode of the scooter. Place the scooter in neutral and roll the scooter to the desired place.

If this is not possible you can carry the scooter by following steps:


1. Switch the scooter off.
2. Remove loose parts (seat, battery housing, shopping basket).
3. Store loose parts in a safe place.
4. Fold the steering column down.
5. Remove the rear frame from the front frame. On the rear frame is a handgrip to carry the rear frame to the desired place.
6. Lift the seperate parts to the desired place.

2.3 Assembly and disassembly of the scooter

- ⚠ WARNING: Risk of injury - Make sure that all parts are assembled properly.**
- ⚠ WARNING: Risk of injury - Assembly and disassembly must be done by trained personnel (e.g. specialist dealer).**
- ⚠ WARNING: Risk of injury - Be careful not to cut or bruise yourself with the cables.**
- ⚠ WARNING: Risk of injury - Turn the scooter off before disassembly / assembly.**

⚠ CAUTION: Risk of pinching - Do not place fingers between the components of the scooter (frame, backrest).

Never use your scooter as a seat in an automobile or other vehicle. Indicate on the serialnumber label with following symbol.

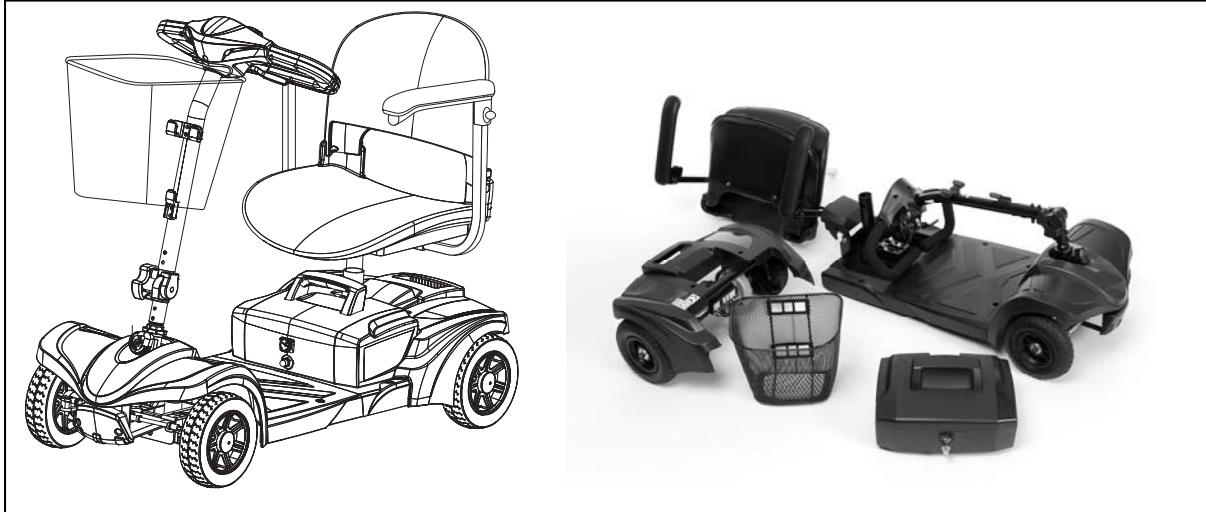No tools are needed to assemble or disassemble your scooter.

To disassemble, please proceed as follows:

- Switch the scooter off.
- Place the free wheel mode lever in the drive position.
- Fold the backrest to the front (see "Folding the backrest" chapter).
- Remove the seat (see "Adjusting the seat" chapter).
- Unlock the battery lock part and rotate the battery lock part.
- Lift the battery housing out the scooter (see "Changing the batteries" chapter).
- Remove the shopping basket (see "Mounting / removing the shopping basket" chapter).
- Fold the steering column down (see "Adjusting the steering column" chapter).
- Lock the front wheels (see "Locking steering wheels" chapter).
- Remove the rear frame from the front frame (see "Mounting / removing rear frame" chapter).

To reassemble, please proceed as follows:

- Place the free wheel mode lever in the drive position.
- Mount the rear and front frame in each other (see "Mounting / removing rear frame" chapter).
- Move the steering column upwards (see "Adjusting the steering column" chapter).
- Mount the shopping basket (see "Mounting / removing the shopping basket" chapter).
- Mount the battery housing (see "Changing the batteries" chapter).
- Mount the seat (see "Adjusting the seat" chapter).
- Fold the backrest to the back (see "Folding the backrest" chapter).
- Unlock the front wheels (see "Locking steering wheels" chapter).
- Turn the key to "ON". Make sure the battery gauge indicates sufficient power.

2.4 Operating the brakes

The scooter is foreseen from regular and parking brakes according STVZO (the German traffic law). Let go of the joystick to stop the scooter and activate the regular brakes / electromagnetic brakes. With the freewheel device you can operate the parking brakes.

2.5 Transfer in and out the scooter

1. Park the scooter as close as possible to the place where you want to transfer.
2. Check that the control panel (scooter) is switched off.
3. If necessary: put the armpad on the side you want to transfer upwards or rotate the seat towards you.
4. Transfer to / from the seat of the scooter.

2.6 Correct position in the scooter

Some recommendations for a comfortable use of the scooter:

1. Position your backside as close as possible to the backrest.
2. Make sure your upper legs are horizontal.

2.7 Driving the scooter

⚠ WARNING: Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.

⚠ WARNING: Risk of unsafe settings - Use only the driving characteristics described in this manual.

2.7.1 Preparing the scooter for use

When you use your scooter for the first time, make sure that it is standing on level ground. All wheels must be in contact with the ground.

1. For first time use, fully recharge the batteries.
2. Make sure that the motor is engaged.
3. Set the steering column in the most comfortable position for yourself and make sure it has been secured properly.
4. See that the seat has been locked in the driving position.
5. Sit down on the seat and verify that both armpads are folded down so you can rest your arms on them.
6. Insert contact key and turn to the right.
7. Check the battery indicator and see that there is enough power for your trip. If not enough capacity recharge the batteries before departure.

Now put the speed switch on the operator control to the minimum position. Your scooter is now ready for use. Only use higher speed setting when you are confident that you can easily operate and control your scooter.

Check the forward/reverse speed lever works correctly and make sure the electromagnetic brake works correctly.

Before using the scooter in busy or potentially dangerous areas, familiarize yourself with the operation of your scooter. Please practice in a wide and open area like a park.

In order to avoid accidents with your scooter while driving, please bear in mind driving motions, such as accelerating, stopping, turning, reversing, up-and down ramps.

2.7.2 Handling after usage

Before getting off from your scooter, make sure that all wheels touch the ground simultaneously. Next turn the key to the "OFF" (charge indicator goes out) to turn off the integrated light.

2.7.3 Parking the scooter

After stopping, turn the main key to OFF and remove the key. The lights will turn OFF. Once your scooter has been turned off, no command can be sent to the driving system. You will be unable to deactivate the electromagnetic brakes before turning on your scooter again. Always park your scooter on sites that are well guarded or clearly visible.

2.7.4 Your first trip

- ⚠ WARNING: Control your scooter - Accustom yourself with the driving behavior of your scooter.**
- ⚠ WARNING: Control your scooter - Do not use both sides of the speed lever simultaneously. This might leave you unable to control your scooter.**
- ⚠ WARNING: Control your scooter, risk of injury - Do not adjust the speed while you are driving.**
- ⚠ WARNING: Risk of injury - Be sure no people or objects are behind you while reversing the scooter.**

- Driving

Once you are seated on your scooter and have started it as described above, take hold of the handgrips with both hands, put your thumbs to pull the speed lever in the required direction, i.e.:

PULL ACTION BY THE RIGHT HAND	=	FORWARD MOVEMENT
PULL ACTION BY THE LEFT HAND	=	BACKWARD MOVEMENT

While using your scooter indoor, do not put your scooter in the highest speed.

- Driving on the pavement

Ensure the 6 km/h limit switch is set to the slowest setting when using the scooter on the pavement.

This will limit the scooter to a maximum speed of 6 km/h.

The maximum speed to drive on the pavement depends from country to country. Check the traffic law from the relevant country.

The limit switch can be set to the fastest setting for use on the road or on private ground.

- Braking

To brake, let go of the speed/drive lever, which will let it return to the zero position and slow down your scooter to a gentle stop. Practice pulling away and braking to get accustomed to the scooter. You need to be able to estimate how your scooter will react when you drive or brake.

Do not turn your scooter "OFF" while driving as this will lead to an emergency stop and possible risk of accident and injury.

- Driving in corners and bends

⚠ WARNING: Risk of tipping over - Reduce your speed before you enter a curve or corner.

⚠ WARNING: Risk of clamping - Always maintain an adequate distance from corners and obstacles.

⚠ WARNING: Risk of tipping over - Do not drive your vehicle in an "S" pattern or make erratic turnings.

For corners and bends, use both hands to turn the handle bar in the direction you wish to take. The front wheels will turn accordingly and steer the scooter into a new direction. It is very important that you ensure that there is enough space allowing you to go around bends and corners. Narrow passages must preferably be approached in a large curve so as to allow you to enter the narrowest part as straight from the front as possible. Bear in mind that your scooter's back part will in most cases be wider than the front part.

Take care not to enter curves and corners diagonally. By 'cutting the corner', chances are that your rear wheels will run into obstacles and destabilize your scooter.

2.7.5 Backward movement

- ⚠ WARNING: Control your scooter - Accustom yourself with the driving behavior of your scooter.**
- ⚠ WARNING: Control your speed - Always go backward in the lowest speed possible.**
- ⚠ WARNING: Risk of collision - Always look to the back when you go backward.**

Rearward driving requires increased concentration and care (LEFT HAND ACTION). This explains why we have greatly reduced the speed of rearward movement in comparison with forward driving. But we still recommend that you put the speed lever on minimum when you drive backward. While reversing the scooter, be aware of people or objects behind you.

Do not forget that the steering direction with rearward driving is the opposite of when you go forward, and that your scooter will turn directly in the required direction.

When you are driving backwards the scooter makes a sound.

2.7.6 Going uphill

- ⚠ WARNING: Control your scooter - Accustom yourself with the driving behavior of your scooter.**
- ⚠ WARNING: Control your scooter - Never put your scooter in neutral on slopes.**
- ⚠ WARNING: Control your speed - Moving on slopes as fast as possible.**
- ⚠ WARNING: Risk of tipping over - Do not exceed the maximum gradient of static stability uphill (see paragraph "Technical specifications").**
- ⚠ WARNING: Never reverse going uphill.**

Always approach slopes directly from the front and, to avoid tipping over, see that all wheels stay in contact with the ground at all times (ramps, driveways, etc.). Always lean forward when climbing a steep gradient. Your scooter is powered through a differential. Both drive wheels should therefore stay in contact with the ground at all times. Should one of the drive wheels no longer be in contact with the ground, a safety device will cut the transmission of energy to the wheels, bringing the scooter to a halt.

If you come to a stop on a slope by letting go of the accelerator, the engine break will prevent your scooter from rolling backward. As soon as the accelerator returns to zero position, the engine break is activated.

To resume your uphill drive, pull the speed lever open fully to ensure the release of a sufficient amount of power. This will allow your scooter to slowly ascend the slope.

If your scooter is unable to drive up, turn the speed control up and try again.

When driving on a gradient, the battery indicator light might move up and down. This is a normal phenomenon so please do not worry.

2.7.7 Going downhill

- ⚠ WARNING: Control your scooter - Accustom yourself with the driving behavior of your scooter.**
- ⚠ WARNING: Control your scooter - Never put your scooter in neutral on slopes.**
- ⚠ WARNING: Control your speed - Moving on slopes as slow as possible.**
- ⚠ WARNING: Risk of tipping over - Avoid sharp bends.**
- ⚠ WARNING: Risk of tipping over - Do not exceed the maximum gradient of static stability downhill (see paragraph "Technical specifications").**

Always approach downhill slopes directly from the front. Slantwise approaches can lead to some of the wheels no longer staying in contact with the ground (danger of tipping over). If one of the back wheels fails to make contact, the transmission of power will be cut and the scooter come to a standstill.

The weight of the scooter will increase your downhill speed. Turn the speed control down and adjust your speed to the conditions.

Avoid sharp bends on downhill slopes. The weight of your scooter could cause your scooter to lift up on one side or even fall over around bends.

2.8 Driving the scooter on ramps


- ⚠ WARNING: Risk of injury - Do not exceed the maximum load of the ramps.**
- ⚠ WARNING: Risk of injury - Choose the correct ramps so there cannot be caused any injury or damage.**
- ⚠ WARNING: Risk of injury - Note that the considerable weight of the electric scooter generated large reverse forces when an attendant is pushing the scooter over the ramps.**
- ⚠ WARNING: Risk of injury - Be sure that the wheel height is high enough to get over the free height of the ramps. The scooter frame may not touch the ramps.**
- ⚠ WARNING: Risk of injury - Do not drive onto steep ramps greater than the specified maximum angle (refer table 1).**

Should you wish to use ramps to drive over an obstacle or drive the scooter in the car, please note the following:


1. Find out from the manufacturer what the maximum load is for the ramps.
2. Drive on the ramps at the lowest speed possible.
3. If you want that your scooter only drives straight, it is possible to lock the front wheels (see "locking steering wheels" chapter).
4. See the instructions in the chapter "your first trip".

2.9 Driving the scooter over steps

It is possible to drive with your scooter over steps from 60 mm. Please start from 500 mm to take these step.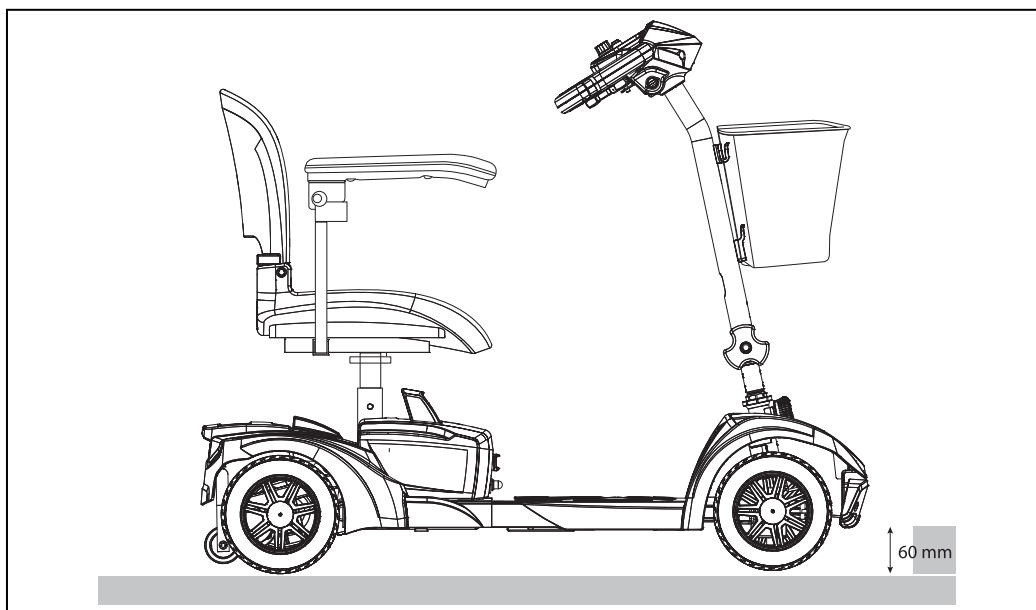


2.10 Operator control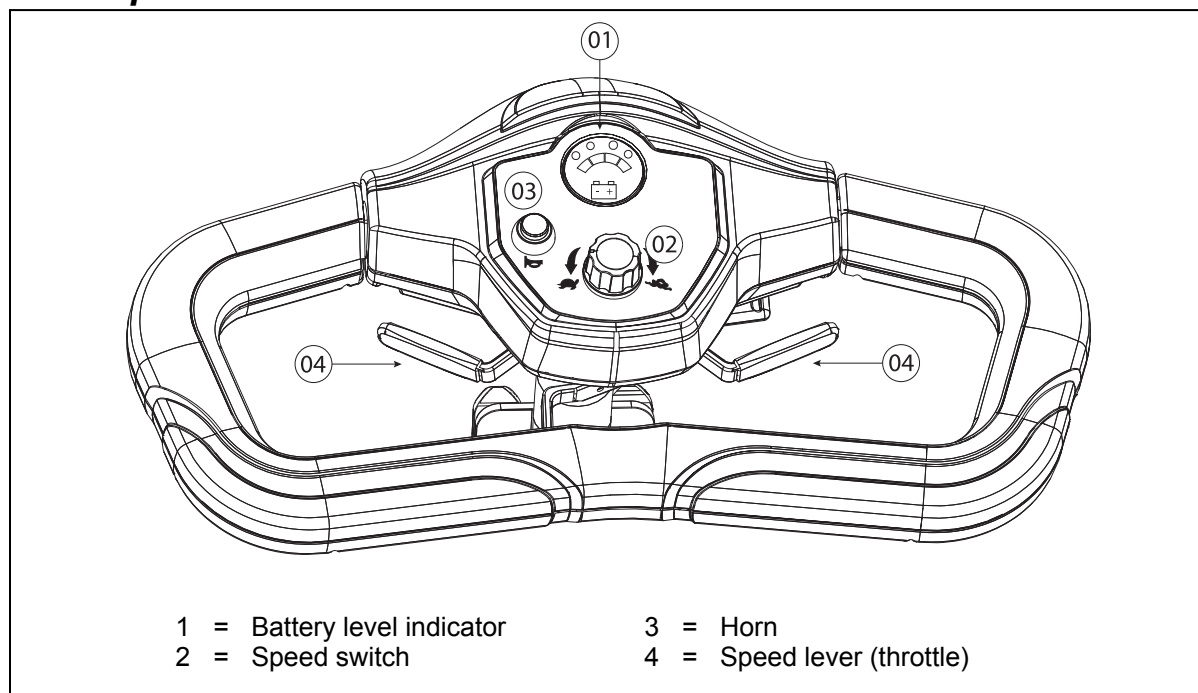


- Turn the ON/OFF key switch (right side operator control) on, to turn the power on. Turn the ON/OFF key switch off, to turn the power off.
- Now the battery level indicator ① will light up and shows the current level of your batteries.

The battery indicator on the operator control uses a color code to indicate the approximate power remaining in your batteries. Green indicates (40 ~ 100 %) capacity, yellow a draining charge (10 ~ 30 %), and red indicates that an immediate recharge is necessary.

The battery capacity depends on the driving time incurred and how you drive with your scooter. Repeated starting, stopping, climbing will consume the power more quickly.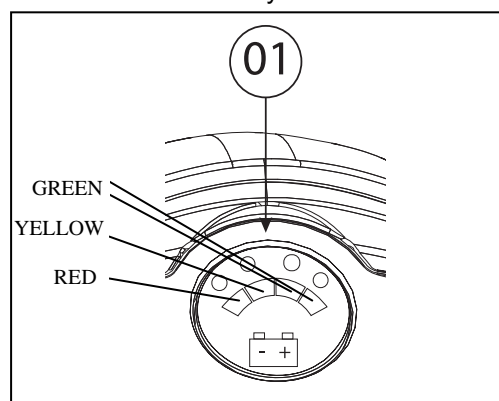

- Now turn the speed switch ② to the desired driving speed. Do not adjust the speed while driving, a sudden change in speed may cause danger to you and others or may damage the scooter.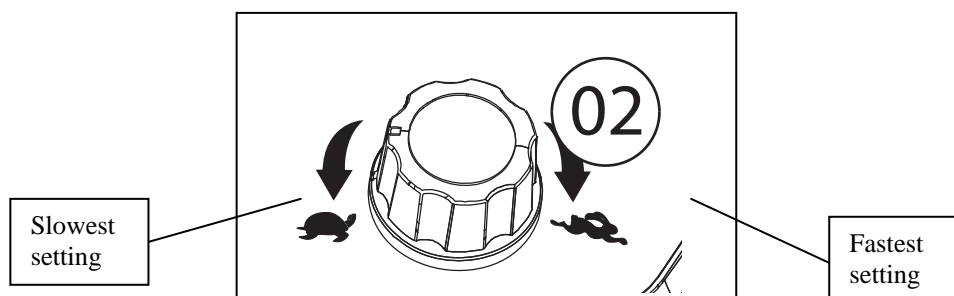

- Grasp the handgrips.
- Pull the drive lever with your fingers toward the handgrips (right lever for forward movement, left lever for backward movement).
- To sound the horn, press the horn button ③. Release the button to stop the horn.
- To stop the scooter loosen the speed lever ④ under the operator control.

Vermeiren is responsible for the changes in the software. For changes in the software contact Vermeiren.


2.11 Neutral

- ⚠ WARNING: Control your scooter - Never put your scooter in neutral while you are driving.**
- ⚠ WARNING: Control your scooter - Never put your scooter in neutral on slopes. It may accidentally roll away.**
- ⚠ WARNING: Risk of injury - Never pick up the electronic drive before stopping the motor/gears to prevent motor from overheating.**

The scooter is fitted with a freewheel device that is accessible and operable by the attendant or the user when not sitting in the scooter. These device shall be used as a parking brake. You should only use the scooter in neutral to transport the scooter or to move it out of a danger zone.

Put the scooter in neutral with the motor stop lever ① on the back side of the scooter.

- Driving
 1. Put the motor stop lever ① on drive. This will again interlock the motor and gears.
 2. Switch the ON/OFF key ON.
 3. Electronically controlled driving is now possible.
- Neutral
 1. Switch the ON/OFF key OFF.
 2. Put the motor stop lever ① on neutral (see label). This separates the motor from the gears.
 3. The scooter can now be pushed without electronic drive.

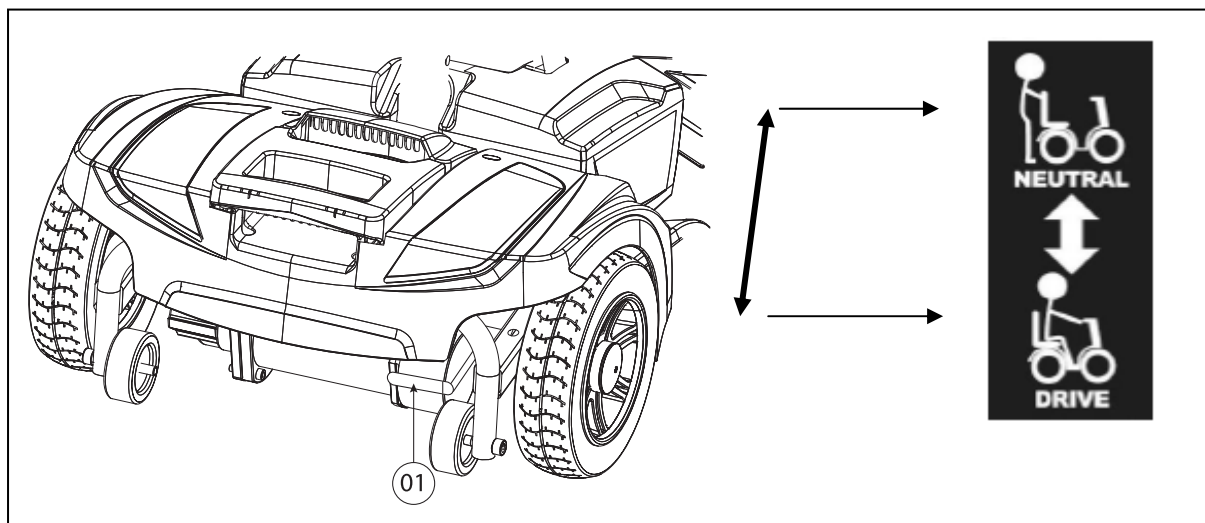2.12 Transport in the car

- ⚠ DANGER: Risk of injury - The scooter is not suited for use as a seat in a motor vehicle.**
- ⚠ WARNING: Risk of injury - Remove all loose parts prior to transportation.**
- ⚠ WARNING: Risk of injury - No people or objects should be under the scooter, on the footplate or seat during transportation.**
- ⚠ WARNING: Risk of injury - See that the scooter is attached properly. So you can avoid injury from the passengers during collision or sudden braking.**
- ⚠ WARNING: Chance of pinching - Do not place fingers between the components of the scooter.**

The best way to transport your scooter fully assembled in the car is to drive the scooter in the car by using ramps.

When you are not experienced to drive the scooter by using of ramps you can also put the scooter in neutral mode and push the scooter in the car by using ramps.

Attach the frame of the scooter securely to the vehicle.

When the scooter does not fit in the car it is also possible to transport the scooter by the following steps:

1. Remove all loose parts prior to transporting (seat + backrest, battery housing, shopping basket).
2. Store loose parts safely.
3. Fold the steering column down with the angle adjustment.
4. Remove the rear frame from the front frame.
5. Place the separate parts in the car and attach them securely to the vehicle if necessary.

2.13 Automatic Power shut down

In order to avoid accidental battery run down, your scooter is equipped with an automatic power shut down facility. If the scooter is switched on, after remaining undisturbed for a period of thirty minutes it will automatically turn off. Should this occur, simply switch your scooter off and back on and it will be ready to use once again.

2.14 Charging the batteries

⚠ WARNING: Risk of injury - Take out the key before you start charging.

⚠ WARNING: Risk of fire - Keep away from flammable objects while charging as it may lead to fire of battery.

⚠ WARNING: Risk of electrical shock - Never connect or disconnect the plug or cable with wet hands while charging.

Fully recharge your new battery before its first time use.

The light emitting diode (LEDs) in the steering column indicates the remaining capacity of your batteries. If the battery charger is turned on and no LED's are lit, check the fuse. If the red LED does not illuminate the charger is defective, consult your specialist dealer.


Recharge the batteries on a daily basis and in a well-ventilated space, not directly exposed to sunlight. Do not charge in surroundings where it is humid or under rainfall and morning dews. If you do not do that and you keep on driving, the output of your scooter will drop very significantly (uphills, bends, dim headlights). If you ignore these signals too, your scooter will turn off. You should now immediately recharge your scooter with the accompanying battery charger. Be sure that you charge the battery fully every time.

After charging or replacing a new battery, drive the vehicle for 2-3 minutes to make sure the battery capacity is sufficient.

The battery capacity will vary by how you drive with your scooter (repeated starting, stopping, climbing, rough terrain) and will consume the capacity more quickly. In wintertime, the battery may respond more slowly and the battery range may be reduced.

Kindly also observe the accompanying operating instructions.

Primary	230V ~ / 50-60 Hz / 270 W max.
Secondary	24 V $\overline{\text{---}}$ / 1,8A max.
Model	4C24020A
Max. battery cap.	12 Ah
Charging	Charge 80% cap. within 8 hrs

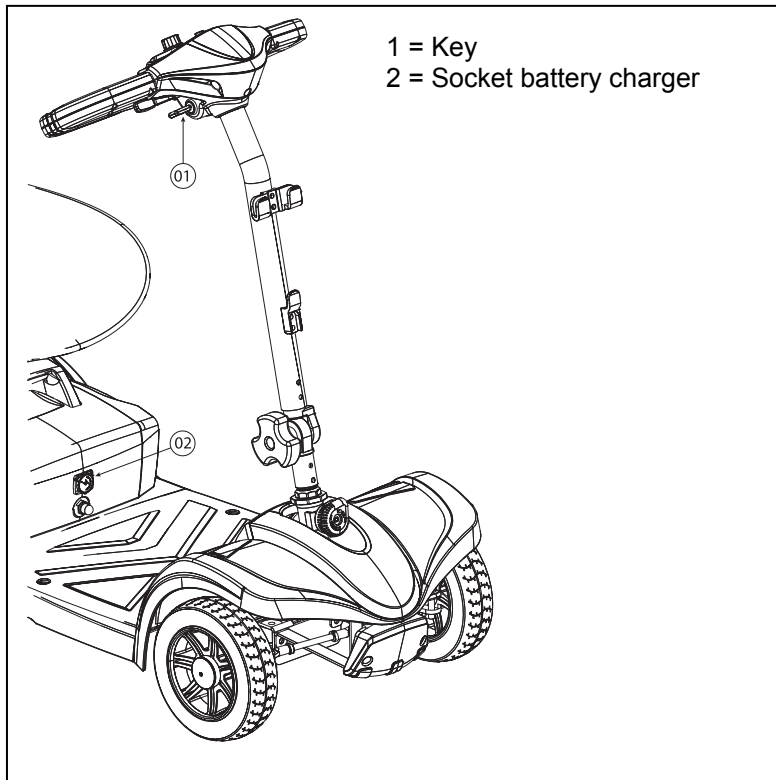1. Turn the key ① to OFF and take it out.
2. Open the protecting flap of the charging socket ②.
3. Insert the plug of the charger into the charging socket of the scooter ②.
4. Insert the mains plug of the charger into the power socket. Switch the charger to ON (a few models have no ON/OFF switch; with these the loading device starts operating automatically when you insert the mains cable).
5. The charger now starts to charge and the LED (red and orange) will be lit, which indicates that it is busy charging. While charging your scooter it is not possible to run, drive your scooter because the immovable function will be activated.

6. Once the charging process is complete, the LED (orange) turns green, thereby indicating that loading is complete. The charging duration is about 6 hours. To ensure optimum performance a 10-hour charge is recommended. But we do not recommend a charging more than 24 consecutive hours.
7. First switch off the charger (if there is no ON/OFF switch, pull out the mains plug).
8. Pull the charger out the power socket.
9. Pull out the charger plug from the charging socket of the scooter. Your scooter is again ready for use.
 - Charge your scooter's batteries strictly in accordance with the description above. If you charge the batteries too early, they will gradually lose capacity, thus cutting your scooter's traveling range.
 - Always complete the charging until the orange LED light turns green. NEVER stop charging before it is complete.
 - The manufacturer shall decline all liability for damage caused by incorrect charging.
 - Use only genuine original batteries. We do not accept any liability for damage caused by using batteries not supplied by us.
 - Do not expose the batteries to temperatures below -10° Celsius or above 50° Celsius.
 - If the batteries are opened, all liability of the manufacturer is voided as well as any claim.
 - After charging do not leave the charger socket plugged into the scooter, as this will cause a power drain on the scooter and temporarily reduce its range.
 - Do not smoke while charging as the battery may release hydrogen gas.

If you decide not to use your scooter for a prolonged period, you must nevertheless recharge it regularly to keep it in a running condition ready for immediate use.

- If the batteries are not being used for a prolonged period, they will discharge slowly by themselves (in-depth discharging). Then it becomes impossible to recharge them with the supplied battery charger. When batteries are not in use, they must be recharged at least every week (depending on the charge indicator).
- Note that if you recharge the batteries too frequently, they will eventually lose their capacity irretrievably.
- Use only the supplied battery charger and no other charging equipment.
- The manufacturer shall decline all liability for damage caused by improper charging.
- In every case the charging cycle must not be interrupted. The battery charger has an indicator showing you when the charging cycle has been completed.
- Do not charge any other electrical equipment with your scooter's battery or battery charger.

3 Installation and adjustment

The instructions in this chapter are for the user and the specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

⚠ WARNING: Risk of tipping over - Variation allowed adjustments can still change the stability of your scooter (tilt back or sideways).

3.1 Tools

To set up the scooter the following tools are needed:

- Screwdriver
- Wrench set n° 10
- Allen keyset n° 6

3.2 Manner of delivery


The scooter shall be delivered with:

- Frame with armrests, steering column, rear and front wheels, anti-tipping wheels
- Footplate
- Seat + backrest
- Batteries, motor (2x)
- Shopping basket
- Accessories
- Manual

3.3 Mounting / removing rear frame

⚠ WARNING: Risk of injury - Be careful that you do not pinch your fingers, hands between the frame.

⚠ CAUTION: Risk of injury - Check that the front and rear frame is fixated properly.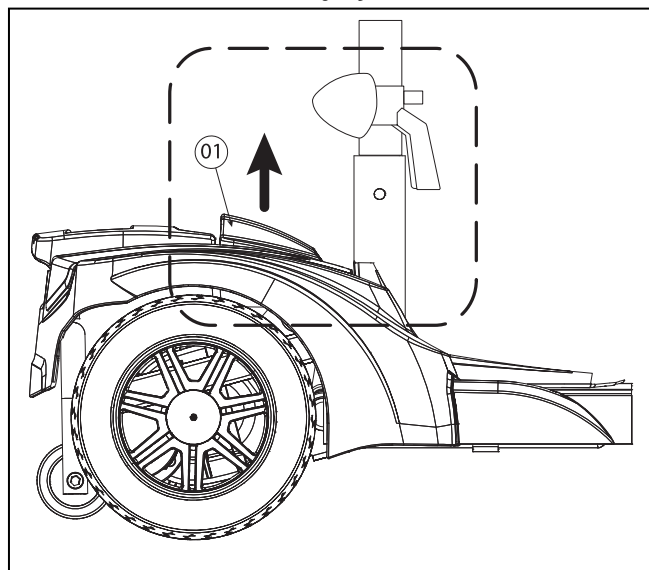

Removing rear frame:

1. Pull the locking plate ① upwards.
2. Move the front frame a little bit backwards by holding the locking plate ①.
3. Slide the front frame out the rear frame by holding the locking plate ①.


Mounting rear frame:

1. Grasp the locking plate located ① on the front frame.
2. On the front frame and rear frame are red lines (marks) check that these are on the same location (tubes of the front frame must be mounted in the hooks of the rear frame).
3. Slide the front frame in the rear frame by holding the locking plate ①.
4. Push the locking plate ① downwards until the two frames clicks in each other.
5. Check that the frames are mounted very well in each other.

3.4 Adjusting the steering column


The steering column can be adjusted in many different angles (stepless) to suit each driver.

- ⚠ WARNING: Risk of injury - Never adjust the angle while you are driving.**
- ⚠ WARNING: Risk of injury - Turn the scooter off before you adjust the angle of the steering column.**
- ⚠ WARNING: Risk of injury - Do not lean with your whole body on the steering column.**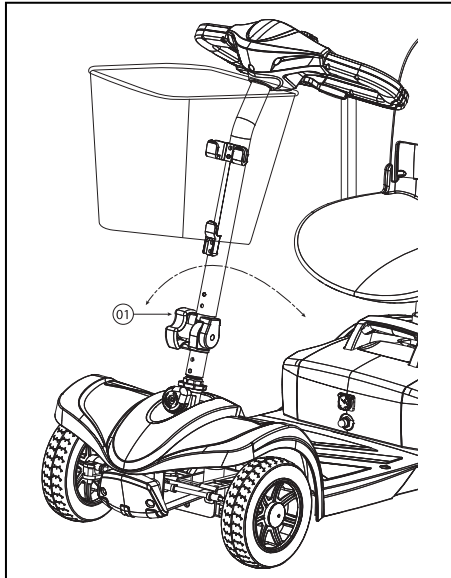

1. Loosen the star knob ①.
2. Adjust the steering column in the desired position.
3. Tighten the star knob ① to secure the steering column.

3.5 Mounting / removing the shopping basket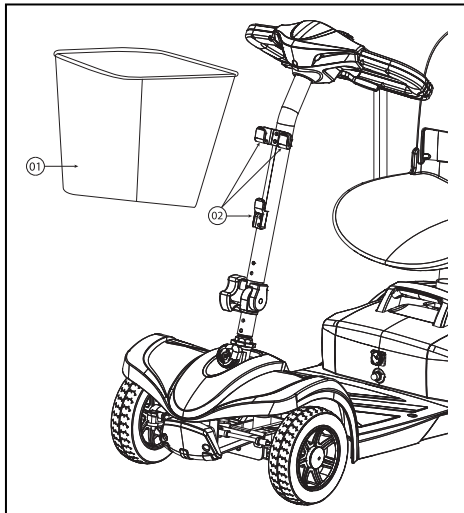

Mounting shopping basket:

1. Grasp the shopping basket ①.
2. Mount the shopping basket ① with the holes on the hooks of the attachment plate ②.
3. Check the shopping basket ① is mounted very well.

Removing shopping basket:

1. Grasp the shopping basket ①.
2. Pull the shopping basket ① upwards until it comes out the hooks of the attachment plate ②.

3.6 Adjusting the seat

- ⚠ WARNING: Risk of injury - Never perform adjustments while you are driving.**
- ⚠ WARNING: Risk of injury - Make sure that the seat is locked securely.**

Remove seat (Fig. A)

1. Pull the seat lever ① upwards.
2. Turn the seat ② a little bit and lift the seat out the adjustment bar ③.
3. Release the seat lever ①.


Lock the seat in place (Fig. A)

1. Pull the seat lever ① upwards.
2. Place the seat ② on the adjustment bar ③ and at the same time keep the seat lever ① pressed in.
3. Release the seat lever ①.
4. Check that the seat ② is firmly secured.

Swivel seat (Fig. A)

1. Pull the seat lever ① upwards.
2. Turn the seat ② in the desired direction.
3. Release the seat lever ①.
4. Swivel the seat again until it locks in position.
5. Check that the seat ② is firmly secured.

Fig. A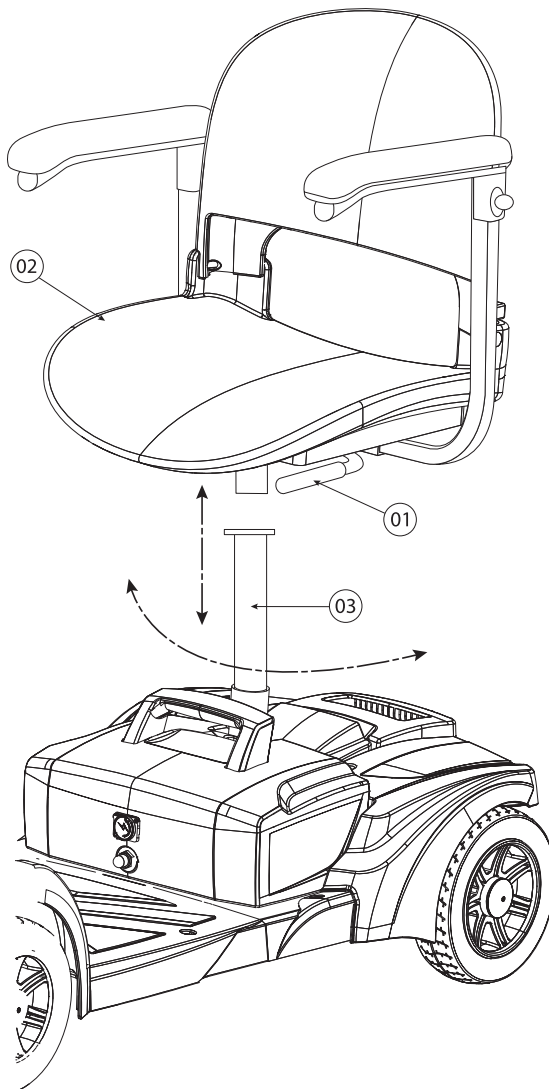



- 1 = Seat lever
- 2 = Seat
- 3 = Adjustment bar

Seat height adjustment (Fig. B)

The seat can be adjusted in 3 different seat heights (increments: 25 mm).

1. Remove the seat from the adjustment bar ③.
2. Remove the screw ②.
3. Move the adjustment bar ③ upwards/downwards in the receiver, and place these in a comfortable seat height.
4. Replace the screw ②.
5. Check the adjustment bar ③ is locked securely.
6. Replace the seat on the adjustment bar ③.

Fig. B

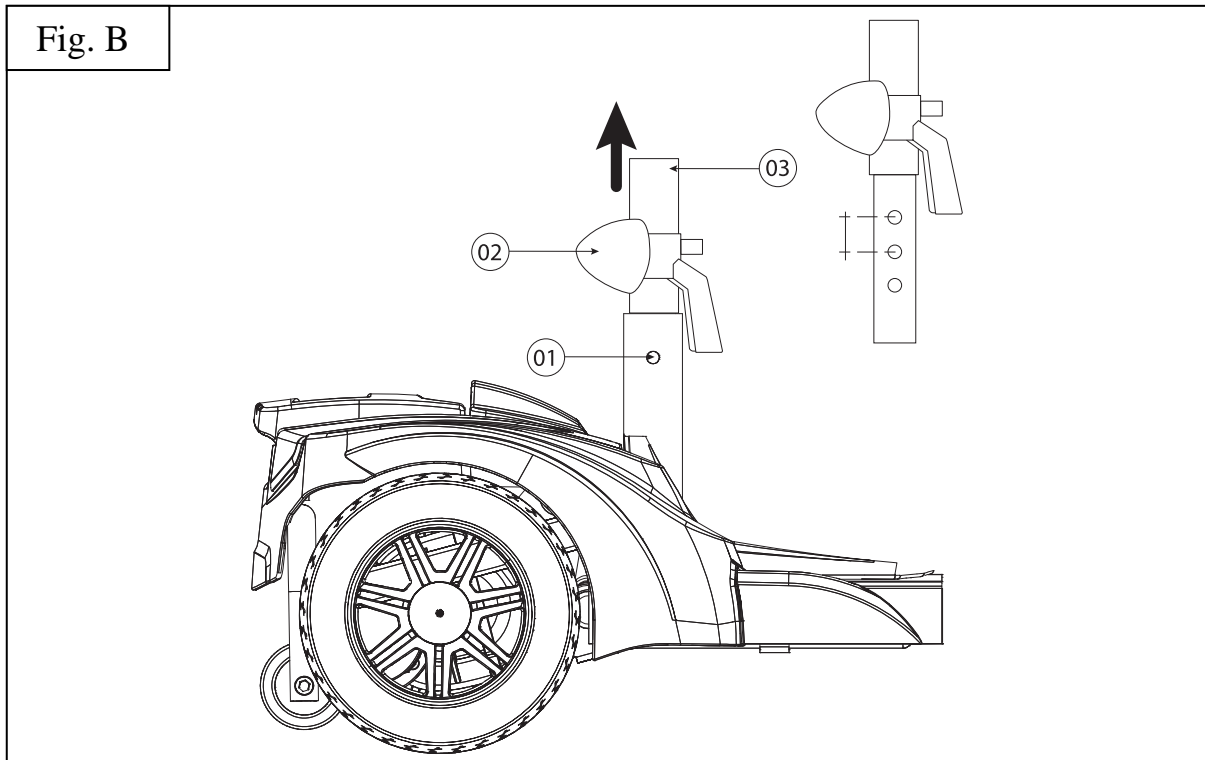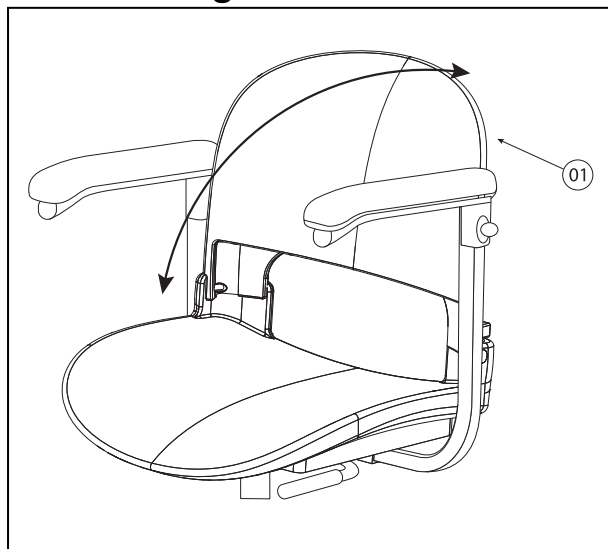**3.7 Folding the backrest**

⚠ WARNING: Risk of injury - Never perform adjustments while you are driving.

⚠ WARNING: Risk of injury - Be careful you do not pinch your fingers at the rear of the backrest when pulling the backrest in the upright position. see label:


Push the backrest ① forward to fold the backrest to the front.

Pull the backrest ① backwards to place the backrest in the upright position.

3.8 Adjusting the armrests

- ⚠ **CAUTION:** Risk of tipping over - Make sure that the armrests are positioned symmetrically from the seat.
- ⚠ **CAUTION:** Risk of injury - Mount the inner tube always with of safe distance of 50 mm in the outer tube.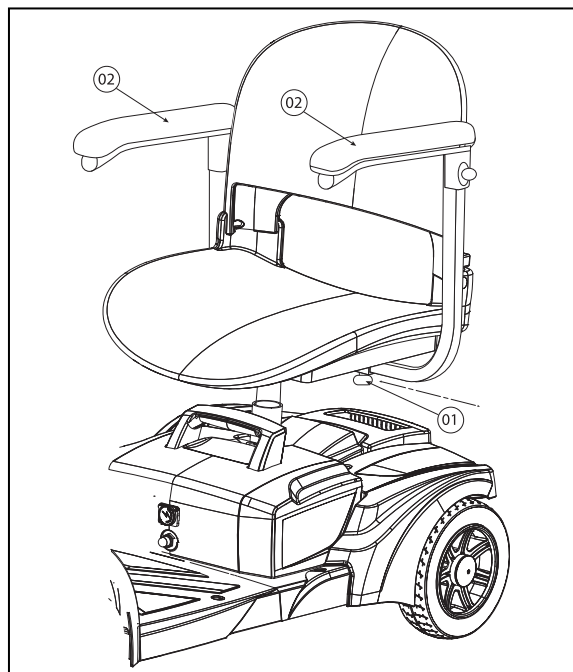


The armrest can be adjusted in width:

1. Loosen the star knob ① on the back of the scooter.
2. Grasp the armpad.
3. Move the armrest ② to the desired width (range: 100 mm each side stepless).
4. Retighten the star knob ①.
5. Check that the armrest is fixated very well.

3.9 Locking steering wheels

- ⚠ **CAUTION:** Risk of injury - Put the steering wheels back on unlocked when you want to drive with your scooter.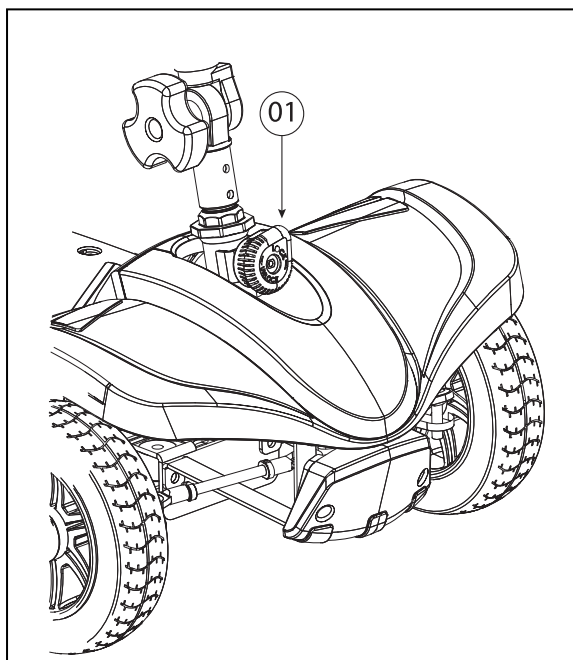


When you want to place the scooter fully assembled in the car you can lock the steering wheels.

1. Pull the locking knob ① out (locking knob is located on the bottom of the steering column).
2. Turn the locking knob ① in the locked position.

When you want to drive with your scooter you **MUST** first unlock the steering wheels.

1. Pull the locking knob ① out.
2. Turn the locking knob ① in the unlocked position.
3. Now you can turn the steering wheels.

3.10 Thermal fuses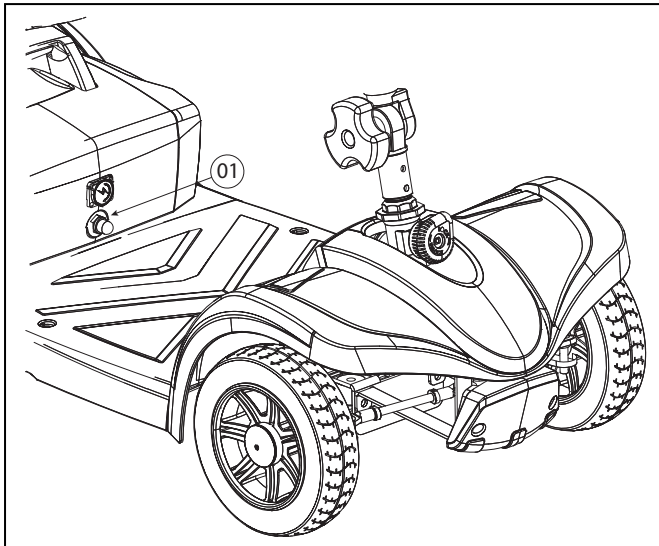


To protect the motor against overload, your scooter have a thermal safety mechanism ① on the front side of the battery housing. The thermal fuse will automatically cut the motor to prevent overheating and thus rapid wear and tear or breakdowns. You can access the thermal safety device through a gap in the back of the synthetic cover.

The thermal safety device can go loose if you go up or down slopes that exceed the maximum gradient indicated. Nominal loads that exceed the maximum permitted could also cause the device to jump out. Likewise, if you keep on driving with the engine brake on, it could result in overload. The maximum values not to exceed are indicated in the chapter “Technical Specifications” of the corresponding manual.

To be able to use the scooter again, remove the overload and wait till the motor has cooled off. Then gently press the thermal safety mechanism back in. Your scooter is now ready for use again.

3.11 Anti-tipping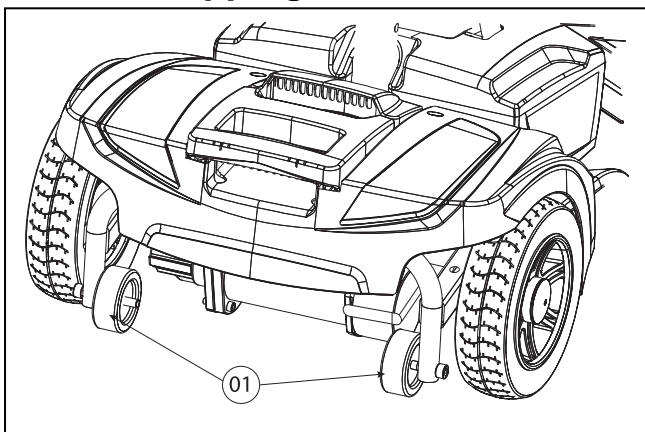

An anti-tipping ① is fixed to the frame of the scooter on the back side. It is not possible to remove these. The anti-tipping is there for your safety. It prevents the scooter from tipping over backward when you drive over small obstacles that do NOT exceed the maximum height specified.


3.12 Changing the batteries

- ⚠ **CAUTION:** Risk of burns - Do not come in contact with the acid from the batteries. See for a good ventilation of the battery compartment.
- ⚠ **CAUTION:** Risk of injury - Be aware for fingers or hands, while mounting the battery housing in the scooter.

The battery shall be changed by trained personel.

The wiring and charger are well situated in the scooter. Do not attempt to re-locate the wiring by yourself. The improper installation of the wiring may result in pinching the wiring between the battery box, which cause failure to the electronic system of your scooter.

Be sure the battery cables are connected to the right battery.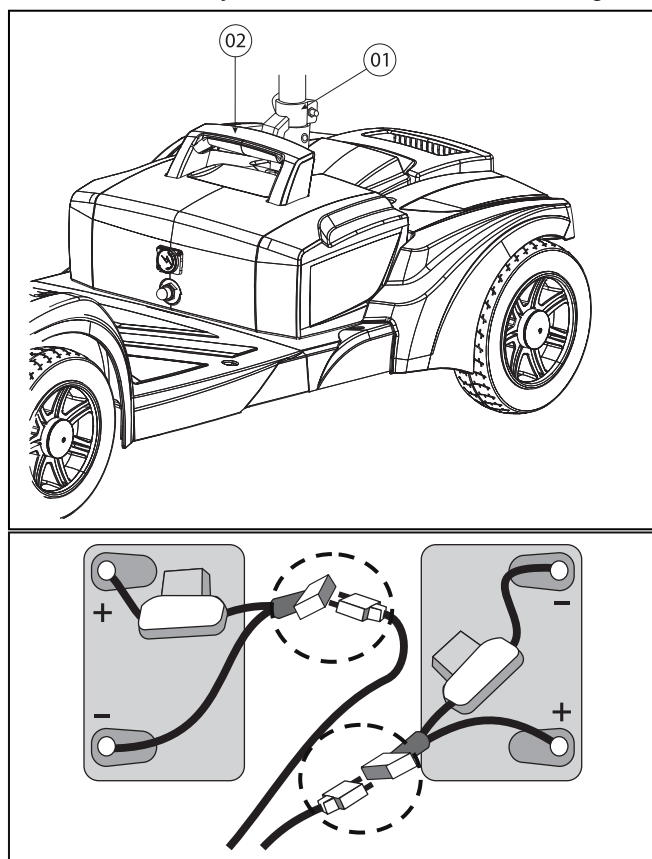


1. Unlock the battery lock part ① with the knob .
2. Rotate the battery lock part ①.
3. Lift the battery housing ② out the scooter with the handgrip on the battery housing ②.
4. Loosen the battery housing ② by loosen the 8 screws on the bottom side of the battery housing ②.
5. Disconnect the plugs from the battery cables.
6. Change the batteries.
7. Connect the plugs to the correct battery.
8. Check that the cables are installed properly and replace both batteries together.
9. Close the battery housing and retighten the 8 screws.
10. Mount the battery housing ② back in the scooter.
11. Lock the battery lock part ① to unlock the battery housing in your scooter.
12. Check the battery housing ② is firmly secured in your scooter.

4 Maintenance

For the maintenance / cleaning instructions refer to the separate maintenance manual of the scooters. Refer for these separate manual to your specialist dealer.

4.1 Troubleshooting Fault codes

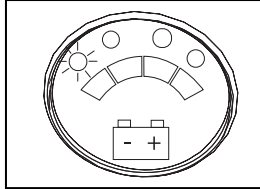The flashes on the battery indicator shows the fault codes. The number of flashes (LED CODE) provides the information below (the following actions and checks should only be carried out by authorised individuals). The LED CODE exist out two numbers, the LED flashes in an interval of the first number, followed by the interval of the second number. These sequence shall be repeated until the fault is solved. (for example: LED CODE: 3,4 → 3 flashes followed by 4 flashes).

LED CODE	PROBLEM / FAULT	SOLUTION
1,1	Over- / under temperature	1. Cool down your scooter. 2. Remove excessive loads. 3. Do not drive in extreme environments.
1,2	Throttle	Contact your dealer.
1,3	Speed limit switch	Contact your dealer.
1,4	Battery voltage too low	1. Charge the battery 2. Check connections of battery or controller.
1,5	Battery voltage too high	1. Replace battery. 2. Check that the charger is not attached to the scooter.
2,1	Controller	Contact your dealer.
2,3	Controller	Contact your dealer.
2,4	Controller	Contact your dealer.
3,1	Wiring	Contact your dealer.
3,2	Magnetic brake goes not open	Contact your dealer.
3,3	Magnetic brake	Contact your dealer.
3,4	Magnetic brake in neutral	Place neutral lever in drive.
3,5	Throttle out neutral position	1. Loosen the throttle. 2. Contact your dealer.
4,1	Current motor too high.	1. Switch scooter off and cool down the motor.
4,2	Current motor too low	Contact your dealer.
4,3	Memory fault powermodule	Contact your dealer.
4,5	Battery cables disconnected	1. Check battery cables and connection points

5 Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15

2920 Kalmthout

Belgium

declares under his sole responsibility that the CE marked devices :

Productgroup: Scooters

Productgroup (GMDN): Wheelchair, attendant / occupant, manual / steer, collapsible
(GMDN 40855)

FAMHP (FAGG) registration

number: BE/CA01/11/2-02965-22-CLI

Brand: Vermeiren

Type: Venus 3 (T3KC), Venus 4 (T4KC)

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 12,

***and is manufactured in full conformity with the European instructions below -
including the latest modifications - and with the national law, that organizes
this directions :***

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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N.V. VERMEIREN N.V.
Vermeirenplein 1/15
B-2920 Kalmthout
BELGIUM

Notes

A series of horizontal dotted lines for writing notes.

Notes

A series of horizontal dotted lines for writing notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Reatime S.R.L.

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.reatime.it
e-mail: info@reatime.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nadrazni 132
702 00 Ostrava 1
Tel: +420 596 133 923
Fax: +420 596 133 277
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

L. Vermeiren Ges. mbH

Winetzhammerstraße 10
A-4030 Linz
Tel: +43(0)732 37 13 66
Fax: +43(0)732 37 13 69
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Hühnerhubelstraße 59
CH-3123 Belp
Tel: +41(0)31 818 40 95
Fax: +41(0)31 818 40 98
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5
Sant Gregori Parc Industrial Edifici A
17150 Sant Gregori (Girona)
Tel: +34 972 42 84 33
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es