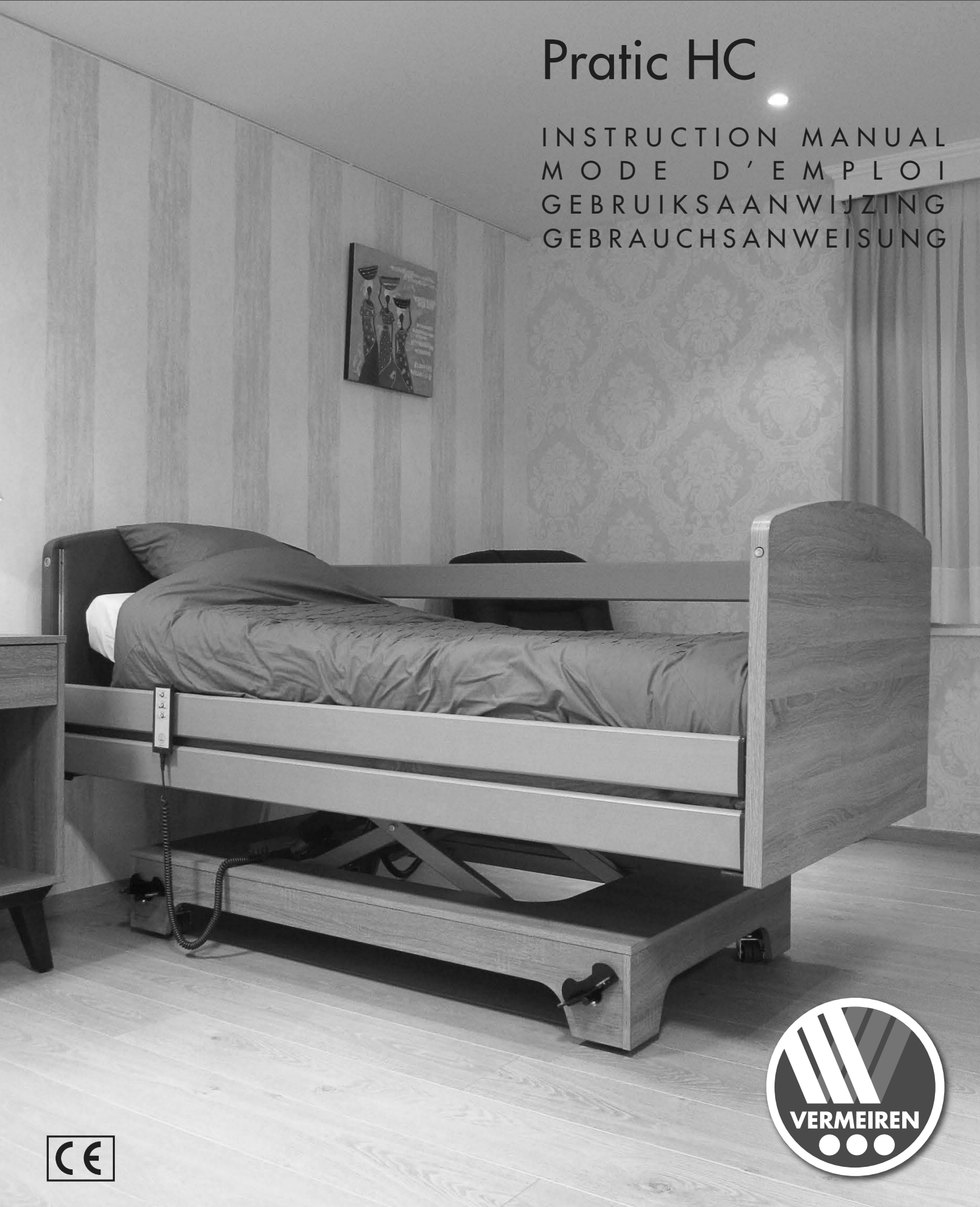
# VERMEIREN

## Pratic HC

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EN

**Instructions for specialist dealer**

This instruction manual is part and parcel of the product and must accompany every product sold.

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Version: B, 2016-08

**All rights reserved, including translation.**

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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## Contents

<b>Preface</b> .....	<b>2</b>
<b>1 Product description</b> .....	<b>3</b>
1.1 Intended Use.....	3
1.2 Technical specifications .....	4
1.3 Components.....	5
1.4 Explanation of symbols .....	5
1.5 Accessories.....	6
1.6 For your safety .....	6
<b>2 Use</b> .....	<b>7</b>
2.1 Handset with lock function .....	7
2.2 Potential equalization .....	8
2.3 Rastomat.....	8
2.4 Operating the steering casters .....	8
2.5 Strain relief .....	9
2.6 Power cable .....	10
2.7 Emergency lowering of the headrest .....	10
2.8 Side railings.....	11
2.9 Lifting pole (as option).....	12
2.10 Transfer in and out the bed.....	13
2.11 Transport .....	13
2.12 Cable placement .....	14
2.13 Remarks on electromagnetic compatibility (EMC) .....	15
<b>3 Installation and adjustment</b> .....	<b>16</b>
3.1 Tools .....	16
3.2 Manner of delivery .....	17
3.3 Assembly.....	17
<b>4 Maintenance</b> .....	<b>19</b>

## Preface

First of all we want to thank you for putting your trust in us by selecting one of our products.

The Vermeiren beds are the result of many years of research and experience. During the development, special attention was given to the ease of use and the serviceability of the bed.

The expected lifetime for your bed is strongly influenced by the care and maintenance of the bed.

This manual will help you get acquainted with the operation of your bed.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

## 1 Product description

### 1.1 Intended Use

With the Pratic model, you have acquired a nursing bed that was developed entirely for your comfort. This nursing bed offers a number of individual adjustment possibilities which are explained below.

The nursing bed was designed for dry indoor use on level, solid floors.

The nursing bed is designed for a single adult patient (from 12 years, up to maximum user weight of 135 kg).

The bed shall be used with a suitable mattress (min. RG 35, 120 mm thick, max. 20 kg).

The complete kit can be used by people who have been disabled by

- Paralysis
- Loss of limbs (leg amputation)
- limb defects / deformations
- Stiff or damaged joints
- Illnesses like cardiovascular insufficiency, vertigo or cachexia, and also for geriatric patients.

When providing for individual requirements:

- body size and weight (max. 135 kg)
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

The bed is suited for application environment 4 of ISO 60601-2-52: "care provided in a domestic area where ME equipment is used to alleviate or compensate for an injury, disability or disease". It is to serve solely as a nursing bed to be used in home care.

Your bed should only be used on surfaces where all four wheels are touching the ground.

It is not to serve as a bed for children (patients younger than 12, or shorter than 1.46 m), a means of transport, a ladder, storage for heavy or hot objects.

The bed might damage rugs, carpets and other loose flooring. The bed (always without patient!) can only be moved with extra exertion on such flooring.

Use only Vermeiren approved accessories. It is not permitted to combine the bed with other products or equipment or other electrically driven appliances. Using the bed in combination with other products, components or electrically powered devices is only allowed with permission from the manufacturer.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the guarantee conditions. A guarantee can only be given when the product is used under the circumstances specified above and for the purposes intended.

Visually impaired people can contact the dealer for the instructions for use.

## 1.2 Technical specifications

(Given in fully assembled condition)

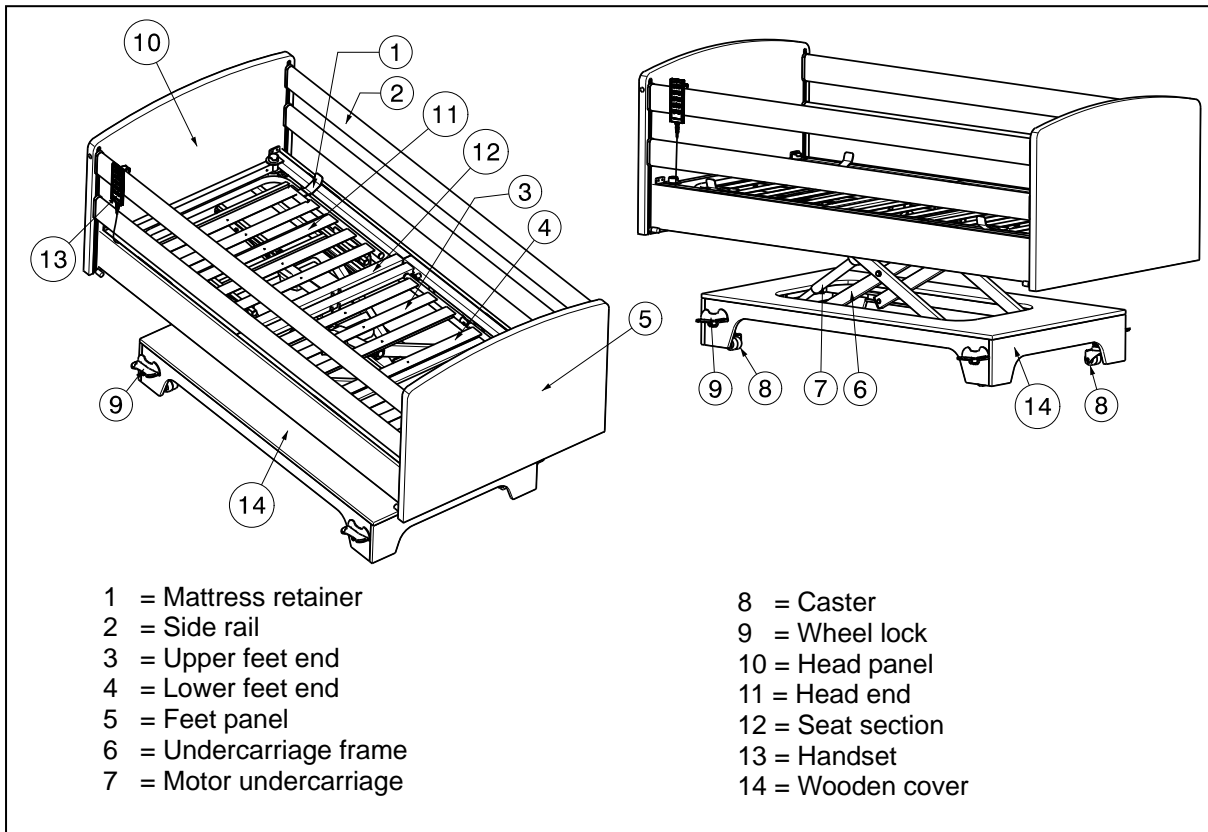
<b>Make</b>	Vermeiren		
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout		
<b>Type</b>	Bed		
<b>Model</b>	Pratic HC		
Description	Min.	Max.	
<b>Maximum occupant mass</b>	<b>Nominal load 135 kg</b>		
Length	2080 mm		
Height	910 mm - 1330 mm		
Width	1040 mm		
Height adjustment of the slatted frame	Continuously from 390 to 810 mm (without mattress)		
Total mass (excl. side rails, lifting pole and triangle)	98 kg		
Adjustment of the head end	70°		
Max. thigh angle	30°		
Max. angle for lower leg	-16°		
Max. safe working load *	170 kg		
Lifting pole	Nominal load 80 kg		
Mattress	Min. RG 35, thickness 120 mm, 2000 x 900 mm, max. 20 kg		
Under clearance for patient hoist	150 mm		
Steering casters (4 pieces)	Ø 50 mm, with central brake		
Motor undercarriage	Linak LA27		
Head end motor	Linak LA27		
Feet end motor	Linak LA27		
Handset	Linak HB7x, IPX4		
Transformer	Linak CB6		
Nominal voltage, transformer	U input: 230 V~, ± 10%, 50 Hz, current: max. 1.5 A, output voltage: 24V ---, max. 70 VA		
Period switched on	10%, Maximum two minutes then 18 minutes for non-use		
Noise level	~ 48 dB(A)		
Class / Kind of protection	Protection class II, application part Type B / IPX4 (spray protection)		
Storage and use temperature	+ 5 °C	+ 40 °C	
Storage and use humidity	30%	70%	
<b>We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / °</b>			
<b>* Max. safe working load = mass user weight + mass mattress + mass other accessories</b>			

*Table 1: Technical specifications*

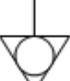
The bed complies to the requirements set up in:

EN 1970:	Adjustable beds for disabled persons: Requirements and test methods
EN 1970+ A1:	Adjustable beds for disabled persons: Requirements and test methods
EN 60601-2-38/A1:	Medical electrical equipment - Part 2-38: Particular requirements for the safety of electrically operated hospital beds

### 1.3 Components

### 1.4 Explanation of symbols

-  Maximum mass
-  CE conformity
-  Safety instructions
-  Observe the instructions for use
-  Protection class II
-  Type B application
-  Type designation
-  Potential equalization (connection)

## 1.5 Accessories

**⚠ WARNING: Precautions shall be taken if accessories not listed below are used.**

- Lifting pole

## 1.6 For your safety

- ⚠ Carefully read the instruction manual before using the nursing bed.
- ⚠ Place and use your nursing bed only on flat and level surfaces and ensure that all four steering castors stand safely and securely on the floor.
- ⚠ When the bed is placed in position, make sure that no obstacles are present inside the entire adjustment range.
- ⚠ If the patient is not supervised, the bed must be adjusted beforehand to the lowest position to reduce the danger of injury by falling when getting into, getting out of, or lying on the bed.
- ⚠ If the patient is not under supervision, the side-rail on both sides should be raised to the highest position. The railings may only be lowered on one or both sides when a supervisor is present.
- ⚠ Watch your hands when assembling, disassembling or using the bed -- danger of squeezing!
- ⚠ The bed is not appropriate for transporting people.
- ⚠ A patient may only be conveyed in the nursing bed when lying in the lowest horizontal position.
- ⚠ When transporting the bed, grasp it only by solid frame parts and wooden parts.
- ⚠ All four steering castors must always be locked before using the nursing bed.
- ⚠ Medical-electrical treatments are forbidden inside the nursing bed!
- ⚠ Do not lean against or on the side railings -- Risk of accidents!
- ⚠ Avoid leaning out sideways with the upper body -- Risk of accidents!
- ⚠ Do not rest your legs on other objects -- Risk of accidents!
- ⚠ When adjusting hinges, joints or other clamps, make sure that no body parts or objects are present -- Risk of squeezing!
- ⚠ Make sure that no extremities are caught in the frame when adjusting its height. Danger of being crushed!
- ⚠ Children need to remain at a safe distance from the bed.
- ⚠ Do not place your hands between the frame and the mattress when adjusting leg or back rests -- Risk of squeezing!
- ⚠ Use only power sockets conforming to VDE (220 V / 230 V, 50 Hz) for plugging in.
- ⚠ Plugs must only be removed by grasping the plug itself; never pull on the cable.
- ⚠ Lock manual operation with the lock function on the manual control each time after use.
- ⚠ The nursing bed must only be used in dry rooms.
- ⚠ Only use mattresses which comply to the prescriptions of the bed manufacturer.
- ⚠ If the lifting pole is damaged, it should be replaced.
- ⚠ The side railings must always be installed even if the relevant side is situated against the wall.
- ⚠ Do not undertake repairs on your own -- risk of accidents! (Have a qualified or authorized person carry out repairs).
- ⚠ Do not open any housings of the driving system -- risk of accidents!
- ⚠ Transportation of the bed shall be only without the patient.
- ⚠ Make sure that all power cables are installed without kinks, shearing or other mechanical damages.
- ⚠ All cables should be wired without making contact to the floor and without any kinks.
- ⚠ No additional mechanical or electric devices may be used or installed on the bed.

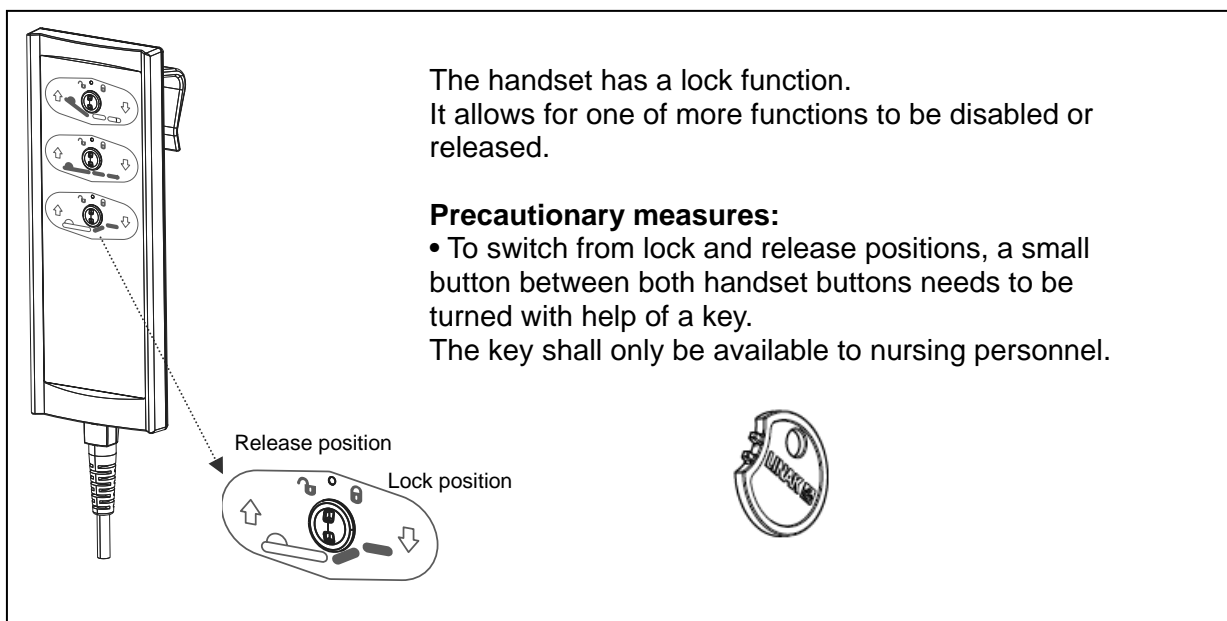
- ⚠ Servicing, inspection, repairs and disinfection should only be undertaken by authorized, trained persons.
- ⚠ Use only genuine replacement parts of the manufacturer for your nursing bed.
- ⚠ Do not exceed the maximum load as stated by the manufacturer.
- ⚠ No open flames or smoldering objects are allowed near the bed (candles, cigarettes, etc.).
- ⚠ Replace the side railings if they are bent or broken, otherwise there is a risk of the patient falling out of the bed.
- ⚠ Use of your bed could influence other electrical devices (e.g. disruptive noises). Avoid using such devices simultaneously.
- ⚠ Obey all safety and operational instructions.
- ⚠ Electrically operated bed is placed with a distance to the power supply plug.
- ⚠ The medical bed shall only be used with a hoist that can be positioned under the limited space of the medical bed.

## 2 Use


This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**

The bed is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the bed are given in § 3.

- ⚠ **WARNING: Risk of electrical shock - Ensure that no cables become squeezed.**
- ⚠ **WARNING: The medical bed shall only be used with a hoist that can be positioned under the limited space of the bed.**


Following adjustments can be used:
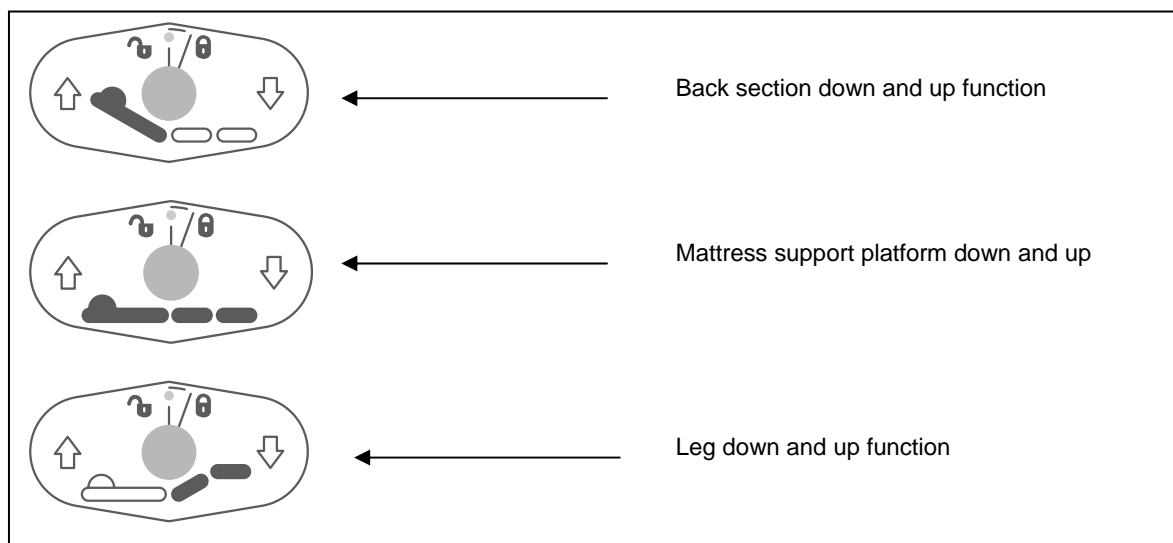
## 2.2 Potential equalization

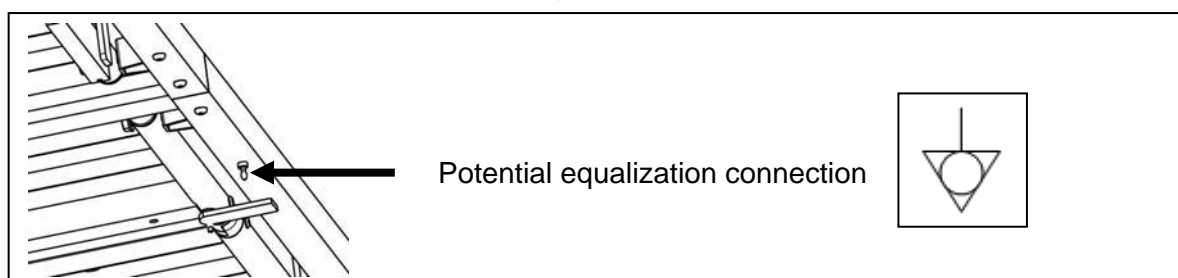A potential connection is located under the mattress frame. This connection is used to switch conducting components to the same potential, should electrical components be used as part of the bed. Connect the potential equalization to the outgoing independent grid. Follow the instructions for the additional electrical components.

⚠ Make sure that the cable connections are fastened tightly.

## 2.3 Rastomat

⚠ **CAUTION: Risk of injury - No people or objects allowed close to where adjusting is being done.**

To adjust the lying surface for the lower leg, grip the foot-end toggle strap in the middle of the frame. Pull the lying surface for the lower leg to the desired height. The Rastomat will click into the notch locks at intervals of approx. 2cm. To lower the surface for the lower leg, use the foot-end toggle strap and pull it to the upper position. The Rastomat will then remove the lock. You can now completely lower the leg surface slowly until it rests on the bed frame.

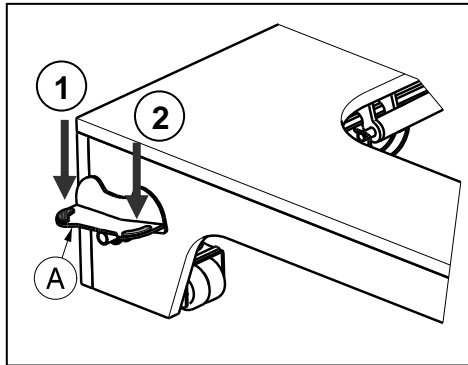
Only make adjustments when the grips are at the foot of the bed. Otherwise the bed can tip over and the adjustment mechanism damaged.

## 2.4 Operating the steering casters

⚠ **WARNING: Good operation of the steering casters is influenced by wear and contamination of the tires (water, oil, ...) – Check the condition of the castors before each use.**

- ⚠ **WARNING:** The casters can wear – Check the operation of the casters before each use.
- ⚠ **WARNING:** Danger of accidents - All casters should always be braked. If only one caster is braked, the bed can still move.

All steering castors can be locked to prevent involuntary rolling.


- 1** Activating the brakes  
Press the brake pedals (A) on the head end and feet end in the direction of the red marking. The castors on the other side blocks automatically.
- 2** Releasing the brakes  
Press the brake pedals (A) on the head end and feet end in the direction of the green marking. The castors on the other side release automatically.

## 2.5 Strain relief

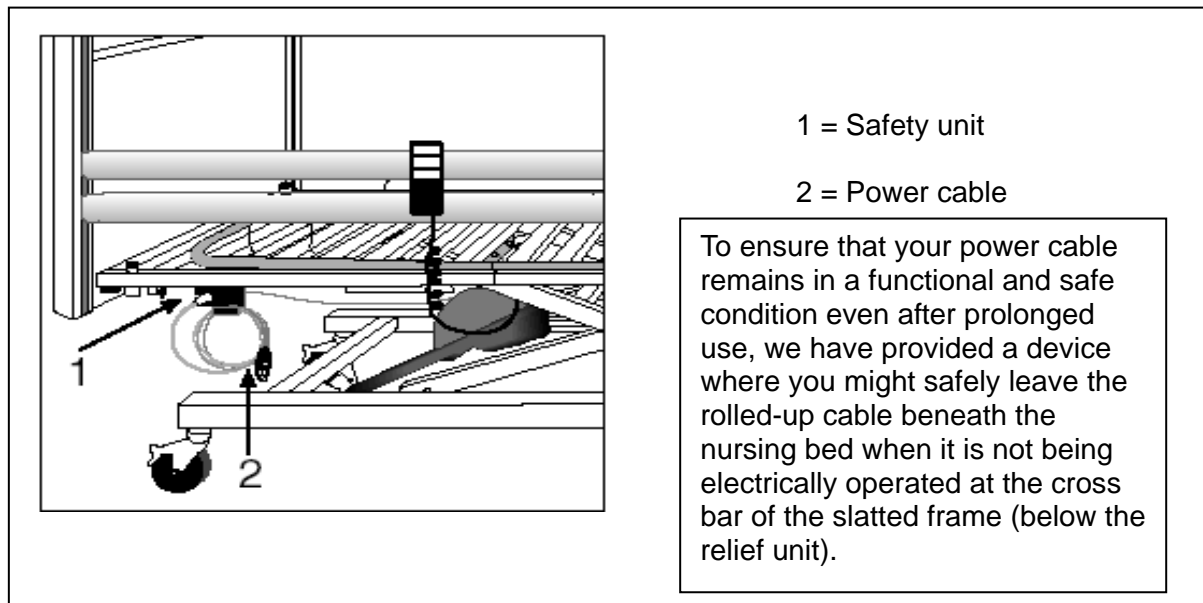- ⚠ **WARNING:** Do not remove the relief unit.
- ⚠ **WARNING:** Nothing must be fitted to or hung on the relief unit.
- ⚠ **WARNING:** The power cable must be carefully positioned without kinks, without pulling, and without touching the floor.


For your safety, the power cable is fitted with a strain relief device to prevent the power plug from being removed from the transformer; this may break the cable or reduce the effectiveness of the system's spray water protection.

At the same time it makes sure that the power cable is not involuntarily driven over, should it lie beneath the bed. This relief unit comes factory installed.

## 2.6 Power cable

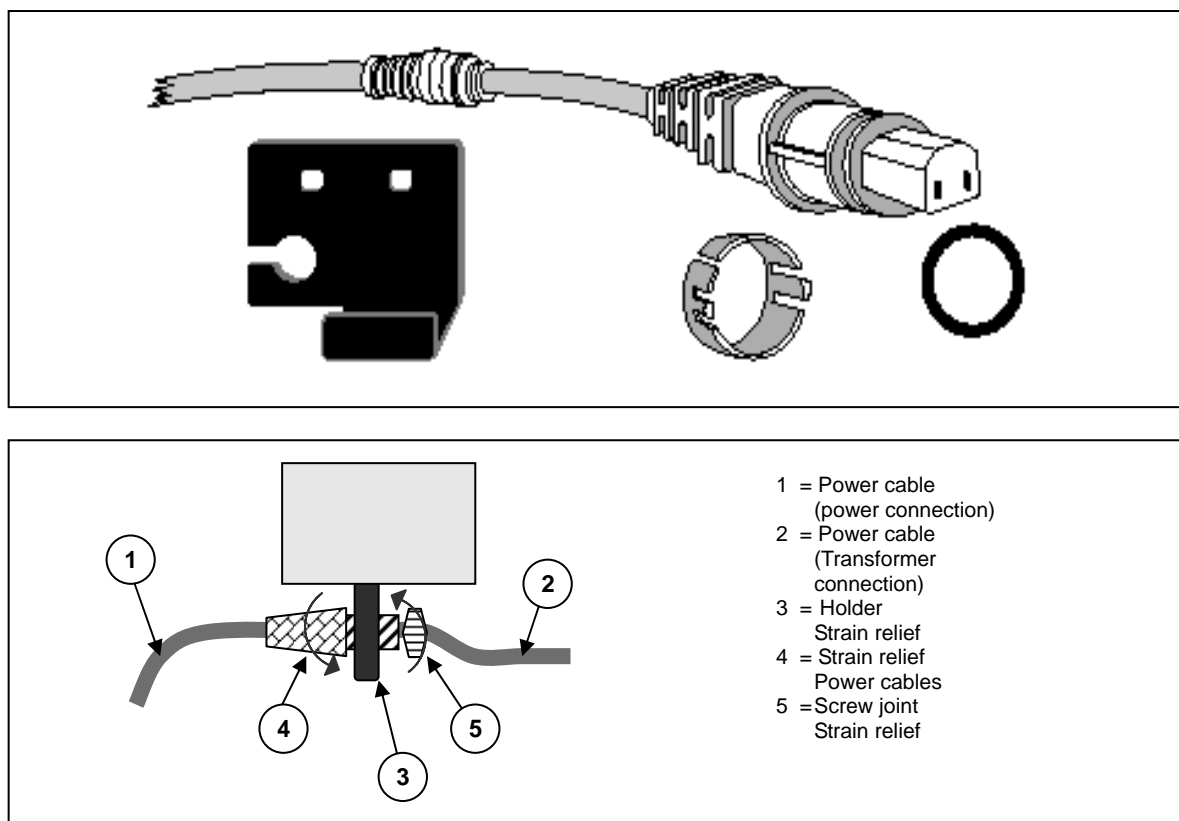Below the cross bar of the head slatted frame there is a socket for attaching the relief unit.


- Insert the power cable relief unit (4) into its intended holder(3).
- Pull the desired length of power cable (1) so that the power cable (2) can be laid under the slatted frame without sagging. Be careful to avoid having the cable (2) come under the traction in any adjustment position.
- Fasten the power cable by properly tightening the power cable relief unit(4).
- Fix the position of the relief unit by manually tightening the screw joints (5).

## 2.7 Emergency lowering of the headrest

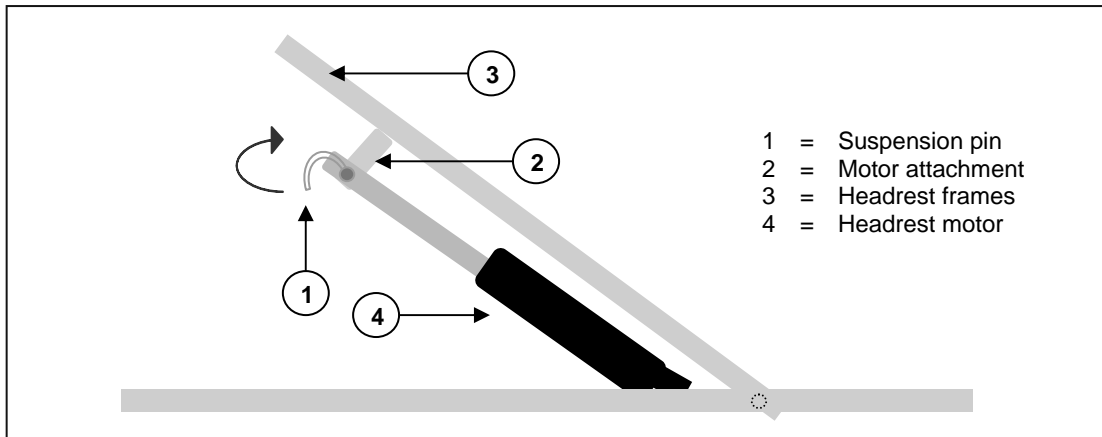- ⚠ **WARNING:** Risk of injury - Make sure that there are no objects or body parts between the headrest and the frame.
- ⚠ **WARNING:** Risk of injury - Emergency lowering is only permitted if at least two persons are available.


The headrest can be lowered manually (emergency lowering in the case of motor failure).

- Procedure:
1. Turn the securing loop of the upper motor over.
  2. One person holds the side of the headrest while another person pulls the suspension pin out and removes the motor from its holder.
  3. Lower the headrest gently.

## 2.8 Side railings

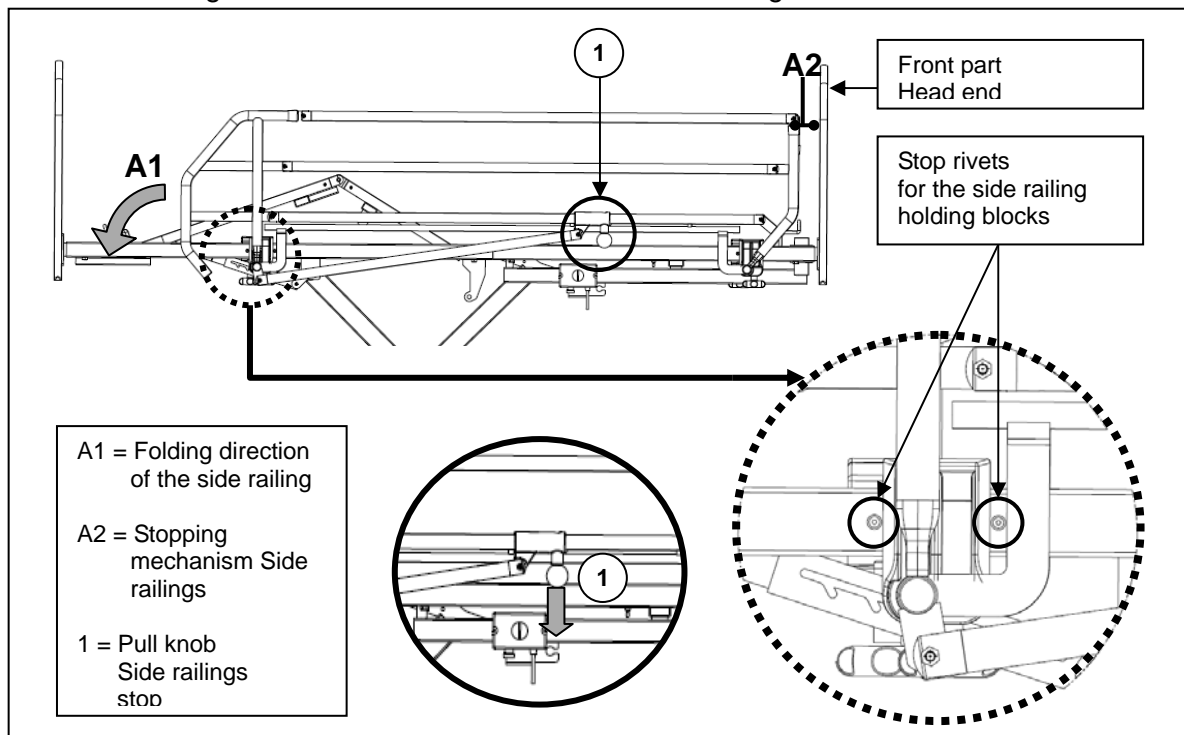
The metal side railings are to be fastened directly to the mattress frame using large star bolts. Please make sure that the side railings are immediately mounted on both sides (the vertical side of the railing needs to be mounted on the front side of the head rest section (A2)). Stop rivets will need to be mounted on the outer edge of the lying surface between which the holding blocks shall be mounted for the side railings.

The metal side railing can be lowered by gripping the upper side railing cross bar and pulling down the knob (1) on the underside of the railing. The side railing can then be folded in down in direction A1. To raise once again, grip the upper side member of the railing and pull the railing upward until you hear the railing pull button go click and the head section of the side railing is vertical to the front section of the head rest surface (A2).

Wooden side railings are also available. The following provides a description of how to install the wooden side railings.

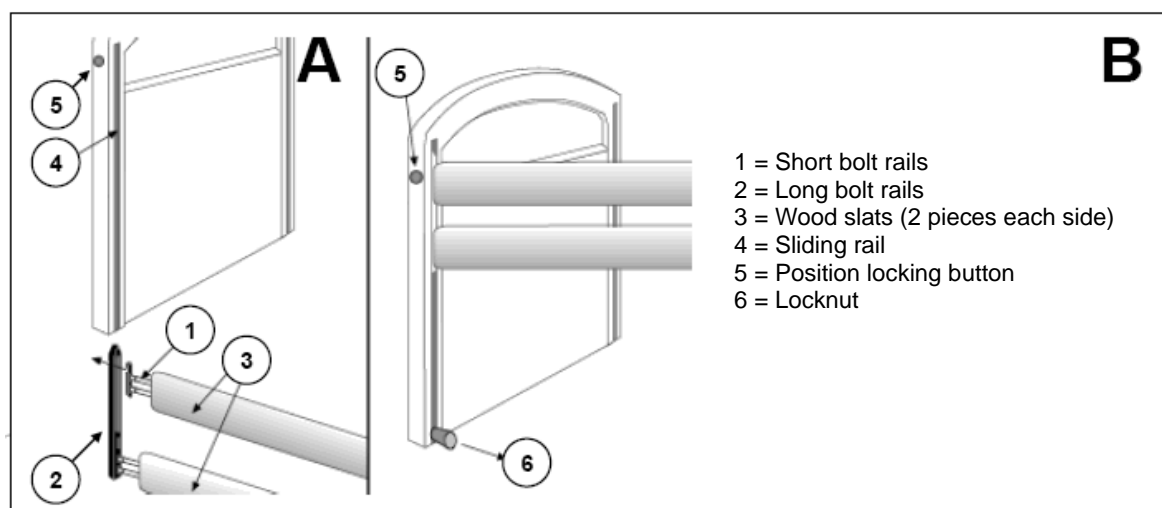
Adjust the height of the bed in the middle. With a too low height it is not possible to mount the side rails. With a too high height you must pull the side rails to far upwards.

- A.** Insert the short bolt rails (1) on the right and left in the upper wood slat. Insert the long bolt rails (2) right and left in the lower wood slat.
- Insert the short bolt rails (1) + the upper wood slat on one side (left or right) in the long bolt rails (2). Slide the entire bolt rail system (1) + (2) in the relevant sliding rail (4) from the head and foot section. Grasp the upper wooden slat upwards until the wooden side rail locks in position. Repeat these steps for the other side.
- B.** Insert the locknuts (6) and tighten them manually.
- To release the wooden side rails press the locking button (5) and pull the upper wooden slat gently upwards.

## 2.9 Lifting pole (as option)

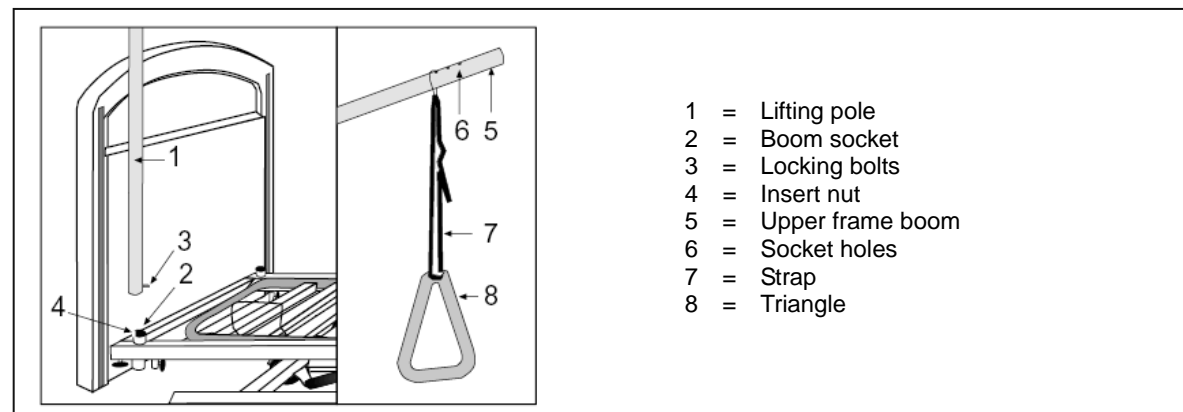The lifting pole can be mounted on either side of the head end.

- Insert the boom (1) as far as it will go into the relevant socket (2).
- Let the arresting bolts (3) slide into the groove (4).
- Pull the metal loop over the end of the upper frame (5) until the bolts engage one of the three socket holes (6).
- Also pull strap (7) into the metal loop and through the triangle (8); then fix its length. Secure the strap.

## 2.10 Transfer in and out the bed

**⚠ CAUTION: Risk of falling - In case you cannot perform the transfer in a safe manner, ask someone to assist you.**

1. Position you as close as possible to the bed.
2. Check the brakes from the bed are in the on position.
3. Remove or fold down the side railings of the bed.
4. Transfer to/from the bed.

## 2.11 Transport

**⚠ WARNING: Risk of injury – Make sure that the bed is attached properly in the car or truck.**

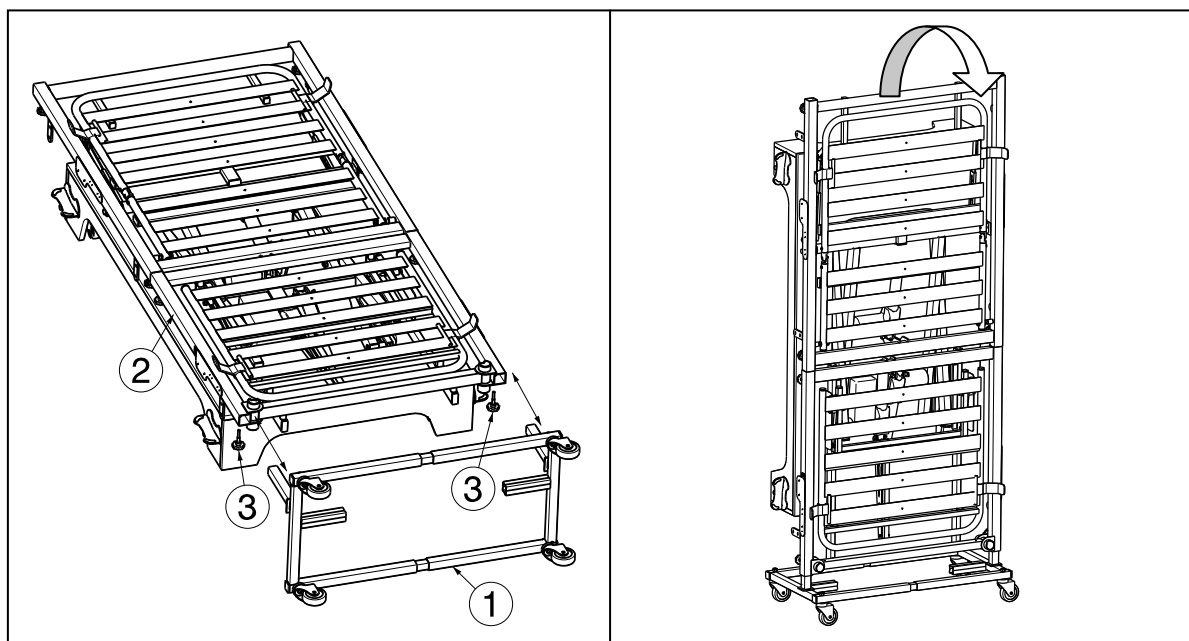
**⚠ WARNING: Risk of injury – Transportation of the bed shall be only without the patient.**

To avoid damage when transported, the lifting pole should be detached. Unplug the power cable and check that no cables hang near the floor.


Should the bed be transported over greater distances, use the transport set to do so.

Installing the transport set:

- Lower the bed into its lowest position and then remove the power cable from the power supply.
- Insert the transport kit (1) in the mattress frame (2) (headside). Fasten the transport kit with the star bolts (3).
- Then turn the mattress frame on the side where the transport kit is mounted thus moving the transport set's castors downward. The bed can now be pushed easily on the rolling castors.

## 2.12 Cable placement

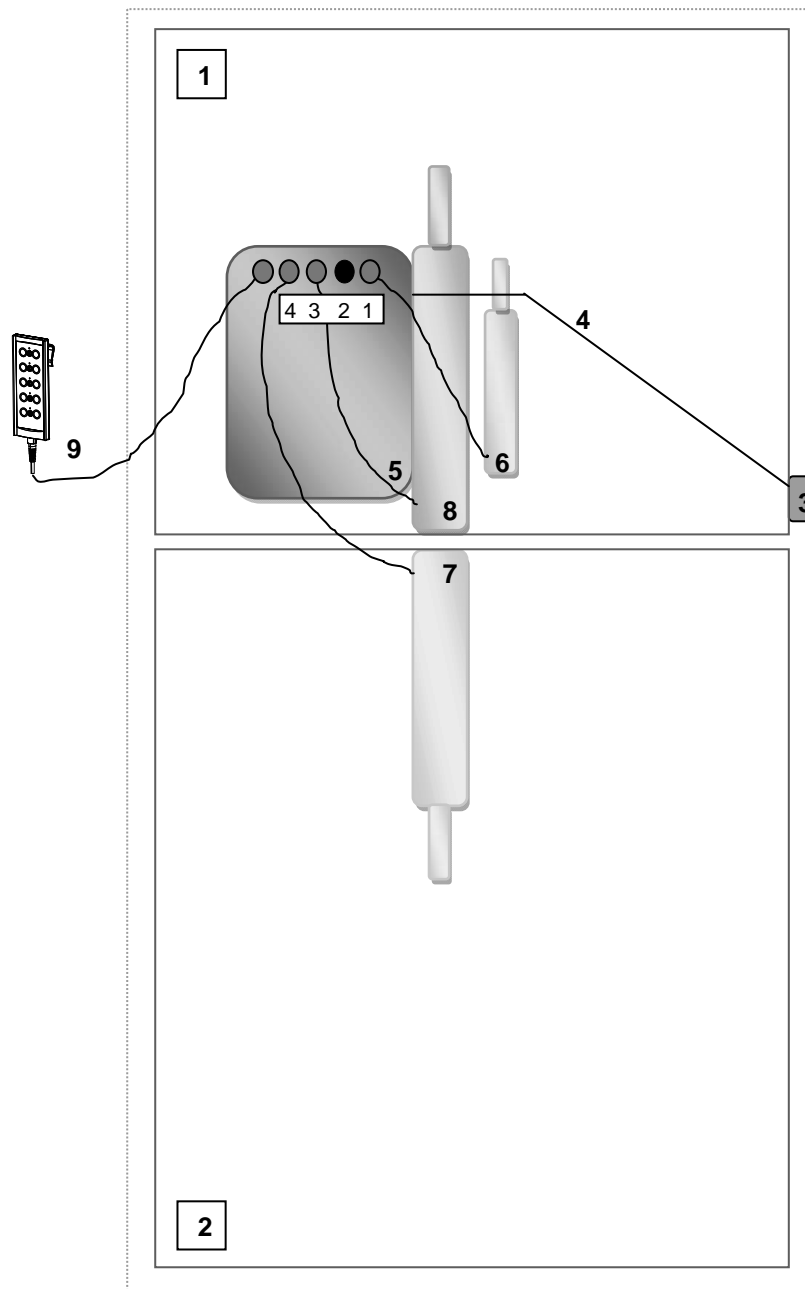- 1 = Mattress slatted frame Head end
- 2 = Mattress slatted frame Feet end
- 3 = Strain relief
- 4 = Power cable
- 5 = Control box
- 6 = Undercarriage motor
- 7 = Feet end motor
- 8 = Head end motor
- 9 = Handset

All wires within the bed frame shall be placed so contact with the floor, deformity and crushing are avoided. Insert the power cables for the motors in the opening on the motor mounts.

## 2.13 Remarks on electromagnetic compatibility (EMC)

Disruption of electrical equipment from electromagnetic fields in the surroundings can not be entirely avoided. Possible consequences are:

Possible sources of radiation include:

- Portable receiver and transmission devices (receiver and transmitter with fixed antennae)
  - Walkie-talkies, cell phones or cordless telephones
  - other personal broadcast devices
- Mobile medium-range transmitting and receiving devices (external car antennas)
  - Hands-free mobile speaking devices (fixed installation)
- Other home appliances
  - Laptop
  - etc.

Devices such as electric shavers and hair dryers should have no influence. However they need to function properly and have proper wiring regardless of their influence. Please read the appropriate instruction manual from the relevant manufacturer.

To reduce the effect of electromagnetic sources of interference, please heed the following warnings:

- ⚠ Do not use any portable television or radio devices in the immediate vicinity of the nursing bed.
- ⚠ Do not use any walkie talkies or mobile phones in the immediate vicinity of the nursing bed.
- ⚠ Should any unintended movements occur, remove the power plug for the nursing bed from the wall socket.

<b><u>Guidelines and manufacturer's declaration - Electromagnetic transmissions</u></b>		
The nursing bed is intended for operations in surroundings described below. The customer or bed user needs to make sure that it will be operated in surroundings of this type.		
<b><u>Electromagnetic radiation measurements</u></b>	<b>Agreement</b>	<b>Electromagnetic surroundings - Guidelines</b>
HF - Radiation pursuant to CISPR 11 (partial)	<u>Group 1</u>	The nursing bed uses HF energy exclusively for its internal function. Therefore its HF radiation is very low and it is unlikely that neighboring electronic devices will be disrupted.
HF - Radiation pursuant to CISPR 11 (partial)	Class B	The nursing bed is suitable for use in all facilities which are solely used as living areas and which have an immediate connection to the public power supply, which also supplies the building which is being used as living quarters.
Transmission of harmonics in accordance with IEC 61000-3-2	Class A	
Transmission of Voltage fluctuations/ flickers pursuant to IEC 61000-3-3	In accordance with	

<b>Guidelines and manufacturer's declaration - Electromagnetic resistance</b>			
The nursing bed is intended for operations in surroundings described below. The customer or the user of the nursing bed needs to make sure that it will be operated in surroundings of this type.			
<b>Resistance tests</b>	<b>IEC 60601 Test level</b>	<b>Compliance level</b>	<b>Electromagnetic surroundings - Guidelines</b>
Discharge of static electricity (ESD) pursuant to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV Air discharge	± 6 kV contact discharge ± 8 kV Air discharge	Flooring needs to be made of wood or concrete or covered with ceramic tiles If the floor is covered with synthetic material, then relative humidity needs to be at least 30%.
Quick stray electrical factors / bursts pursuant to IEC 61000-4-4	± 2 kV for power cables ± 1 kV for input and output cables	± 2 kV for power cables ± 1 kV for input and output cables	The quality of the power voltage needs to meet standards typical in a commercial or hospital environment.
Surges pursuant to IEC 61000-4-5	± 1 kV Series mode voltage ± 2 kV Common-mode voltage	± 1 kV Series mode voltage ± 2 kV Common-mode voltage	The quality of the power voltage needs to meet standards typical in a commercial or hospital environment.
Voltage drops, Short-term disruptions and Fluctuations in the supply voltage pursuant to IEC 61000-4-11	< 5 % $V_T$ (>95 % drop in $V_T$ ) for ½ period  40 % $V_T$ (60 % drop in $V_T$ ) for 5 periods  70 % $V_T$ (30 % drop in $V_T$ ) for 25 periods  < 5 % $V_T$ (>95 % drop in $V_T$ ) for 5 seconds	< 5 % $V_T$ (>95 % drop in $V_T$ ) for ½ period  40 % $V_T$ (60 % drop in $V_T$ ) for 5 periods  70 % $V_T$ (30 % drop in $V_T$ ) for 25 periods  < 5 % $V_T$ (>95 % drop in $V_T$ ) for 5 seconds	The quality of the power voltage needs to meet standards typical in a commercial or hospital environment. Should the user of the nursing bed require the continuous function even with the occurrence of disruptions to the power supply, it is recommended that the nursing bed receive undisturbed power from an emergency source or a battery.
Magnetic field for the Supply frequency (50/60 Hz) pursuant to IEC 61000-4-8	3 A/m	3 A/m	Magnetic fields with the grid frequency should comply to those normal in the commercial or hospital environments.
NOTE $V_T$ is the grid alternating voltage before applying the test level.			

### 3 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

**⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

#### 3.1 Tools

To set up the bed the following tools are needed.

- Allen key
- Screwdriver

### 3.2 Manner of delivery

The PRACTIC HC bed has two configuration options. A mattress frame is available with or without an optional motorized knee bend.

- Undercarriage with motor LA27
- Slatted frame, foot-end with Rastomat
- Slatted frame, headrest incl. transformer CB6 and motor LA27
- Wooden blind (head and foot-end) (965)
- Wooden side railings
- Handset HB7x
- Allen key (for assembly) (with 965)
- Instruction Manual

The nursing bed may only be used with the parts listed here. Should components not be installed or equipment, components, or electrical equipment from third parties be installed then the bed shall no longer be considered the bed delivered by the manufacturer. In such a case the use of the bed is then at one's own risk. Then the CE conformity and the manufacturer warranty shall no longer apply.

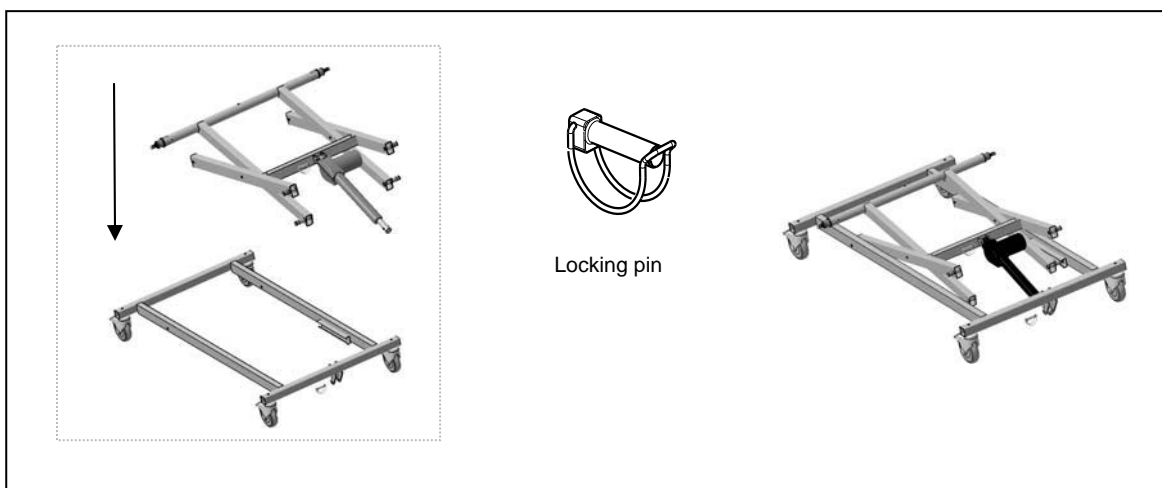
### 3.3 Assembly

- ⚠ **CAUTION: Chance of pinching - Do not place fingers between the components of the bed.**
- ⚠ **CAUTION: Risk of clamping - Keep fingers away from moving parts of the bed.**
- ⚠ **CAUTION: Risk of injury - No third persons or any objects may come in the assembly area.**

You should obey the assembly instructions of your nursing bed as explained below.


#### 3.3.1 Undercarriage

1. Place the undercarriage along with the rolling castors on a flat non-porous surface. Block the four castors by activating their brakes (note also the "Steering Castor" chapter).
2. Insert the shears with the bearings in the c-profile and pin them securely on the other side.
3. Secure the motor with the locking pins.

### 3.3.2 Slatted Frame

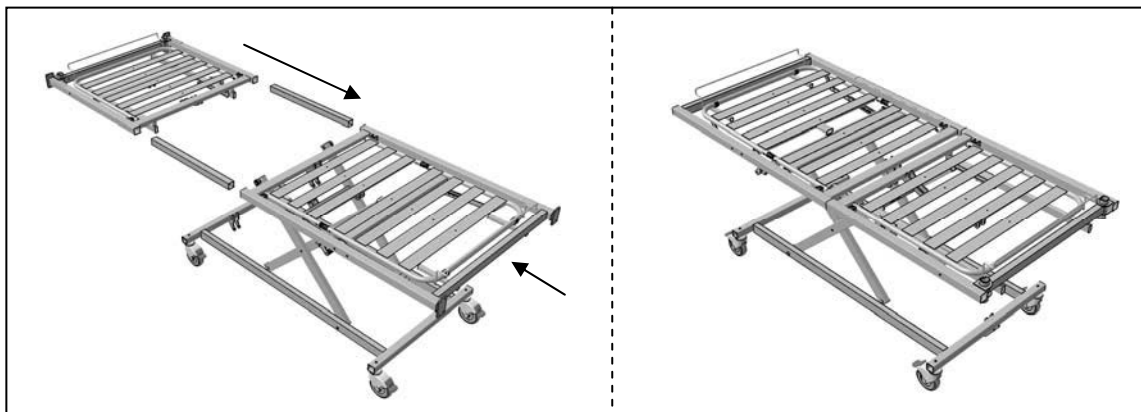
1. Insert head end using the c-profile on the bearing receiving means for the undercarriage.
2. Insert the feet end using the connection pipe to the head end. Connect the feet end to the undercarriage using locking pins.
3. Carefully fasten the screws in the connection pipes to the mattress slatted frame.

### 3.3.3 Head panel / feet panel

**⚠ CAUTION: Risk of clamping - Make sure that the motor cables cannot be crushed.**

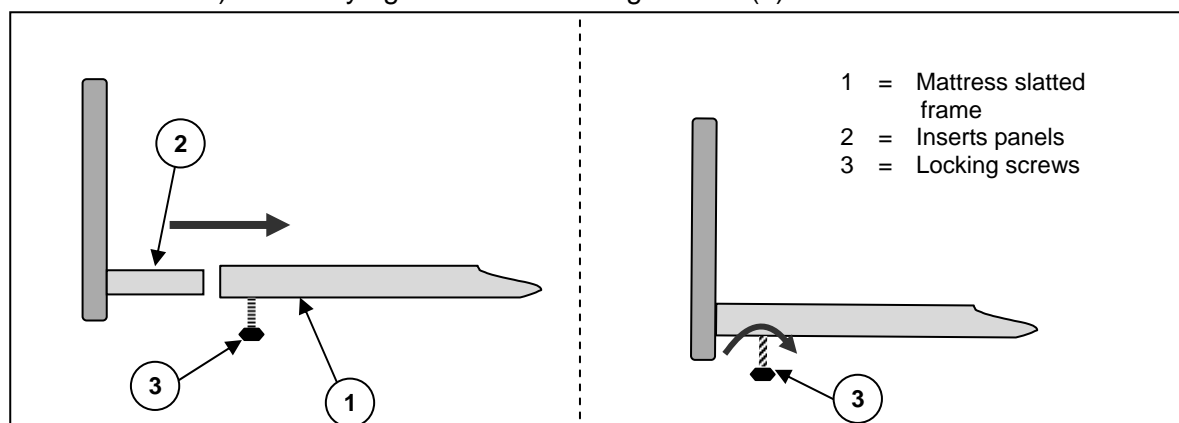1. Insert the front ends (2) in the open pipe ends on the mattress slatted frame (1), both on the head end and the feet end until they click (the locking screws (3) need to be loosened first). Carefully tighten all the locking screws (3) to the mattress slatted frame.


Fasten all components with the locking screws. While doing so, the locking screws need to be tight enough to guarantee that the front section is securely fastened.

### 3.3.4 Transformer connections

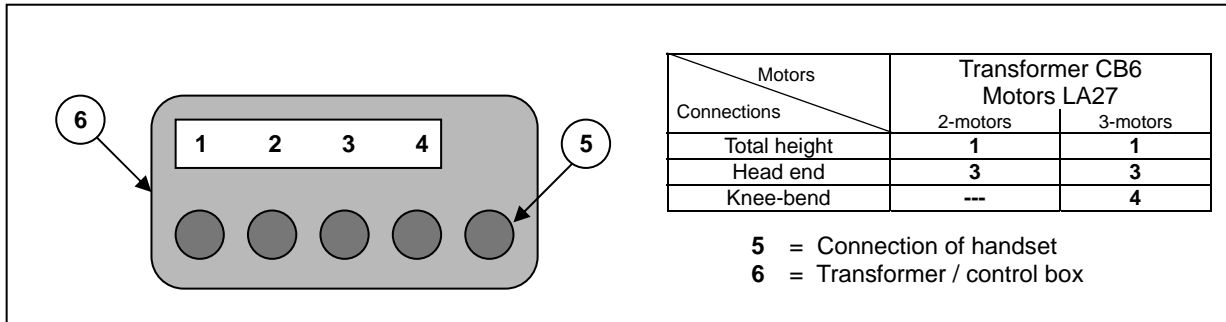**⚠ CAUTION: Risk of injury - Do not touch the plug with wet hands.**

**⚠ CAUTION: Risk of injury - Do not place any objects on the connection and the cable, and protect these against mechanical loads.**

1. A 220 V / 230 V, 50 Hz power socket conforming to the VDE specifications is required
2. The diagram for the transformer connections is given in the figure below:

**The motor connection plugs are labeled with the same number as the connections on the transformer.**

Take hold of the plug only to remove it from the power socket.  
 The power connection must not have any loops or kinks and should not pass around sharp edges. Protect the power connection and the cable leading to the manual switch from being crushed.

## 4 Maintenance

For the maintenance manual of the beds refer to the Vermeiren website:  
[www.vermeiren.com](http://www.vermeiren.com).

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## ENGLISH

### CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

### APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

### EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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---

## Notes

A series of horizontal dotted lines for taking notes.

---

## Notes

A series of horizontal dotted lines for taking notes.

## Belgium

### N.V. Vermeiren N.V.

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)

## France

### Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin  
F-59710 Avelin  
Tel: +33(0)3 28 55 07 98  
Fax: +33(0)3 20 90 28 89  
website: [www.vermeiren.fr](http://www.vermeiren.fr)  
e-mail: [info@vermeiren.fr](mailto:info@vermeiren.fr)

## Italy

### Vermeiren Italia

Viale delle Industrie 5  
I-20020 Arese MI  
Tel: +39 02 99 77 07  
Fax: +39 02 93 58 56 17  
website: [www.vermeiren.it](http://www.vermeiren.it)  
e-mail: [info@vermeiren.it](mailto:info@vermeiren.it)

## Poland

### Vermeiren Polska Sp. z o.o

ul. Łączna 1  
PL-55-100 Trzebnica  
Tel: +48(0)71 387 42 00  
Fax: +48(0)71 387 05 74  
website: [www.vermeiren.pl](http://www.vermeiren.pl)  
e-mail: [info@vermeiren.pl](mailto:info@vermeiren.pl)

## Czech Republic

### Vermeiren ČR S.R.O.

Nádražní 132  
702 00 Ostrava 1  
Tel: +420 596 133 923  
Fax: +420 596 121 976  
website: [www.vermeiren.cz](http://www.vermeiren.cz)  
e-mail: [info@vermeiren.cz](mailto:info@vermeiren.cz)

## Germany

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Wahlerstraße 12 a  
D-40472 Düsseldorf  
Tel: +49(0)211 94 27 90  
Fax: +49(0)211 65 36 00  
website: [www.vermeiren.de](http://www.vermeiren.de)  
e-mail: [info@vermeiren.de](mailto:info@vermeiren.de)

## Austria

### Vermeiren Austria GmbH

Schärddinger Strasse 4  
A-4061 Pasching  
Tel: +43(0)7229 64900  
Fax: +43(0)7229 64900-90  
website: [www.vermeiren.at](http://www.vermeiren.at)  
e-mail: [info@vermeiren.at](mailto:info@vermeiren.at)

## Switzerland

### Vermeiren Suisse S.A.

Eisenbahnstrasse 62  
3645 Gwatt (Thun)  
Tel: +41(0)33 335 14 75  
Fax: +41(0)33 335 14 67  
website: [www.vermeiren.ch](http://www.vermeiren.ch)  
e-mail: [info@vermeiren.ch](mailto:info@vermeiren.ch)

## Spain / Portugal

### Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5  
Sant Gregori Parc Industrial Edifici A  
17150 Sant Gregori (Girona)  
Tel: +34 972 42 84 33  
Fax: +34 972 40 50 54  
website: [www.vermeiren.es](http://www.vermeiren.es)  
e-mail: [info@vermeiren.es](mailto:info@vermeiren.es)

## Manufacturer

### Vermeiren GROUP

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Tel: +32(0)3 620 20 20  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)