# VERMEIREN

2030

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EN

### **Instructions for specialist dealer**

This instruction manual is part and parcel of the product and must accompany every product sold.

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Version: B, 2017-05

### **All rights reserved, including translation.**

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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## Contents

<b>Preface</b> .....	<b>2</b>
<b>1 Product description</b> .....	<b>3</b>
1.1 Intended use.....	3
1.2 Technical specifications .....	4
1.3 Components.....	5
1.4 Explanation of symbols .....	5
1.5 Accessories.....	6
1.6 For your safety .....	6
<b>2 Use</b> .....	<b>7</b>
2.1 Remarks on electromagnetic compatibility (EMC) .....	7
2.2 Operating the steering casters .....	9
2.3 Strain relief.....	9
2.4 Power cable .....	10
2.5 Operating leg section (knee bending).....	10
2.6 Handset.....	11
2.7 Use of the side railings .....	12
2.8 Transfer in and out the bed.....	12
2.9 Transport .....	12
2.10 Cable placement .....	13
<b>3 Installation and adjustment</b> .....	<b>13</b>
3.1 Tools .....	13
3.2 Manner of delivery .....	14
3.3 Assembly.....	14
3.4 Disassembly .....	18
<b>4 Maintenance</b> .....	<b>18</b>

## Preface

First of all we want to thank you for putting your trust in us by selecting one of our products.

The Vermeiren beds are the result of many years of research and experience. During the development, special attention was given to the ease of use and the serviceability of the bed.

The expected lifetime for your bed is strongly influenced by the care and maintenance of the bed.

This manual will help you get acquainted with the operation of your bed.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

# 1 Product description

## 1.1 *Intended use*

With the 2030 model, you have acquired a nursing bed that was developed entirely for your comfort. This nursing bed offers a number of individual adjustment possibilities which are explained below.

The nursing bed is designed for a single adult patient (from 12 years, up to maximum user weight 135 kg).

The bed shall be used with a suitable mattress (min. RG 35, 120 mm thick, max. 20 kg).

The modular construction of the bed allows full use by disabled persons, elderly people or during revalidation in home environment.

When providing for individual requirements:

- body size and weight 135 kg
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

The bed is suited for application environment 4 of ISO 60601-2-52: "care provided in a domestic area where medical supervision is necessary and appropriate supervision is provided or nursing care provided in a home environment where ME equipment is used to alleviate or compensate for an injury, disability or disease".

Your bed should only be used on surfaces where all four wheels are touching the ground.

The bed should not be used as a ladder, nor is it a transport for heavy or hot objects.

When used on mats, carpeted floors or loose floor coverings, the floor covering can get damaged and it is difficult to push over such surfaces (without patient).

Use only Vermeiren approved accessories. It is not permitted to combine the bed with other products or equipment or other electrically driven appliances.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the guarantee conditions.

Left and right references are to be understood when you lie on your back in the bed.

## 1.2 Technical specifications

(Given in fully assembled condition)

<b>Make</b>	<b>Vermeiren</b>	
<b>Address</b>	<b>Vermeirenplein 1/15, B-2920 Kalmthout</b>	
<b>Type</b>	<b>Bed</b>	
<b>Model</b>	<b>2030 bed</b>	
<b>Maximum occupant mass</b>	<b>Nominal load 135 kg</b>	
Total length	2152 mm	
Total width	1010 mm (without side rails); 1075 mm (with side rails)	
Total height (without lifting pole)	840 mm - 1245 mm	
Total height (with lifting pole)	1833 mm - 2233 mm	
Height adjustment of the mattress support platform	350 mm - 750 mm	
Dimensions dismantled (carton box)	Head section and Leg section: 310 mm x 900 mm x 1400 mm Lifting pole: 135 mm x 500 mm x 1810 mm Side rails: 135 mm x 500 mm x 1810 mm Head panel and foot panel: 170 mm x 960 mm x 970 mm	
Total mass (excl. side rails, lifting pole)	± 70 kg	
Adjustment of the back section	78°	
Max. thigh angle	28°	
Max. angle for lower leg	20°	
Max. safe working load *	170 kg	
Lifting pole	Nominal load 75 kg	
Mattress	Min. RG 35, 100 - 120 mm thickness, 1900 x 900 mm, max. 20 kg With extension part 1950 x 900 mm	
Under clearance for patient lift	148 mm	
Steering casters (4 pieces)	Ø 100 mm, can be locked	
Board motor	LA40	
Head section motor	LA40	
Leg section motor	LA40	
Handset	HB80 (4 functions)	
Transformer	CA40	
Nominal voltage, transformer	U input: 230 V~, 50/60 Hz, current: max. 8 A/CH, output voltage: 24V ---, max. 120W	
Period switched on	10%, 2/18 min on/off	
Noise level	< 65 dB(A)	
Class / Kind of protection	Protection class II, application part Type B / IPX4 (spray protection)	
Storage temperature	-10 °C - +50 °C	
Use temperature	+5 °C - +40 °C	
Storage and use humidity	20% - 90%	
<p>We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / ° * Max. safe working load = mass user weight + mass mattress + mass other accessories</p>		


*Table 1: Technical specifications*

The bed complies to the requirements set up in:

EN 60601-2-52/A1: Medical electrical equipment - Part 2-52: Particular requirements for the safety and essential performance of medical beds

### 1.3 Components

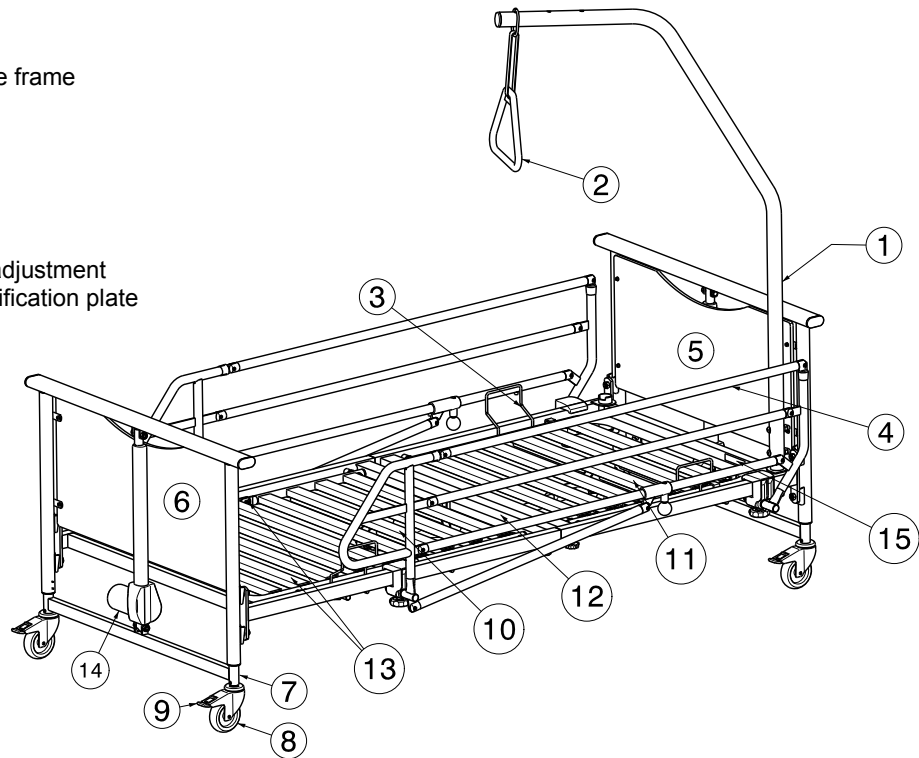- 1 = Lifting pole
- 2 = Triangle
- 3 = Mattress retainer
- 4 = Side rails
- 5 = Headboard
- 6 = Footboard
- 7 = Under carriage frame
- 8 = Caster
- 9 = Wheel lock
- 10 = Leg section
- 11 = Back section
- 12 = Seat section
- 13 = Knee bend
- 14 = Motor height adjustment
- 15 = Location identification plate

### 1.4 Explanation of symbols

- Maximum user weight
- CE conformity
- Safety instructions
- Observe the instructions for use
- Max mattress thickness
- Max safe working load
- Mattress dimensions
- Protection class II
- Type B application
- Type designation

## 1.5 Accessories

- ⚠ **WARNING: Precautions shall be taken if accessories not listed below are used.**
- Extension part between head section and leg section

## 1.6 For your safety

- ⚠ Carefully read the instruction manual before using the nursing bed. Respect all safety and user instructions.
- ⚠ Place and use your nursing bed only on straight and level surfaces and ensure that all four steering castors stand securely on the floor.
- ⚠ Be careful when using in hot or cold environments (sunshine, extreme cold, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.
- ⚠ All four steering castors must always be locked before using the nursing bed.
- ⚠ When the bed is placed in position, make sure that no obstacles are present inside the entire adjustment range.
- ⚠ It is always forbidden to go under the bed.
- ⚠ If the patient is not supervised, the bed must be adjusted beforehand to the lowest position to reduce the danger of injury by falling when getting into, getting out of, or lying on the bed.
- ⚠ Watch your hands when assembling, disassembling or using the bed -- danger of squeezing!
- ⚠ When transporting the bed, grasp it only by solid frame parts (do not grasp on the lifting pole).
- ⚠ Transportation of the bed shall be only without the patient.
- ⚠ Medical-electrical treatments are forbidden inside the nursing bed!
- ⚠ Avoid leaning out sideways with the upper body -- Risk of accidents!
- ⚠ Do not rest your legs on other objects -- Risk of accidents!
- ⚠ Do not place your hands between the frame and the mattress when adjusting leg or back rests -- Risk of squeezing!
- ⚠ The nursing bed must only be used in dry rooms.
- ⚠ The medical bed shall only be used with a patient hoist that can be positioned under the limited space of the medical bed.
- ⚠ Only use mattresses which comply to the prescriptions of the bed manufacturer.
- ⚠ If the lifting pole is damaged, it should be replaced.
- ⚠ Do not undertake repairs on your own -- risk of accidents!
- ⚠ Do not open any housings of the driving system
- ⚠ No additional mechanical or electric appliances may be used or fitted.
- ⚠ Servicing, inspection, repairs and disinfection should only be undertaken by authorized, trained persons.
- ⚠ Use only genuine replacement parts of the manufacturer for your nursing bed.
- ⚠ Use only Vermeiren approved accessories.
- ⚠ Do not exceed the maximum load as stated by the manufacturer (**MAX load 170 kg**).
- ⚠ The use of open flames like candles or glowing objects like cigarettes is forbidden inside the nursing bed.
- ⚠ Replace the side railings if they are bent or broken, otherwise there is a risk of the patient falling out of the bed.
- ⚠ The side railings must be fitted even if the relevant side is situated against the wall.
- ⚠ If the patient is not under supervision, the side rail on both sides should be raised to the highest position. The railings may only be lowered on one or both sides when a supervisor is present.

- ⚠ Use only power sockets conforming to VDE (220 V / 230 V, 50 Hz) for plugging in.
- ⚠ Plugs must only be pulled out by grasping the plug itself; never pull on the cable.
- ⚠ Disconnect the plug of the power cable out the supply socket before transporting the nursing bed.
- ⚠ Make sure that all power cables are laid without kinks, shearing or other mechanical damages.
- ⚠ All cables should be laid without making contact to the floor and without kinks.
- ⚠ Use of your bed could influence other electrical devices (noises). Avoid using such devices at the same time.
- ⚠ Electrically operated bed is placed with a distance to the power supply plug.
- ⚠ When the nursing bed is not connected to the supply socket, hang the power cable around the hook of the mattress support platform (head section).
- ⚠ Do not play with the handset. A thermal fuse is provided in the transformer to prevent overheating.
- ⚠ We advise not to use power strip and extension cord. Use the bed directly on the supply socket. We advise not to use supply sockets with a time switch.

## 2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**

The bed is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the bed are given in § 3.

- ⚠ **WARNING: The medical bed shall only be used with a patient hoist that can be positioned under the limited space of the bed.**
- ⚠ **WARNING: Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.**

To activate / deactivate the bed you need to connect / disconnect the power cable in the power supply.

If the bed is activated a light is lit on the control box (transformer).

### 2.1 Remarks on electromagnetic compatibility (EMC)

Disruption of electrical equipment from electromagnetic fields in the surroundings can not be entirely avoided. Possible consequences are:

Possible consequences are: In the presence of very strong or long-lasting disruptive fields, the electrical systems might break down completely and suffer permanent damage.

Possible sources of radiation include:

- Portable receiver and transmission devices (receiver and transmitter with fixed antennae)
  - Walkie-talkies, cell phones or cordless telephones
  - Portable TV, radio and navigation devices
  - other personal broadcast devices
- Mobile medium-range transmitting and receiving devices (external car antennas)
  - Fixed transmitting and receiving sets
  - Hands-free mobile speaking devices (fixed installation)
  - Fixed radio, TV and navigation systems
- Other home appliances
  - CD-player
  - Laptop
  - Microwave oven
  - Cassette recorder
  - etc.

Devices such as electric shavers and hair dryers should have no influence. However they need to function properly and have proper wiring regardless of their influence. Please read the appropriate instruction manual from the relevant manufacturer.

To reduce the effect of electromagnetic sources of interference, please read the following warnings:

- ⚠ Do not use any portable television or radio devices in the immediate vicinity of the nursing bed.
- ⚠ Do not use any walkie talkies or mobile phones in the immediate vicinity of the nursing bed.
- ⚠ Should any unintended movements occur, remove the power plug for the nursing bed from the wall socket.

<b>Guidelines and manufacturer's declaration - Electromagnetic transmissions</b>		
The nursing bed is intended for operations in surroundings described below. The customer or bed user needs to make sure that it will be operated in surroundings of this type.		
<b>Electromagnetic radiation measurements</b>	<b>Agreement</b>	<b>Electromagnetic surroundings - Guidelines</b>
HF - Radiation pursuant to CISPR 11 (partial)	<u>Group 1</u>	The nursing bed uses HF energy exclusively for its internal function. Therefore its HF radiation is very low and it is unlikely that neighboring electronic devices will be disrupted.
HF - Radiation pursuant to CISPR 11 (partial)	Class B	The nursing bed is suitable for use in all facilities which are solely used as living areas and which have an immediate connection to the public power supply, which also supplies the building which is being used as living quarters.
Transmission of harmonics in accordance with IEC 61000-3-2	Class A	
Transmission of Voltage fluctuations/ flickers pursuant to IEC 61000-3-3	In accordance with	

<b>Guidelines and manufacturer's declaration - Electromagnetic resistance</b>			
The nursing bed is intended for operations in surroundings described below. The customer or the user of the nursing bed needs to make sure that it will be operated in surroundings of this type.			
<b>Resistance tests</b>	<b>IEC 60601 Test level</b>	<b>Compliance level</b>	<b>Electromagnetic surroundings - Guidelines</b>
Discharge of static electricity (ESD) pursuant to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV Air discharge	± 6 kV contact discharge ± 8 kV Air discharge	Flooring needs to be made of wood or concrete or covered with ceramic tiles. If the floor is covered with synthetic material, then relative humidity needs to be at least 30%.
Quick stray electrical factors / bursts pursuant to IEC 61000-4-4	± 2 kV for power cables ± 1 kV for input and output cables	± 2 kV for power cables ± 1 kV for input and output cables	The quality of the power voltage needs to meet standards typical in a commercial or hospital environment.
Surges pursuant to IEC 61000-4-5	± 1 kV Series mode voltage ± 2 kV Common-mode voltage	± 1 kV Series mode voltage ± 2 kV Common-mode voltage	The quality of the power voltage needs to meet standards typical in a commercial or hospital environment.
Voltage drops, Short-term disruptions And Fluctuations in the supply voltage pursuant to IEC 61000-4-11	< 5 % $V_T$ (>95 % drop in $V_T$ ) for ½ period  40 % $V_T$ (60 % drop in $V_T$ ) for 5 periods  70 % $V_T$ (30 % drop in $V_T$ ) for 25 periods  < 5 % $V_T$ (>95 % drop in $V_T$ ) for 5 seconds	< 5 % $V_T$ (>95 % drop in $V_T$ ) for ½ period  40 % $V_T$ (60 % drop in $V_T$ ) for 5 periods  70 % $V_T$ (30 % drop in $V_T$ ) for 25 periods  < 5 % $V_T$ (>95 % drop in $V_T$ ) for 5 seconds	The quality of the power voltage needs to meet standards typical in a commercial or hospital environment. Should the user of the nursing bed require the continuous function even with the occurrence of disruptions to the power supply, it is recommended that the nursing bed receive undisturbed power from an emergency source or a battery.

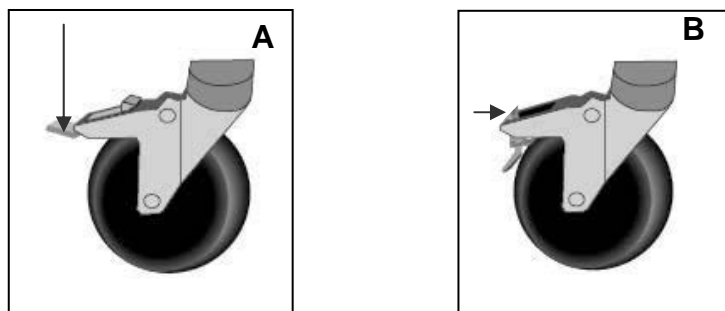
Magnetic field for the Supply frequency (50/60 Hz) pursuant to IEC 61000-4-8	3 A/m	3 A/m	Magnetic fields with the grid frequency should comply to those normal in the commercial or hospital environments.
NOTE $V_T$ is the grid alternating voltage before applying the test level.			

## 2.2 Operating the steering casters

- ⚠ **WARNING:** Good operation of the steering casters is influenced by wear and contamination of the casters (water, oil, ...) – Check the condition of the casters before each use.
- ⚠ **WARNING:** The steering casters can wear – Check the operation of the steering casters before each use.
- ⚠ **WARNING:** Danger of accidents - All steering casters should always be braked. If only one caster is braked, the bed can still move.

All casters can be locked to prevent involuntary movements.

- **Securing (A)**  
Push the surface of the tilting lever downwards by foot until it brakes the caster.
- **Releasing (B)**  
Push the surface of the tilting lever forwards by foot until the caster is released.



## 2.3 Strain relief

- ⚠ **WARNING:** Do not remove the relief unit.
- ⚠ **WARNING:** Nothing must be fitted to or hung on the relief unit.

Below the spar of the mattress support platform there is a socket for the tightening of the relief unit.

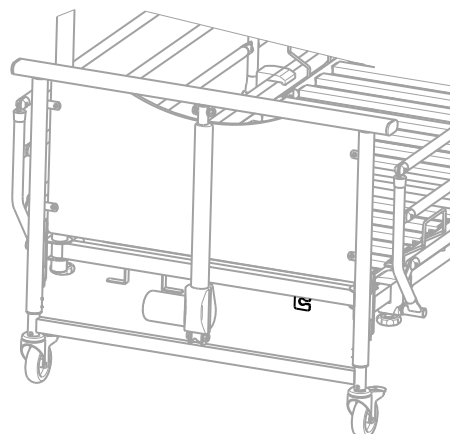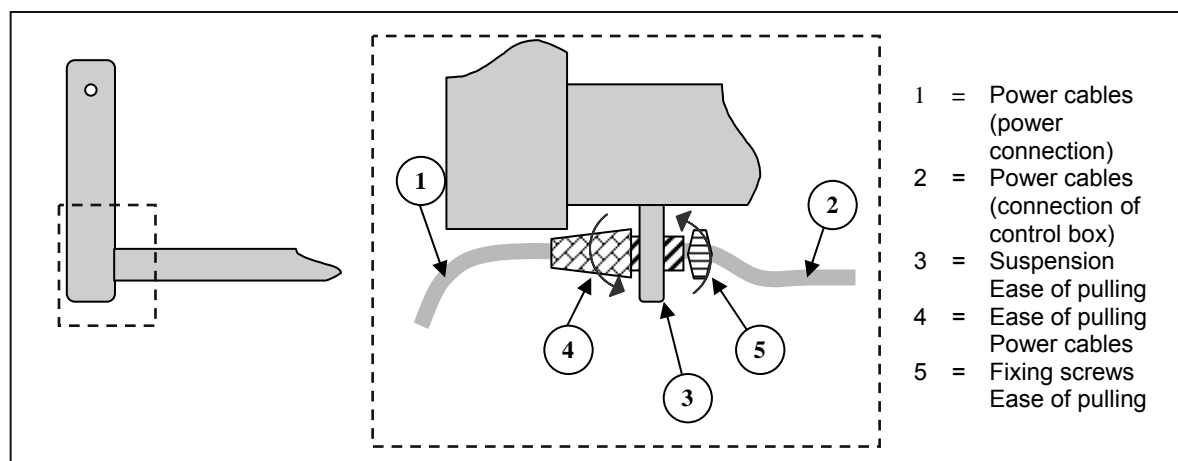- Insert the relief unit of the power cable (4) into the suspension (3).
- Pull the desired length of the power cable (1) in such a way that the power cable (2) can be laid under the mattress support platform without sagging. Be careful to avoid that the cable (2) come under the traction in any adjustment position.
- Fasten the power cable by properly tightening the relief unit of the power cable (4).
- Fasten the position of the relief unit by properly tightening the fixing screws (5).


## 2.4 Power cable


**⚠ WARNING:** Nothing but the power cable should be suspended here.

**⚠ WARNING:** The power cable can be damaged - The power cable must be carefully positioned without kinks, pulling, shearing or other mechanical damages and without making contact with the floor.

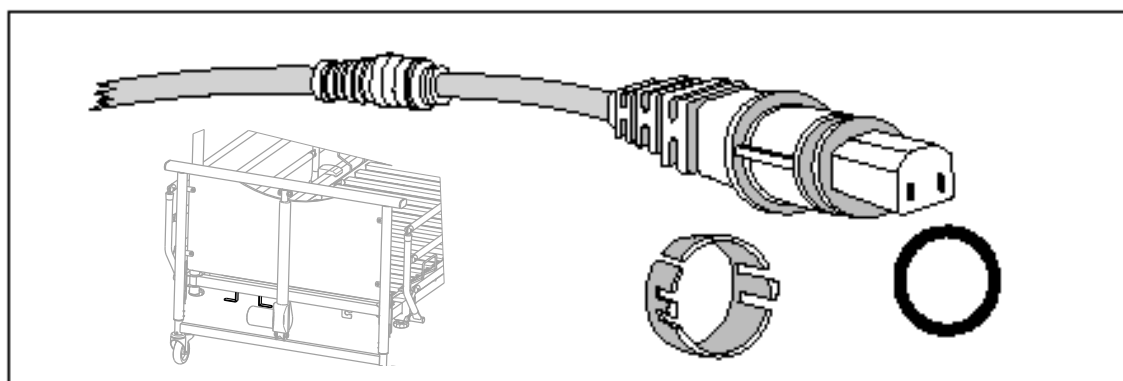To ensure that your power cable stays properly functional and in a technically secure condition even after protracted use, and in the case that you do not operate your bed electrically, we have at the mattress support platform (head section) provided a holder where you might safely leave the rolled-up cable beneath the nursing bed.

## 2.5 Operating leg section (knee bending)

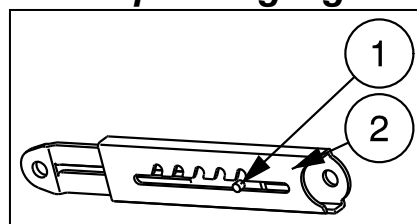**⚠ WARNING:** Risk of injury - Check that for the angle adjustment of the knee bending the pin ① of the tooth adjustment system ② is firmly secured.

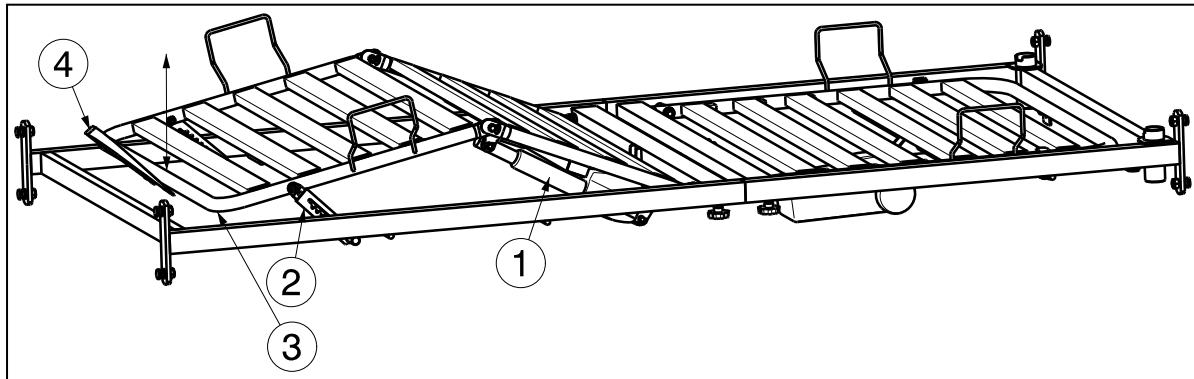
In order to prevent unwanted operation during transport, the leg section is foreseen of a locking hook. First slide this to the side or remove this before you can operate the knee bend.

The angle of the upper leg section is adjustable with a motor ①. This you can adjust by using the handset.

To adjust the angle of the lower leg section (knee bend) a mechanical mechanism with tooth adjustment ② is available. When pulling the toggle strap ④ on the end of the lower leg section ③ upwards, you can adjust the leg section in 6 different positions (-2°, -5°; -8°, -11°, -14°, -19°). With every adjustment you hear a clear click. Check that the tooth adjustment is placed in the same position at both sides and is firmly secured.

To place the knee bend in his lowest position, pull the lower leg section completely upwards with the toggle strap ④ and afterwards let the lower leg section completely down.

## 2.6 Handset

The height of the bed surface, the angle of the headrest and the leg rest can be adjusted with the handset. Lock and mount the handset on the side rail after use.

Up      Down

Release position

← Head / back section down and up function

← Legrest down and up function

← Mattress support platform (Height adjustment) down and up

← Anti-trendelenburg

The handset has a lock function. It allows for one of more functions to be disabled or released.

**Precautionary measures:**

- To switch from lock and release positions, a small button between both handset buttons needs to be turned with help of a key.

The key shall only be available to nursing personnel.

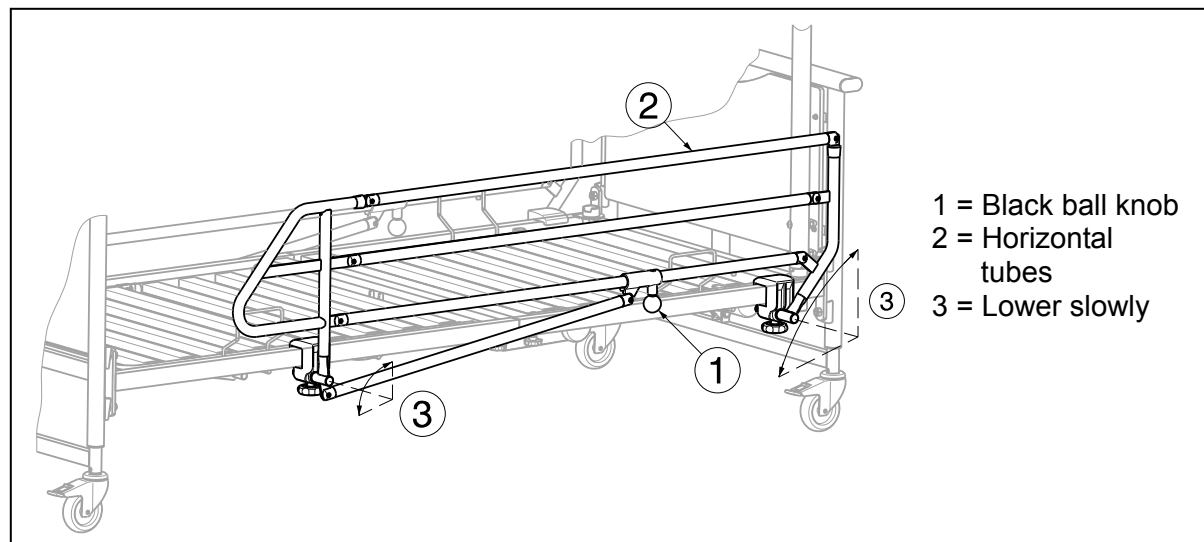
## 2.7 Use of the side railings

The side railings can be lowered on one side.

- **Lowering**
  - ⚠ **WARNING: Risk of injury - Secure the side railings against falling involuntarily.**
  - ⚠ **WARNING: Risk of injury - No objects or body parts may extend above the lying surface!**
  - ⚠ **WARNING: Risk of injury - Do not lower any of the side railings on one side if the patient is not supervised.**

To lower the side railings, pull the black ball knob ① out and push the horizontal tubes ② at the same time down, in the direction of the footboard.

To adjust the side rails back in the upright position, pull the horizontal tubes ② upwards, in the direction of the headboard. The ball knob snaps automatically.

Make sure that the bed's functions are not impaired when raising or lowering the side-rails, for example when using wedges to ensure the patient's safety.

## 2.8 Transfer in and out the bed

- ⚠ **CAUTION: Risk of falling - In case you cannot perform the transfer in a safe manner, ask someone to assist you.**
1. Position you as close as possible to the bed.
  2. Check the brakes from the bed are in the on position.
  3. Remove or fold down the side railings of the bed.
  4. Transfer to/from the bed.

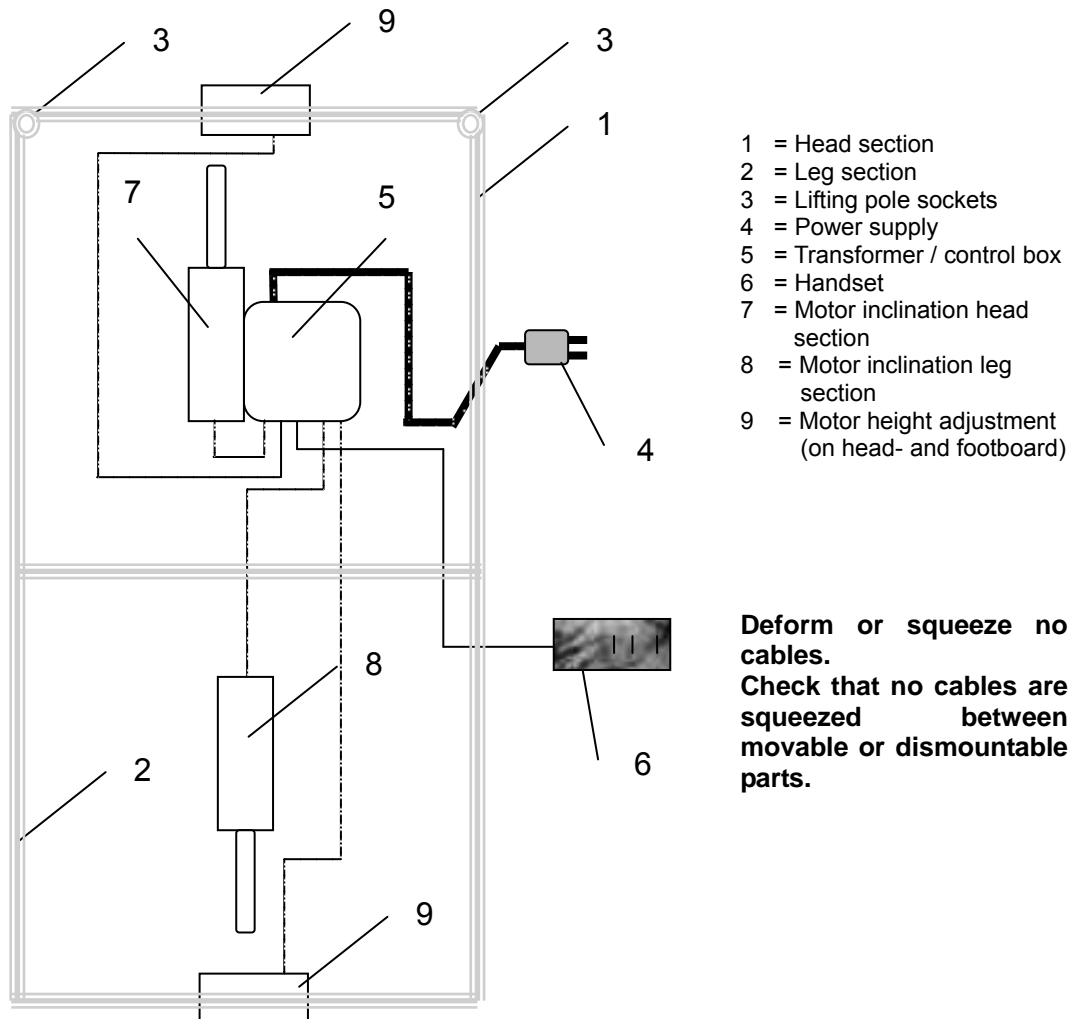
## 2.9 Transport

- ⚠ **WARNING: Risk of injury – Make sure that the bed is attached properly in the car or truck.**
- ⚠ **WARNING: Risk of injury – Transportation of the bed shall be only without the patient.**

To avoid damage when transported, the lifting pole should be detached. Unplug the power cable and check that no cables hang near the floor.

## 2.10 Cable placement

**⚠ WARNING: Risk of electrical shock - Ensure that no cables become squeezed.**

## 3 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

**⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

### 3.1 Tools

To set up the bed the following tools are needed.

- Allen key
- Screwdriver

### 3.2 Manner of delivery

The Vermeiren 2030 beds shall be delivered with:

- |  |        |                           |
|--|--------|---------------------------|
| • Head or front board, electric              | (976D) | Approx. 14,4 kg/piece     |
| • Foot or rear board, electric               | (976D) | Approx. 14,4 kg/piece     |
| • Head section electric                      | (977D) |                           |
| Without PVC-slat                             |        | Approx. 15,85 kg/piece    |
| With PVC slat                                |        | Approx. 17,65 kg/piece    |
| • Leg section, electric                      | (977D) |                           |
| Without PVC-slat                             |        | Approx. 16,50 kg/piece    |
| With PVC slat                                |        | Approx. 19,20 kg/piece    |
| • Set PVC-slat for mattress support platform |        | Approx. 4,54 kg/15 pieces |
| • Side railings                              | (989C) | Approx. 8,26 kg/piece     |
| • Lifting pole + triangular grip             | (968)  | Approx. 7,45 kg/piece     |
| • Tools                                      |        |                           |
| • Instruction manual                         |        |                           |

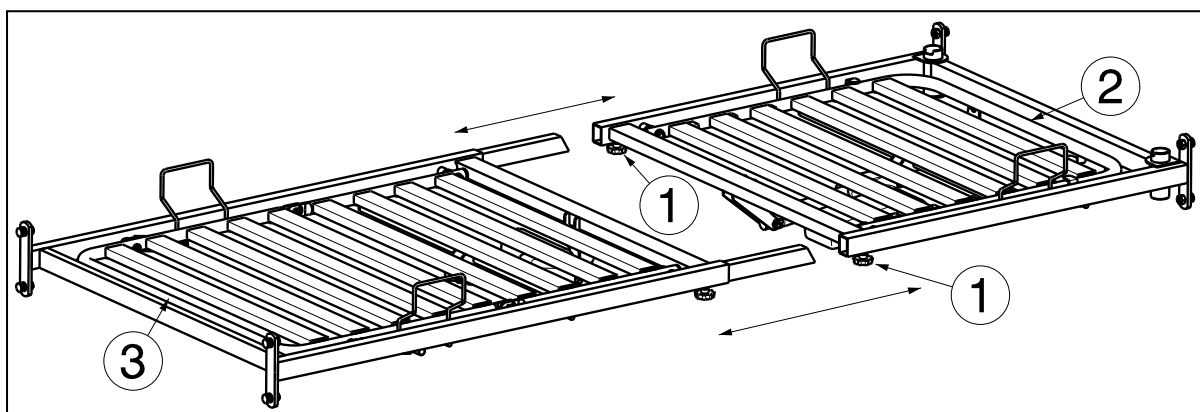
The nursing bed may only be used with the parts listed here. If parts are omitted or if foreign furnishings, components or electrical systems are included, then this bed is not regarded as a nursing bed supplied by the manufacturer, and you use it at your own risk. Then the CE conformity and the warranty of the manufacturer do not apply.

### 3.3 Assembly

- ⚠ **WARNING:** Risk of injury - Check all star knobs, screws are secured before using the bed.
- ⚠ **CAUTION:** Chance of pinching - Do not place fingers between the components (moving) of the bed.

Please take note of the following: It offers assistance in the assembly of your nursing bed.


#### 3.3.1 Attachment head section and leg section

1. Loosen the star knobs of the head section a little bit ① to make it easier to slide into each other.
2. Slide the 2 sections of the mattress support platform, head section ② and leg section ③, into each other.
3. Retighten the star knobs ①. Check also that the star knobs of the leg section are firmly secured.

### 3.3.2 Mounting head and footboard to the mattress support platform

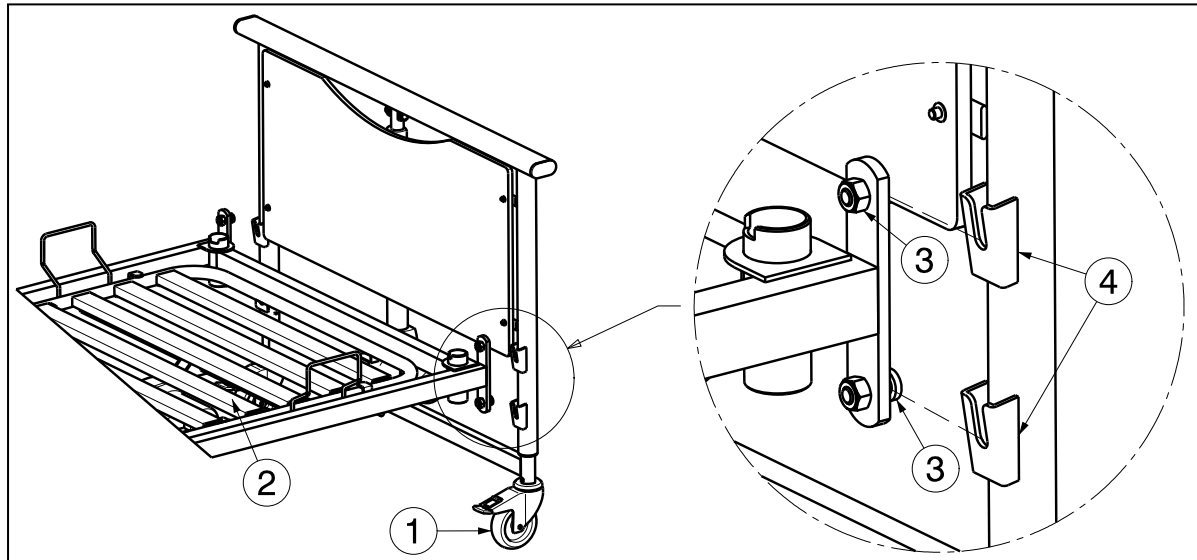
**⚠ CAUTION: Risk of injury - Lock the wheels of the head and footboard.**

1. Lock the wheels ① of the head and footboard.
2. Lift the mattress support platform ② with 2 persons and keep the bed at a slight angle.
3. Slide the attachments of the mattress support platform ③ in the holders of the head and footboard ④.

### 3.3.3 Side rails

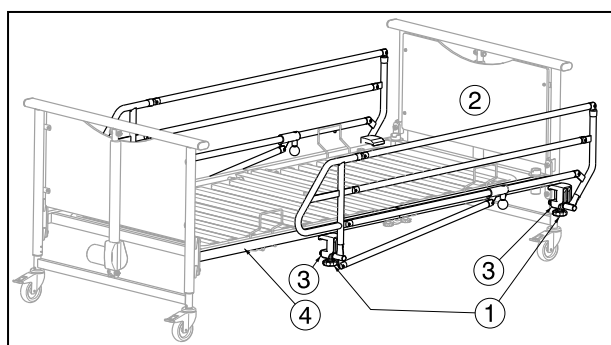**⚠ WARNING: Risk of clamping - Be careful that your head and other body parts are not pinched between the front of the side rails and the headboard.**


Mounting side rails:

1. Loosen the star knobs ① from the plastic blocks ③.
2. Grasp the side rail that the round part of the side rail is pointed to the footboard ② from the nursing bed.
3. Slide the plastic blocks ③ over the side tube of the mattress support platform ④.
4. Retighten the star knobs ①.

Mount the side rails within the predetermined distances according to standard EN 60601-2-52.

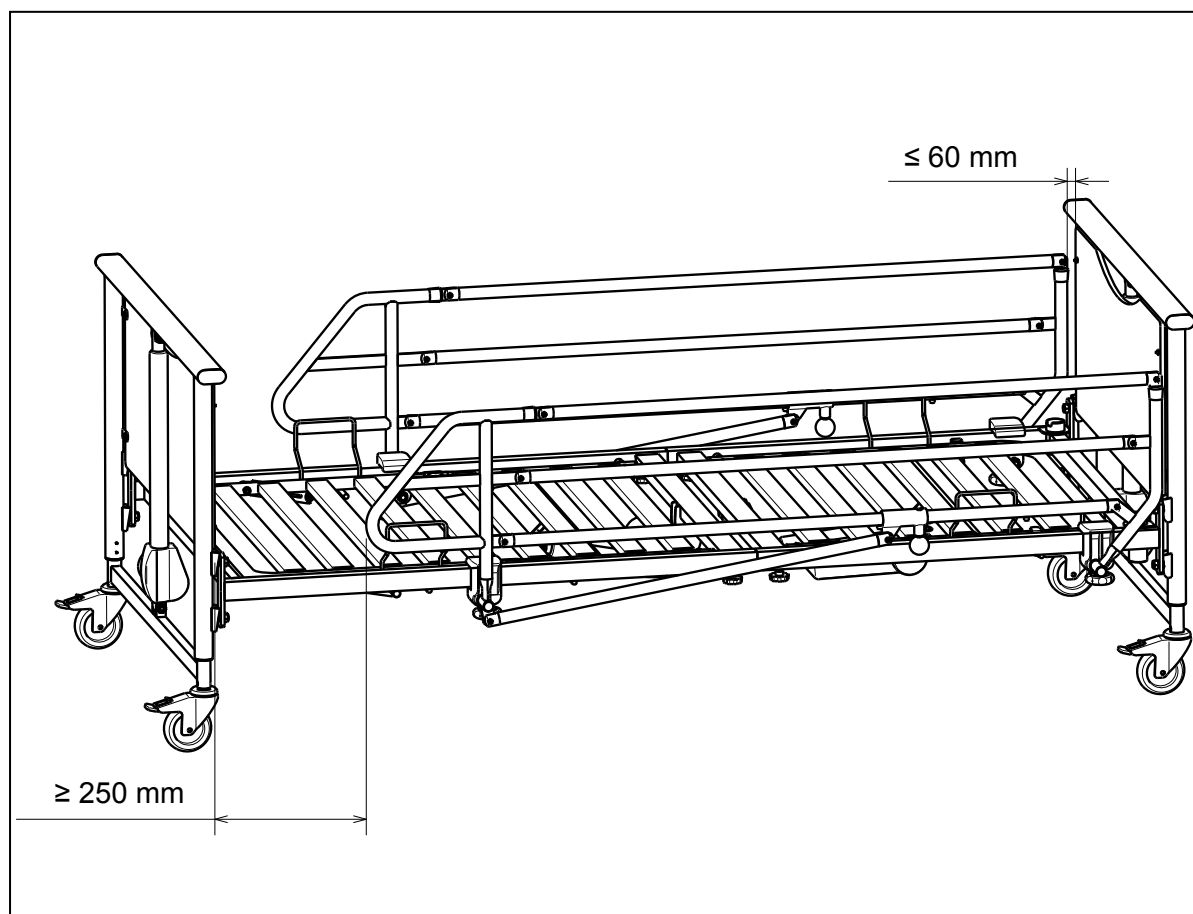The horizontal distance between headboard and side rail must be smaller or equal to 60 mm  
The horizontal distance between footboard and side rail must be greater or equal to 250 mm.


### 3.3.4 Lifting pole

**⚠ CAUTION: Risk of tipping over - Check that the lifting pole is not mounted to the outside of the welded bushing of the mattress support platform. If it is placed to the outside there is a risk of tipping over the nursing bed.**

The lifting pole can be mounted on both sides of the head section.

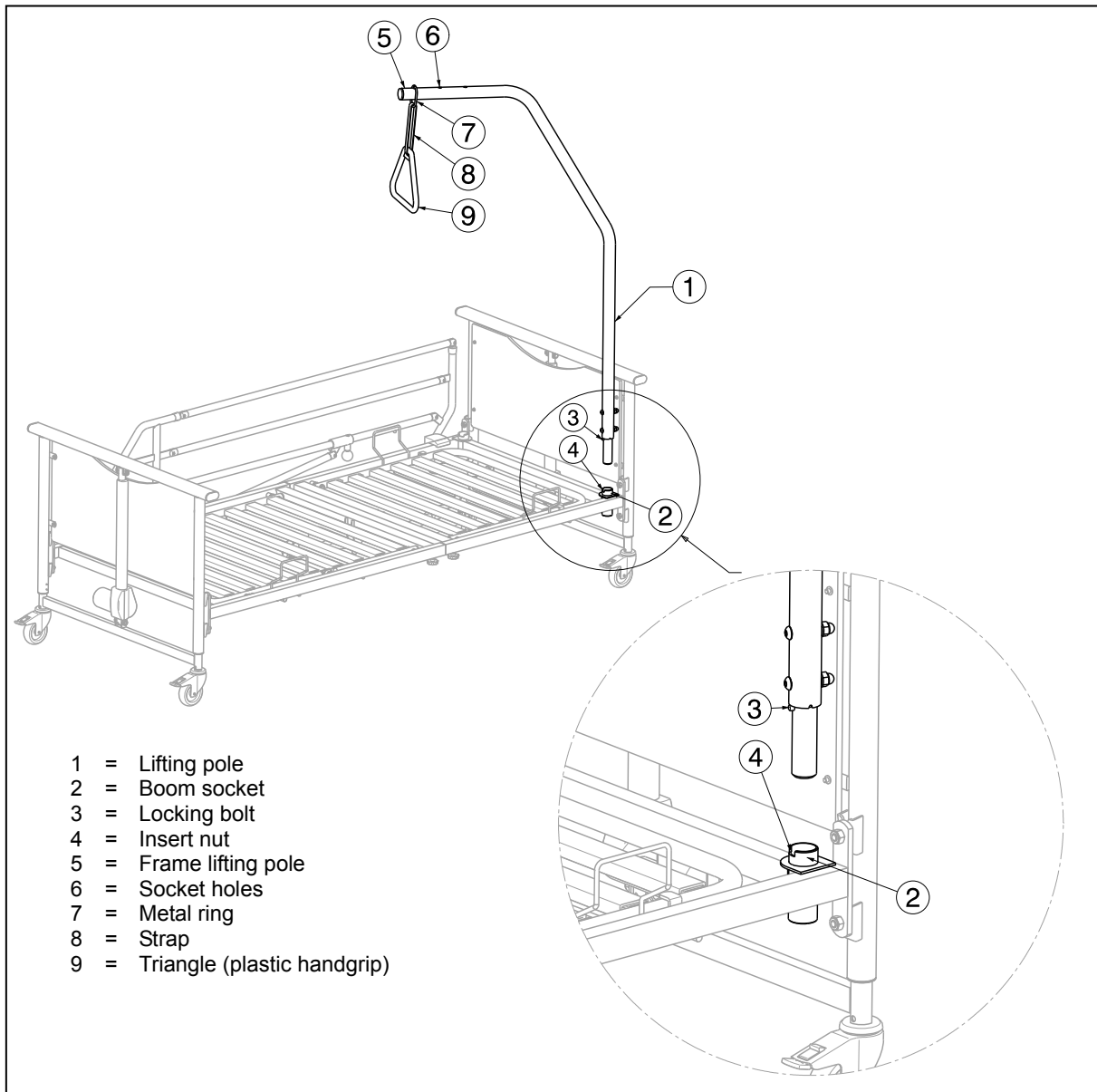1. Insert the lifting pole ① as far as it will go into the relevant socket ② on the head end of the mattress support platform.
2. Let the arresting bolt ③ slide into the groove ④.
3. Pull the metal ring ⑦ over the end of the upper frame ⑤ until the bolts engage one of the socket holes ⑥.
4. Also pull strap ⑧ into the metal loop and through the triangle ⑨; then fix its length.
5. Secure the strap.


### 3.3.5 Transformer connections

- ⚠ **CAUTION: Risk of injury - Do not touch the plug with wet hands.**
- ⚠ **CAUTION: Risk of injury - Do not place any objects on the connection and the cable, and protect these against mechanical loads.**

A 220 V / 230 V, 50 Hz power socket conforming to the VDE specifications is required. Take hold of the plug only to remove it from the power socket. The power cable must be laid without any loops or kinks and should not pass around sharp edges. The power cable leading to the handset must be protected against squeezing.

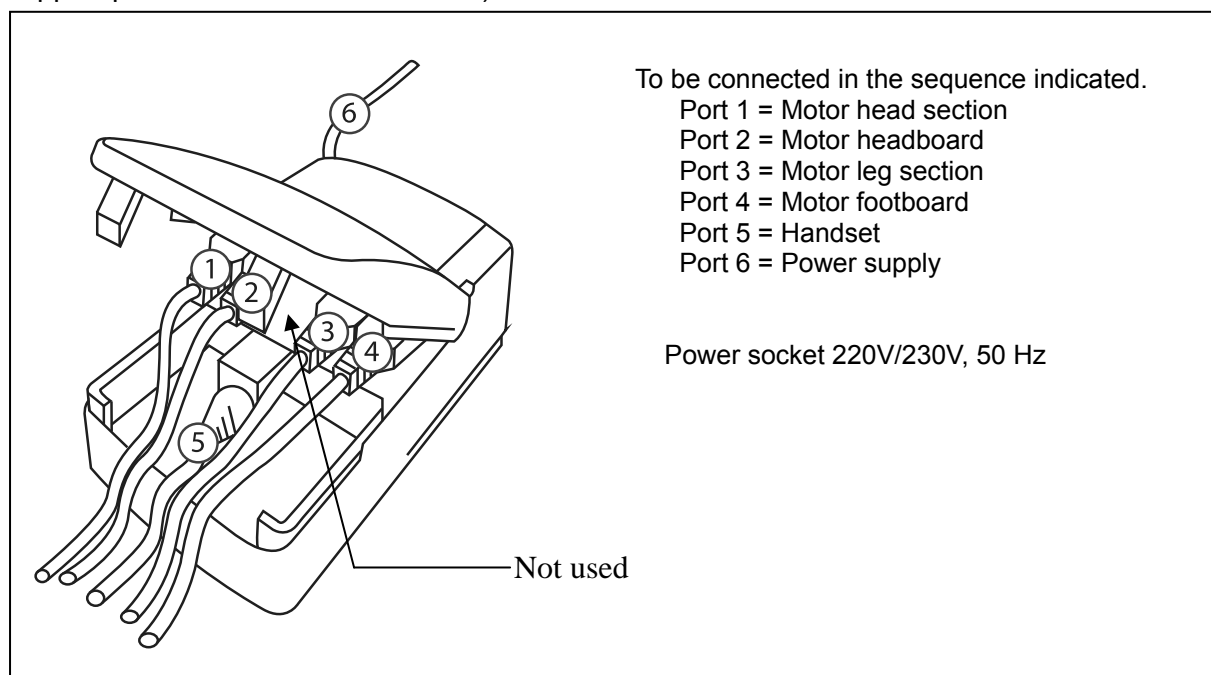Diagram for the transformer connections (the transformer is located below the mattress support platform on the head section):

### 3.4 Disassembly

1. Lower the nursing bed to the lowest horizontal position.
2. Lock at least 2 castors by using the brakes.
3. Remove the lifting pole.
4. Remove the side rails.
5. Remove the plug of the power cable out the supply socket.
6. Disconnect all cables out the transformer.
7. Lift the mattress support platform out the grooves of the head- and footboard.
8. Loosen the star knobs of the headrest and slide the headrest section out the leg section.

## 4 Maintenance

For the maintenance manual of the beds refer to the Vermeiren website:  
[www.vermeiren.com](http://www.vermeiren.com).

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**CONTRACTUAL WARRANTY**

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

**APPLICATION CONDITIONS**

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

**EXCEPTIONS**

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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---

## Notes

A series of horizontal dotted lines for taking notes.

---

## Notes

A series of horizontal dotted lines for taking notes.

## **Belgium**

### **N.V. Vermeiren N.V.**

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)

## **France**

### **Vermeiren France S.A.**

Z. I., 5, Rue d'Ennevelin  
F-59710 Avelin  
Tel: +33(0)3 28 55 07 98  
Fax: +33(0)3 20 90 28 89  
website: [www.vermeiren.fr](http://www.vermeiren.fr)  
e-mail: [info@vermeiren.fr](mailto:info@vermeiren.fr)

## **Italy**

### **Vermeiren Italia**

Viale delle Industrie 5  
I-20020 Arese MI  
Tel: +39 02 99 77 07  
Fax: +39 02 93 58 56 17  
website: [www.vermeiren.it](http://www.vermeiren.it)  
e-mail: [info@vermeiren.it](mailto:info@vermeiren.it)

## **Poland**

### **Vermeiren Polska Sp. z o.o**

ul. Łączna 1  
PL-55-100 Trzebnica  
Tel: +48(0)71 387 42 00  
Fax: +48(0)71 387 05 74  
website: [www.vermeiren.pl](http://www.vermeiren.pl)  
e-mail: [info@vermeiren.pl](mailto:info@vermeiren.pl)

## **Czech Republic**

### **Vermeiren ČR S.R.O.**

Nádražní 132  
702 00 Ostrava 1  
Tel: +420 596 133 923  
Fax: +420 596 121 976  
website: [www.vermeiren.cz](http://www.vermeiren.cz)  
e-mail: [info@vermeiren.cz](mailto:info@vermeiren.cz)

## **Germany**

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Wahlerstraße 12 a  
D-40472 Düsseldorf  
Tel: +49(0)211 94 27 90  
Fax: +49(0)211 65 36 00  
website: [www.vermeiren.de](http://www.vermeiren.de)  
e-mail: [info@vermeiren.de](mailto:info@vermeiren.de)

## **Austria**

### **Vermeiren Austria GmbH**

Schärddinger Strasse 4  
A-4061 Pasching  
Tel: +43(0)7229 64900  
Fax: +43(0)7229 64900-90  
website: [www.vermeiren.at](http://www.vermeiren.at)  
e-mail: [info@vermeiren.at](mailto:info@vermeiren.at)

## **Switzerland**

### **Vermeiren Suisse S.A.**

Eisenbahnstrasse 62  
3645 Gwatt (Thun)  
Tel: +41(0)33 335 14 75  
Fax: +41(0)33 335 14 67  
website: [www.vermeiren.ch](http://www.vermeiren.ch)  
e-mail: [info@vermeiren.ch](mailto:info@vermeiren.ch)

## **Spain / Portugal**

### **Vermeiren Iberica, S.L.U.**

Carratera de Cartellà, Km 0,5  
Sant Gregori Parc Industrial Edifici A  
17150 Sant Gregori (Girona)  
Tel: +34 972 42 84 33  
Fax: +34 972 40 50 54  
website: [www.vermeiren.es](http://www.vermeiren.es)  
e-mail: [info@vermeiren.es](mailto:info@vermeiren.es)

## **Manufacturer**

### **Vermeiren GROUP**

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Tel: +32(0)3 620 20 20  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)