# USER MANUAL

**VICAIR®**

**CENTRE RELIEF**

relief by air

<b>EN</b>	USER MANUAL	3
<b>NL</b>	GEbruikersHANDLEIDING	8
<b>DE</b>	GEBRAUCHSANWEISUNG	13
<b>FR</b>	MANUEL DE L'UTILISATEUR	18
<b>DA</b>	BRUGERVEJLEDNING	23
<b>SV</b>	BRUKSANVISNING	28
<b>NO</b>	BRUKERMANUAL	33
<b>FI</b>	KÄYTTÖOHJE	38
<b>ES</b>	MANUAL DEL USUARIO	43
<b>PT</b>	MANUAL DE UTILIZAÇÃO	48
<b>IT</b>	MANUALE UTENTE	53
<b>GR</b>	ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ	58
<b>HR</b>	KORISNIČKI PRIRUČNIK	63
<b>SL</b>	UPORABNIŠKI PRIROČNIK	68
<b>PL</b>	INSTRUKCJA OBSŁUGI	73
<b>ZH</b>	用户手册	78
<b>JP</b>	ユーザーマニュアル	82

# EN VICAIR® CENTRE RELIEF USER MANUAL

Dear customer,

Thank you for choosing a Vicair® product. We are sure you will appreciate the simplicity, reliability and fantastic effect. For quick installation you can use the Quick Installation Guide provided with your cushion. In addition, please read and understand this user manual before use. Save both the Quick Installation Guide and this manual for future reference.


For more information or to (re)order manuals, contact your country's distributor.

Go to [vicair.com](http://vicair.com) for distributor information.

## INTRODUCTION

The Vicair Centre Relief is a wheelchair seat cushion that provides a very high level of skin protection for persons with a high risk of developing pressure ulcers. It is used to completely offload the rectal area in cases of localized deep pressure ulcers, to provide comfort and stable body support. It is a high performance addition to a pressure ulcer treatment plan (stage I-IV).

## MAIN COMPONENTS

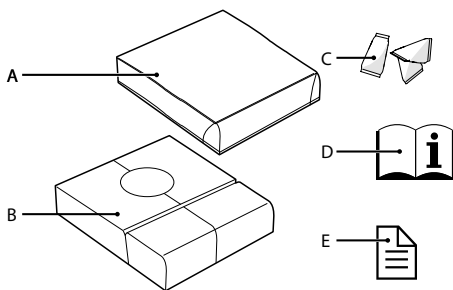- A** Cushion cover
- B** Cushion
- C** SmartCells™, placed in 5 compartments
- D** User manual
- E** Quick Installation Guide

## GENERAL PRODUCT SPECIFICATIONS

**Product name:** Vicair Centre Relief

**Manufactured by:** Vicair B.V., The Netherlands

**Warranty:** 24 months

**Weight (excl. cover):** Smallest size: 0,5 kg  
Medium size: 0,8 kg  
Largest size: 1 kg

**Available sizes:** See the grid in the back of the manual

**Used materials:** All latex free

**Cushion:** Polyester fabric with polyurethane coating

**Filling:** Air-filled SmartCells with low friction coating

### Cushion Covers:

**Comfair Cover:** High quality stretchable knitted double face fabric, with a bottom of Polyester fabric with polyurethane anti-slip coating.

**Incontinence Cover:** Elastic fabric, coated with fluid resistant breathable polyurethane, with a bottom of polyester fabric with polyurethane anti-slip coating.

All covers: Machine washable. Covers comply with EN1021-1, 1021-2 and CAL117.

## PRECAUTIONS AND SAFETY INSTRUCTIONS

---

A Vicair wheelchair seat cushion may only be used as a wheelchair seat cushion. The use of this product in any other way, is considered to be misuse. Vicair B.V. cannot be held responsible for damage caused by misuse, improper use or negligence. Make sure that you read and understand this manual completely.

**Warning!** Vicair B.V. recommends to install the Vicair Centre Relief under direct supervision of a therapist or technical advisor.

**Warning!** Your tailbone and pelvic bones should not touch the bottom of the cushion. Checking for “bottoming out” should be done during first use and after adjustments to the cushion filling.

**Warning!** Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or technical advisor.

**Warning!** Vicair B.V. recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

**Warning!** No testing data is currently available about the safe use of Vicair wheelchair cushions in motor vehicles. Therefore we do not encourage using the wheelchair cushion in motor vehicles.

**Warning!** In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination.

**Warning!** Every three months, check the product for wear and deflated cells. In case of serious damage, or a large number of deflated cells, do not continue to use the product. Consult your dealer.

**Caution!** Do not use or leave the product close to a naked flame or a high temperature heat source.

**Caution!** Do not steam clean or autoclave the product. The product cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product and/or the product's cover will age more rapid.

**Caution!** Do not wash this cushion in a washing machine.

**Caution!** Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

**Caution!** Make sure the cushion cannot interfere with the wheel spokes.

## INSTALLATION

---

**Warning!** Vicair B.V. recommends to install the Vicair Centre Relief under direct supervision of a therapist or technical advisor.

**Caution!** Make sure the cushion cannot interfere with the wheel spokes.

**Notice!** Always use cushion with cover.

**Notice!** Make sure that the wheelchair seat is clean and degreased.

**Tip:** When you prefer not to use the Velcro, you can remove the Velcro strips. Note that the strips confected to the bottom of the cushion cover cannot be removed.

### Perform the following steps when installing the Vicair Centre Relief

These steps correspond to the installation images on the Quick Installation Guide:

- Check cushion size. Width (X) of the cushion and seat should match. When the cushion rests against armrests or side-guards, the width may be up to 2,5 cm (1") wider. Depth (Y) of the cushion and seat should match.
- Check if the size of the cover matches the size of the cushion.
- To offload the rectal area, adjust the number of SmartCells in the centre circle compartment. The number of cells removed depends on the required level of offloading.

- Insert cushion in cover.
- Pass loop through eyelet in cover.
- Close zipper. Make sure to not damage the cushion.
- Remove protective backing tape from self-adhesive Velcro strips attached to the cushion cover.
- Place cushion by pressing firmly.
- Sit on the cushion for a few minutes, to allow the cushion to adapt to your body shape.
- If necessary, adjust footrest(s).

#### Checking for “bottoming out”:

**Warning!** Your tailbone and pelvic bones should not touch the bottom of the cushion. Checking for “bottoming out” should be done during first use and after adjustments to the cushion filling.

**Warning!** Vicair B.V. recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

- Check for bottoming out, after sitting on the cushion for approximately 5 minutes.
- While leaning forward, let your therapist or technical advisor slide a hand between your buttocks and the Vicair cushion.
- Ask your therapist or technical advisor to check if there is at least 2,5 cm / 1” of space between your lowest bony prominences and the seat surface underneath the cushion. This prevents them from touching the wheelchair seat.
- Do not use the cushion if bottoming out occurs. In most cases, adjusting the filling grade of the compartments will solve the problem.

## USE

---

#### Checking for redness of the skin

**Warning!** Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or technical advisor.

#### Adjusting the filling grade

**Warning!** Vicair B.V. recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

#### Ready for use

This Vicair product is ready for use. It has its own unique filling grade, depending on size and model. However – if necessary – the filling grade can be adjusted by removing or adding SmartCells from/to the various compartments. To offload the rectal area, you remove SmartCells from the centre circle compartment (if necessary). You can find the default filling grade on the label attached to the cushion.

#### Conditions for use and storage

- **User weight:**  
Minimum: 15 kg, maximum: 250 kg

- **Temperature:**  
Minimum: -20 °C, maximum: 55 °C

- **Use at high altitude**

**Notice!** Vicair B.V. recommends not to use the cushion above 2400 m (8000 ft) above sea level or below 50 m (164 ft) below sea level. Because each SmartCell is filled with air, the cushion may behave differently when used at high altitude or in an airplane. Under such circumstances, the cushion’s firmness can be adjusted by removing a few cells from each compartment.

- **Air humidity:**  
Minimum: 0%, maximum 100%

#### Daily fluff-up

Vicair B.V. recommends to fluff-up your Vicair cushion before use on a daily basis.

#### Use of Vicair seat cushions in motor vehicles

**Warning!** No testing data is currently available about the safe use of Vicair wheelchair cushions in motor vehicles. Therefore we do not encourage using the wheelchair cushion in

motor vehicles. As an alternative for your Vicair wheelchair cushion, the Vicair AllRounder (which can be fastened to the user's body) can be used if you require additional protection during transportation.

### Re-use – Transferring the product to another user

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. The product has to be thoroughly cleaned and disinfected, before another person is allowed to use the it. See chapter 'Maintenance and cleaning' for instructions. **Notice!** Make sure the product is correctly adjusted to the other person.

### Replacing the cushion cover

Under normal use, Vicair B.V. recommends to replace the cushion cover every year. Replacement covers can be purchased separately.

### Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulation. Ensure the product is cleaned prior to disposal to avoid any risk of contamination.

## MAINTENANCE AND CLEANING

---

**Warning!** Every three months, check the product for wear and deflated cells. In case of serious damage, or a large number of deflated cells, do not continue to use the product. Consult your dealer.

**Caution!** Do not steam clean or autoclave the product. It cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product will age more rapid.

**Caution!** Do not use aggressive or abrasive

cleaners, or sharp objects to clean the cushion. **Caution!** Do not wash this cushion in a washing machine.

### Cleaning the Vicair Centre Relief

- **Cleaning the Vicair Centre Relief cushion cover**

The cushion cover can be machine washed. Follow the washing instructions on the label attached to the inside of the cover.

- **Cleaning the Vicair Centre Relief cushion**

Do not machine wash the cushion. Do not immerse the cushion in water or any other liquid. Follow the washing instructions on the label attached to the cushion. Remove the cover and wipe the cushion with a damp cloth using a mild detergent. Difficult spots or stains can be cleaned by first saturating the spot and then using a soft brush to loosen the dirt, or by machine washing the emptied cushion (**remove SmartCells**) at 60°C. If it is necessary to remove the cells for cleaning, note the number of cells in each compartment.

### Disinfecting your Vicair Centre Relief

**Warning!** In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination. Clean the cushion and cover as instructed above. Preferably use an anti-bacterial detergent. After cleaning spray your cushion and cover with a disinfectant. Bend compartments outwards to make sure the spray gets into the gaps.

## **WARRANTY**

---

Vicair B.V. offers a limited warranty against defects in workmanship and materials, provided the product has been used normally. Any damage arising from abnormal use, or caused by improper handling, cleaning, or negligence, is excluded from this warranty.

### **Warranty period**

From original date of purchase: 24 months.

Should a defect in materials or workmanship occur within the above mentioned warranty period, Vicair B.V. will – at its own choice – repair or replace the product free of charge. The cushion cover is excluded from this warranty. Remedies for breach of express warranties herein are limited to repair or replacement of the product.

In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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