User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

Grace

EN

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Verze: B, 2023-06

All rights reserved, including translation.

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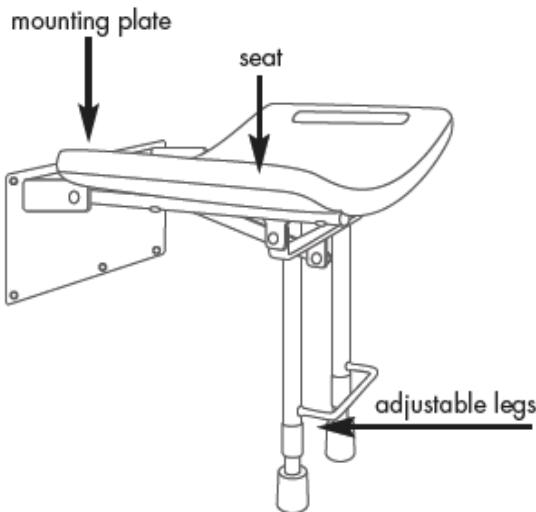
Shower bench Grace

1 Upon delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- 1 foldable shower bench
- 6 screws
- 6 wall plugs
- Manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

2 Mounting the shower bench

CAUTION

Risk of injuries

- Make sure that the shower bench is mounted securely to the wall. Check the fixation on a regular basis.
- Be sure that there are no electrical cables or other tubes in the mounting area of the shower bench.
- The plugs delivered with the product are suitable for concrete walls or full brick walls. For other types of walls, please inquire about the correct type of mounting equipment.

To install Grace:

1. Set the leg and rail height before securing the mounting plate to the wall. Ensure that both legs are of the same height.
To adjust the height, grab the horseshoe clip between forefinger and thumb and pull it out. Slide the leg out to the desired position and push the clip firmly back into the hole.
2. Mark the fixation holes on the wall.

3. Drill holes in the wall.
4. Mount the plugs.
5. Place the mounting plate against the wall.
6. Place the screws and fasten firmly.

If you have any doubts, do not hesitate to contact a professional to install Grace for you.

3 Using the bathroom bar

CAUTION

Risk of burns

- Be careful when using in extreme hot or cold environments (sunshine, extreme cold, saunas, etc.).
- If you notice any defects, refrain from using Grace.
- Be careful for your fingers when you fold away the shower bench.

The maximum user weight for the shower bench is 130 kg.

4 Maintenance

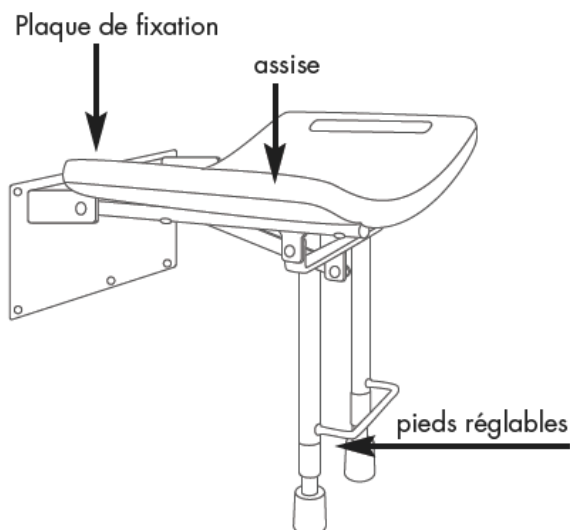
Grace can be cleaned and disinfected with normal household products. Avoid using abrasive cleaning products that can cause discoloration.

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

4.1 Expected lifespan

The shower bench is designed to have an average lifespan of 5 years.

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**Service registration form**

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Vermeiren GROUP
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com