# Installation manual

MANUEL D'INSTALLATION  
INSTALLATIEHANDLEIDING  
INSTALLATIONSANLEITUNG  
MANUALE DI INSTALLAZIONE  
MANUAL DE INSTALACIÓN  
INSTRUKCJA INSTALACJI  
PŘÍRUČKU PRO INSTALACI

## 2030

CE

# Contents

<b>Preface</b>	<b>4</b>
<b>1. This product</b>	<b>5</b>
<b>2. Scope of delivery</b>	<b>6</b>
<b>3. Transport</b>	<b>6</b>
3.1. Loading the transport trolley	6
<b>4. Assembling the bed</b>	<b>8</b>
4.1. Tools	9
4.2. Assembly	9
4.3. Disassembly	15

# Preface

To support you on the installation and repairs of this nursing bed, we provide you with this installation manual. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following bed(s):

- 2030

## Important note

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.


## Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Installation instructions For specialist dealer
	Service manual for beds For specialist dealer
	Drawings of (spare) parts For specialist dealer
	EC declaration of conformity

# 1. This product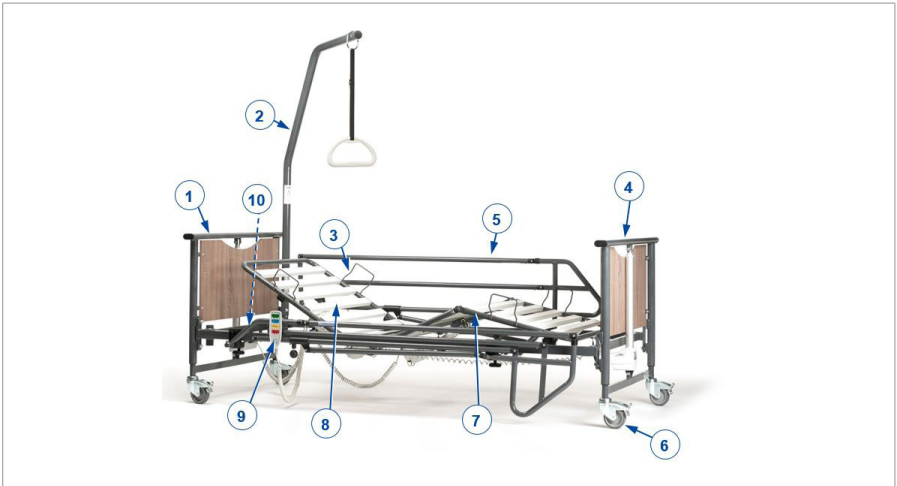

1. Head panel
2. Lifting pole / Trapeze handle
3. Mattress retainer
4. Foot panel
5. Side rail
6. Castor wheel with brake
7. Foot section
8. Head section
9. Handset
10. Identification plate

## 2. Scope of delivery

The following components are part of the delivery:

- Head and feet end incl. motors
- Slatted frame, foot-end incl. motor
- Slatted frame, head-end incl. transformer and motor
- Lifting pole / Trapeze handle
- Foldable side rail (L&R)
- Handset
- User manual
- Options

Before use, check if everything is included and that no products are damaged.

## 3. Transport

### CAUTION

#### **Risk of injuries or damage**

- Make sure that the bed is attached properly in the car or truck.
- Only transport the bed without the patient in it.


To avoid damage when transported, the lifting pole should be detached. Unplug the power cable and check that no cables hang near the floor.

### 3.1. Loading the transport trolley


#### CAUTION

#### **Risk of injuries or damage**


- Check if all parts are well secured.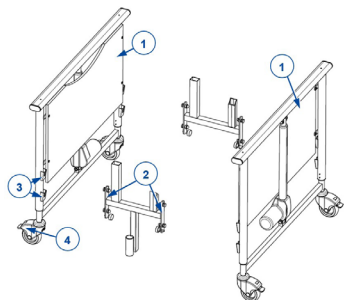


1. Grasp the panels (1).
2. Mount the attachment of the transport set (2) in the attachment of the panels (3).
3. Lock the brakes of the panels (4).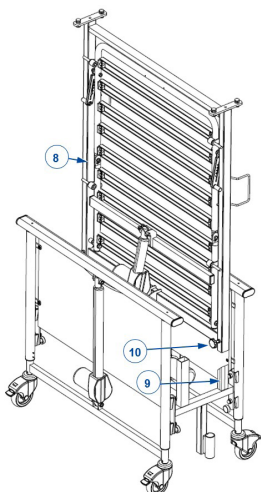

4. Grasp the foot end (8) and hold it with the motor located on the inside.
5. Place the foot end on the supports (9) of the transport set and tighten the star knob (10)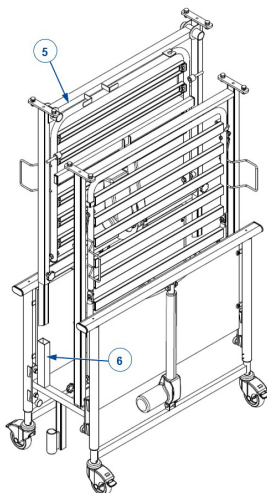

6. Grasp the head end (5) and hold this with the motor located to the bottom side.
7. Slide the tube of the head end (5) in the tube of the transport set (6).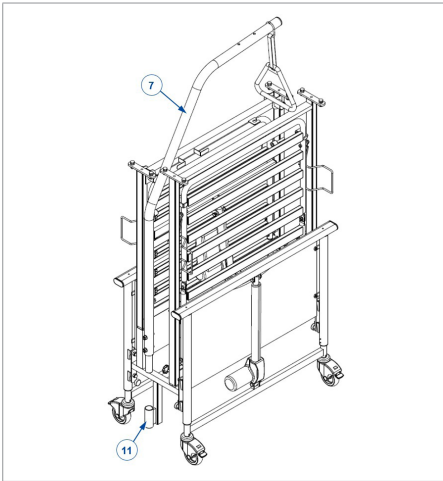

8. Place the lifting pole (7) in the receiver (11) of the transport set.

## 4. Assembling the bed

### **⚠ CAUTION**

#### **Risk of injuries or damage**

- The bed needs to be (dis)assembled by a specialist dealer according to the instructions in this chapter.
  - Only use the settings described in this manual.
  - Only use parts described in this manual.
  - Only use genuine spare or replacement parts approved by Vermeiren.
  - Prevent bystanders from accessing the working area.
  - Keep the (dis)assembling area clean and tidy, to prevent falling.
  - Collect all small parts in a box or bag.
  - While (dis)assembling and operating the bed, make sure no objects or body parts get caught between parts. In particular, pay attention to the position of the cables.
- i** Read the user manual of this bed. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

## 4.1. Tools


To set up the bed, the following tools are needed:

- Screwdriver
- Allen key set

## 4.2. Assembly


### 4.2.1. Slatted frame

1. Loosen the star knobs of the head section a little bit (1) to make it easier to slide into each other.
2. Slide the 2 sections of the mattress support platform, head section (2) and leg section (3), into each other.
3. Retighten the star knobs (1). Check also that the star knobs of the leg section are firmly secured.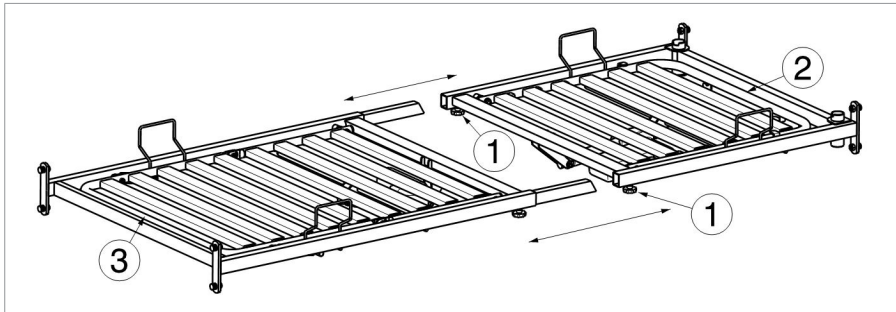


#### 4.2.2. Head panel / foot panel

1. Lock the wheels (1) of the head and footboard.
2. Lift the mattress support platform (2) with 2 persons and keep the bed at a slight angle.
3. Slide the attachments of the mattress support platform (3) in the holders of the head and footboard (4).
4. Once the frame is mounted, the mattress support sections can be released from their 3 locking clamps. Press the spring on the clamps and turn them downwards so the mattress support sections can move freely.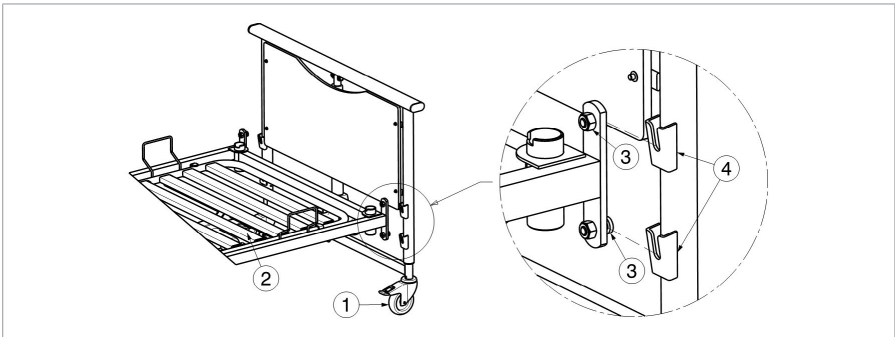

#### 4.2.3. Metal side rails

1. Place the hooks (1) over the frame tube (2).
2. Tighten the star knob (3). Make sure that both sides have been tightened securely.
3. Repeat these steps for the other side rail.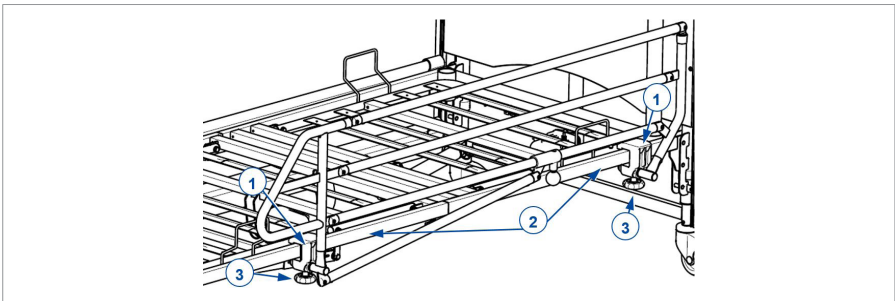

- i** The horizontal distance between headboard and side rail must be smaller or equal to 60 mm.


The horizontal distance between footboard and side rail must be greater or equal to 250 mm.

#### 4.2.4. Transformer connections

##### **⚠ CAUTION**

##### **Risk of injuries**

- Do not place any objects on the connection and the cable, and protect them against mechanical loads.
1. Connection requires a 220V/230V, 50Hz power socket conform to VDE specifications.
  2. The diagram for the transformer connections is given in the figure below. To insert the cables, open the top lid of the transformer. Otherwise the cables will not be secured into place.
  3. When all cables have been secured, close the top lid.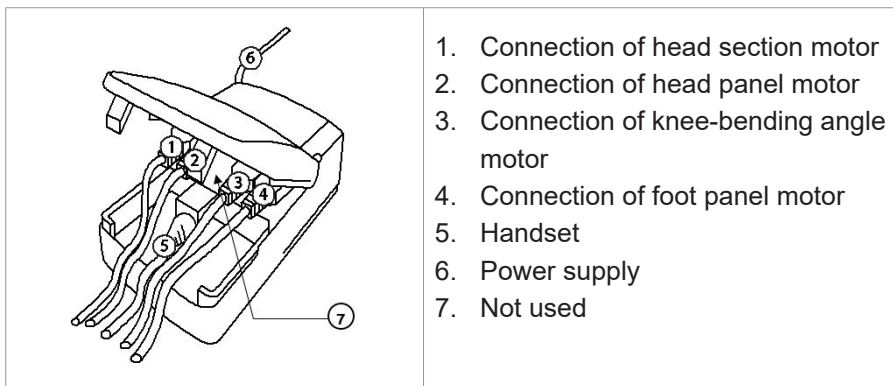

1. Connection of head section motor
2. Connection of head panel motor
3. Connection of knee-bending angle motor
4. Connection of foot panel motor
5. Handset
6. Power supply
7. Not used

Take hold of the plug only to remove it from the power socket.

The power connection must not have any loops or kinks and should not pass around sharp edges. Protect the power connection and the cable leading to the handset from being crushed.

## 4.2.5. Strain relief and power cable


### **⚠ CAUTION**

#### **Risk of damage**


- The power cable must be carefully positioned without kinks, without pulling, and without touching the floor.

Below the cross bar of the head slatted frame, there is a strain relief hook for the power cable.

1. Insert the relief unit of the power cable (4) into the suspension (3).
2. Pull the desired length of the power cable (1) in such a way that the power cable (2) can be laid under the mattress support platform without sagging. Be careful to avoid that the cable (2) come under the traction in any adjustment position.
3. Fasten the power cable by properly tightening the relief unit of the power cable (4).
4. Fasten the position of the relief unit by properly tightening the fixing screws (5).

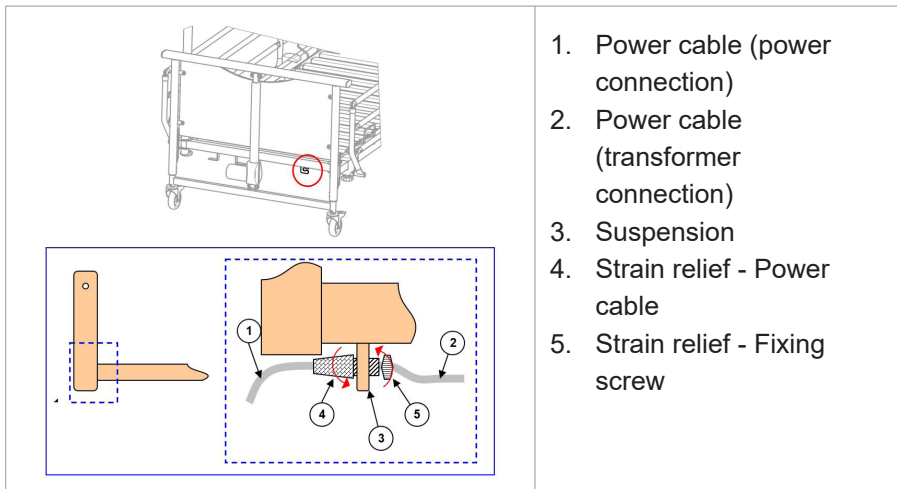
## 4.2.6. Cable placement
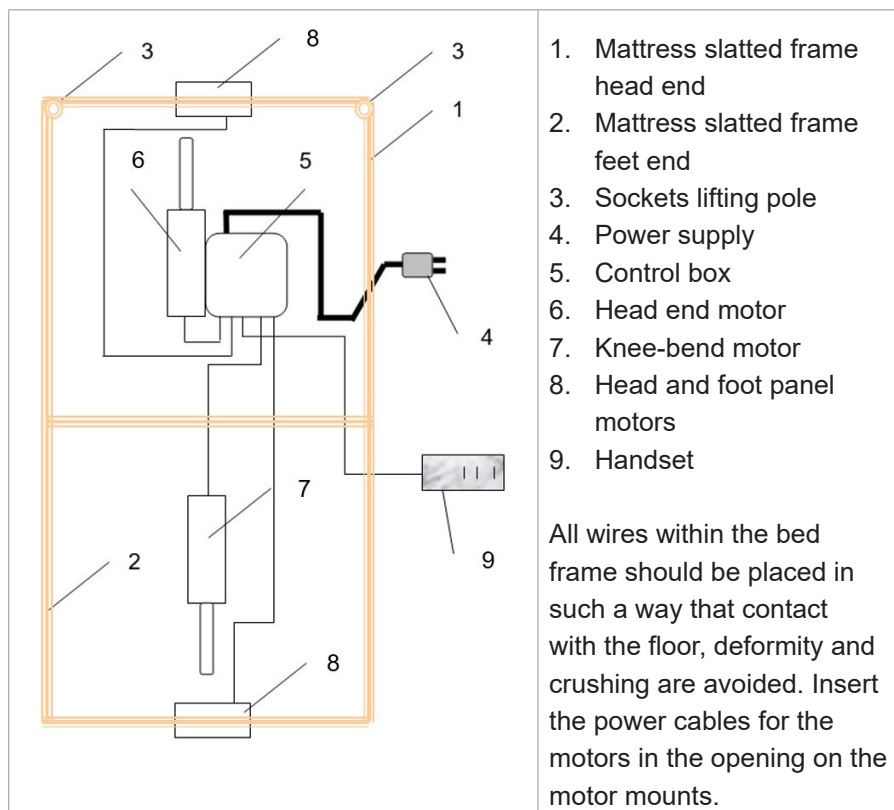
### 4.2.7. Lifting pole (optional)

The lifting pole can be mounted on either side of the head end.

1. Insert the pole (1) as far as it will go into the relevant socket (2).
2. Let the arresting bolt (3) slide into the groove (4).
3. Pull the metal ring (7) over the end of the upper frame (5) until the bolts engage one of the socket holes (6).
4. Also pull strap (8) into the metal loop and through the triangle (9); then fix its length.
5. Secure the strap.

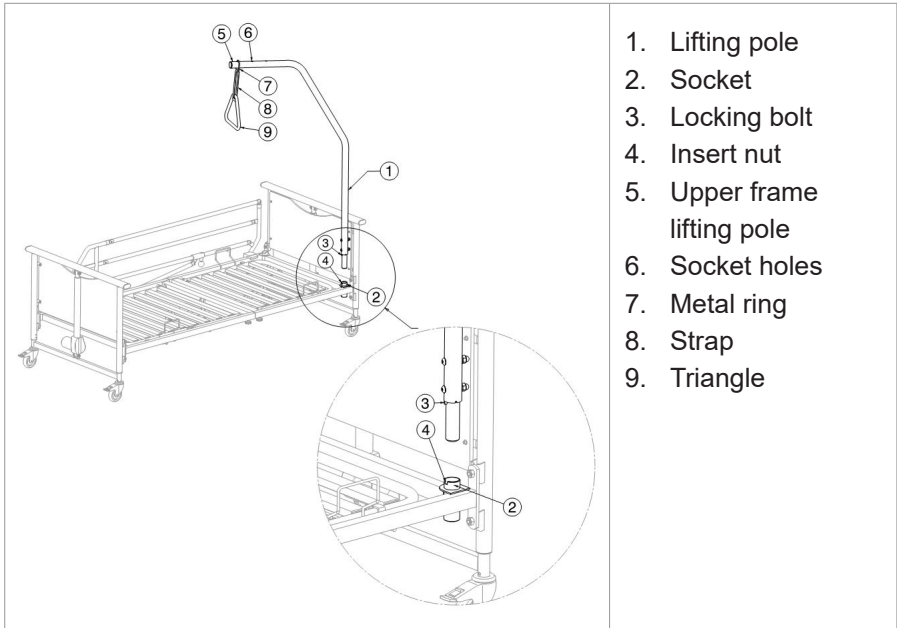### 4.3. Disassembly

1. Lower the nursing bed to the lowest horizontal position.
2. Lock at least 2 castors by using the brakes.
3. Remove the lifting pole.
4. Remove the side rails.
5. Remove the plug of the power cable out the supply socket.
6. Disconnect all cables from the transformer.
7. Lift the mattress support platform out the grooves of the head- and footboard.
8. Loosen the star knobs of the headrest and slide the headrest section out of the leg section.

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All rights reserved, including translation.