Manual EN

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Manuel FR

Anleitung DE

Manual ES

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Bruksavisning NO

**Declaration of
Conformity** Doc

IMPULSE-II SMART

24/6

24/8

The IMPULSE-II SMART has been designed with double insulation. We advise you to carefully read this instruction manual as well as to observe the Safety Instructions before taking the IMPULSE-II SMART into operation.

1. SYMBOLS

	<p>A triangle containing an exclamation mark indicates important information in these operating instructions which is to be strictly followed.</p>
	<p>Read instruction manual.</p>
	<p>Double insulation.</p>
	<p>Products containing hazardous substances are marked by this symbol. This symbol also indicates that it is prohibited to dispose of these apparatuses in the household waste.</p>
	<p>This product is CE-tested and thus meets the EMC directive 2014/30/EU and the low-voltage directive 2014/35/EU.</p>
	<p>Indoor use only.</p>

2. SPECIFICATIONS

Primary voltage	: 230Vac - 50/60Hz - 1-phase
Secondary nominal voltage	: 24V
Charge voltage 'absorption'	: 28,8V
Charge voltage 'float'	: 27,6V
Charge voltage 'storage'	: 26,4V
Maximum charge current	: IMPULSE-II SMART 24/6: 6A; : IMPULSE-II SMART 24/8: 8A
Battery type	: Standard: Gel, AGM Lithium or custom settings available via Bluetooth with the VictronConnect smartphone app.
Recommended battery capacity	:

	Min.	Max. (in order to charge 80% capacity within 8 hours)
IMPULSE-II SMART 24/6	40Ah	60Ah
IMPULSE-II SMART 24/8	60Ah	85Ah

Protection	: - output short circuit, over voltage and reversed polarity - high temperature
Input power	IMPULSE-II SMART 24/6: 180W, IMPULSE-II SMART 24/8: 240W
Efficiency	: min. 90% (at full load)
Ambient temperature	: -20°C to +40°C
Protection class	: IP44 or IP65
Dimensions (hxwx d)	: 60x105x190mm
Safety class	II
Weight	: 1kg
Storage temperature	: -30°C to +50°C
Humidity	: max. 95% (non-condensing)

3. SAFETY INSTRUCTIONS

- Always provide proper ventilation during charging.
- Avoid covering the charger.
- Never try to charge non-rechargeable - or frozen batteries.
- Never place the charger on top of the battery when charging.
- Prevent sparks close to the battery. A battery being charged could emit explosive gasses.
- Battery acid is corrosive. Rinse immediately with water if acid comes into contact with skin.
- This appliance is not designed for use by young children or people who cannot read or understand the manual unless they are under the supervision of a responsible person to ensure that they can use the battery charger safely. Store and use the battery charger out of the reach of children, and ensure that children cannot play with the charger.
- Connection to the mains supply must be in accordance with the national regulations for electrical installations. In case of a damaged supply cord please contact the manufacturer or your service agent.

4. BATTERY INSTRUCTIONS

- We recommend to recharge the batteries after every trip.
- Interrupting the charge process by disconnecting the charger is allowed.
- We recommend to keep the charger connected to the batteries after charging. The charger will automatically perform a short refresh charge every week to keep the batteries fully charged.
- Batteries should be recharged every month to keep them in fully charged condition.
- Deep discharge of the batteries should be avoided because it shortens service life.
- Deeply discharged batteries should be recharged immediately in order to avoid irreparable damage.

5. SETUP

When placing the charger it should be observed that it has sufficient ventilation all around. A space of at least 10cm should be allowed. If there is not sufficient ventilation or the temperature is high, the charger will lower the output current (resulting in longer charge time).

The enclosure is made of plastic and suitable as a table model.

During normal operation the enclosure will become warm. This is normal.

6. DC CONNECTION

The charging cable is supplied with an XLR connector, whereby pin 1 of the XLR-connector is the positive pole (+), pin 2 is the negative pole (-) and pin 3 is the drive inhibit signal.

If the battery is connected to the charger with reversed polarity, the charger will not start charging and both LEDs are blinking (error).

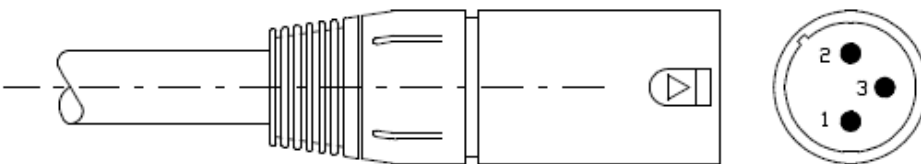
Figure 1 XLR-connector

7. OPERATION

- a) Connect the charger to the battery.
- b) Connect the charger to the AC mains.
- c) Charging will start: the red LED will switch on.
- d) When the battery is fully charged the green LED will switch on.

Note 1: When disconnecting the charger from the battery, it is allowed to leave the mains plug in the wall socket. The green LED will be on.

When connecting to the battery the green LED will switch off and the red LED will switch on.

Exception: if the battery is (nearly) fully charged the green LED will remain on.

Note 2: Both LEDs will blink in case of an error.

Note 3: If the charger is connected to the AC mains (b) but no battery was connected (a) the red LED will blink. When there is still no battery connected after +- 30min the charger will switch to Float. Then the red LED will switch off and the green LED will switch on.

Attention: If a battery is connected and charging is interrupted, this cannot be detected by the charger. The charger will indicate Charging (red LED on) or Full (green LED on).

8. INDICATIONS

	Red LED 	Green LED 	Note
Charger off (mains not connected)	○	○	
Charger on (mains connected)	☼	○	Bulk, Absorption, <u>no battery detected</u>
Charging	●	○	Bulk, Absorption, <u>battery detected</u>
Full	○	●	Float, Storage, Repeated absorption
Error	☼	☼	No blinking codes. Possible errors: <ul style="list-style-type: none"> • Short circuit • Reverse polarity • Over-voltage

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