User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

Inovys II + L70

Contents

Preface	4
1. Your product	6
1.1. Options	7
2. Before use	7
2.1. Intended use	7
2.2. General safety instructions	8
2.3. Symbols on the wheelchair	8
2.4. First use and storage	9
2.5. Transport	10
3. Using your wheelchair	13
3.1. Operating the brakes	13
3.2. Moving to/from the wheelchair	15
3.3. Comfort adjustments	16
3.4. Riding the wheelchair	21
4. Installation and adjustment	25
4.1. Tools	26
4.2. Scope of delivery	26
4.3. Assembly	26
4.4. Adjustments	27
5. Maintenance	28
5.1. Points of maintenance	28
5.2. Maintenance instructions	29
5.3. Troubleshooting	30
5.4. Expected lifespan	30
5.5. Reuse	30
5.6. End of use	31
5.7. Warranty	31
6. Technical specifications	31

Preface

Congratulations! You are now owner of a Vermeiren wheelchair!

This wheelchair is made by qualified and committed personnel. It is designed and produced according to high quality standards, guarded by Vermeiren.

Thank you for your trust in the products of Vermeiren. To support you on the use of this wheelchair and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your wheelchair.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety, and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual of L70 granule seat system For user and specialist dealer
	User manual of Inovys II For user and specialist dealer
	Installation instructions For specialist dealer
	Service manual for wheelchairs For specialist dealer
	EC declaration of conformity

1. Your product

1. Headrest
2. Backrest
3. Arm pads (optional)
4. Armrests
5. Seat
6. Leg rest
7. Footplates
8. Brakes
9. Front wheel
10. Frame
11. Rear wheels
12. Handgrips
13. Push bar
14. Tipping aid
15. Identification plate

1.1. Options

Contact your specialist dealer about options. He will gladly advise you.

2. Before use

2.1. Intended use

- This product is a medical device.
- The L70 granule seat system is designed to be used in combination with the Inovys II wheelchair with T30 wheel system and frame alterations for the granule seat system.
- Indications and contraindications: The Inovys II wheelchair with L70 granule seat system is intended for passive wheelchair users, such as elderly people or people with walking difficulties or no walking abilities (e.g. paralysis, loss of limbs, limb defects or deformations, stiff or damaged joints, heart insufficiencies and poor blood circulation, balance disturbances, cachexia (decrease in muscle), ...). The user has the wheelchair pushed by an attendant.
- This wheelchair is designed and produced solely to transport/transfer one **(1)** person with a maximum weight of 135kg. It is not designed to transport goods or objects, nor for any other use than previously described.
- This wheelchair is suitable for indoor and outdoor use.
- Only use accessories and spare parts approved by Vermeiren.
- Please read all technical details and limits of your wheelchair in chapter 6..
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

2.2. General safety instructions

 CAUTION

Risk of injuries and/or damage

- Carefully read and follow the instructions in this manual. Otherwise you may get injured or your wheelchair may get damaged.

Keep the following general warnings in mind during use:

- As the attendant, do not operate the wheelchair if you are under influence of alcohol, medicines or other substances that may influence your riding abilities.
- Be aware that some parts of this wheelchair may get very hot or cold due to ambient temperature, solar radiation, heating devices. Be careful when touching. Use protective clothing if the weather is cold.
- Do not modify your product in any way.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

Be aware that your wheelchair may interfere with some types of anti-theft systems, depending on the settings used. This may cause the shop's alarm to go off.

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

2.3. Symbols on the wheelchair

	Maximum weight of the user in kg
	Type designation
	Catalogue number

	Serial number
	Medical device
	Manufacturer
	Date of manufacture
	Declaration of conformity
	Attention: important information
	It is advised to read the manual
	Risk of entrapment
	For indoor use only (battery charger)
	Maximum safe slope in ° (degrees).
	Crash-tested product ; can be used as a seat in a motor vehicle
	Can be used as a seat in a motor vehicle ; indicates attachment points

2.4. First use and storage

CAUTION

Risk of damage

- Make sure that your wheelchair is stored in a dry environment to prevent mould from growing and the upholstery from being damaged, see also chapter 6..

2.5. Transport

2.5.1. Moving out of the way

Move the wheelchair by using its wheels and rolling it to the destination.

If this is not possible, and disassembling is not desired, ask help to lift the wheelchair. Firmly grasp the frame with both hands and lift it from the floor. Do not use foot or arm rests, or wheels to grasp the wheelchair.

2.5.2. Transport by vehicle, as luggage

WARNING

Risk of injuries

- NEVER use the same seatbelt to secure the wheelchair and passenger.
-  You can use a ramp to push the wheelchair inside the vehicle, see §3.4.3.. Apply the parking brakes after loading, see §3.1..

2.5.3. Transport by vehicle, as seat for the user

WARNING

Risk of injuries

- The wheelchair has passed the crash test of ISO 7176-19: 2022 and, as such, has been designed and tested for use only as forward-facing seat in a motor vehicle.
- The wheelchair's pelvic belt alone is not suited as an occupant restraint belt.
- Use the wheelchair's pelvic belt and the applicable three-points belt in the vehicle to prevent head and chest impacts with the vehicle.
- Do not use postural supports to restrain the user in the vehicle unless they are labelled as being in accordance with the requirements specified in ISO 7176-19:2022.
- Following involvement in any type of vehicle collision, have your wheelchair inspected by the specialist dealer or manufacturer's representative before reuse.

The wheelchair is tested using the four-point strap-tie system and a 3-point occupant-restraint system.

Whenever feasible, use the seat of the vehicle and store the wheelchair in the cargo area.

2.5.3.1. Steps to secure the wheelchair in a motor vehicle:

1. Check that the vehicle is equipped with a suitable wheelchair tie down and occupant-restraint system, conform ISO 10542.
2. Check that the components of the wheelchair tie down and occupant restraint system are not frayed, contaminated, damaged or broken.
3. If equipped with an adjustable seat and/or back tilt, make sure that the wheelchair user is sitting as upright as possible. If the user's condition prevents this, a risk assessment should be done to evaluate the user's safety during transit.
4. Remove all mounted accessories such as trays and respiratory equipment, and secure them in a safe place.
5. Position the wheelchair facing forward in the travelling direction, centrally between the tie-down rails mounted in the floor of the vehicle.
6. Mount the front securement straps according to the instructions of the strap-system manufacturer at the indicated place (figure 1). This place is marked on the wheelchair with a symbol (figure 2).
7. Roll back the wheelchair until the front straps are tight.
8. Apply the wheelchair brake.
9. Mount the back securement straps according to the instructions of the strap-system manufacturer at the indicated place (figure 1). This place is marked on the wheelchair with a symbol (figure 2).

Figure 1


Figure 2

2.5.3.2. Steps to secure the wheelchair user:

1. Remove both armrests.
2. If present, attach the wheelchair's pelvic belt.
3. Attach the occupant restraint belts according to the instructions of the strap-system manufacturer.

i Wear the pelvic belt low across the front of the pelvis, so that the angle of the pelvic belt is within the preferred zone of 30° to 75° to the horizontal, similar to that shown below. A steeper (greater) angle within the preferred zone is desirable.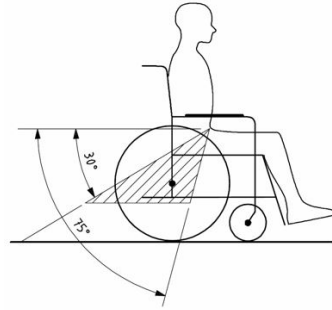

4. Adjust the belt tightly according to the instructions of the strap-system manufacturer, consistent with the user's comfort.
5. Ensure that the restraint belt connects in a straight line to the anchor point in the vehicle and that no bends in the belt are visible, for instance at the axle of the rear wheel.
6. Install the armrests, if desired. Make sure that belts are not twisted or held away from the body by wheelchair components such as armrests or wheels.
7. Position the seatbelt buckle so that the release button will not be contacted by wheelchair components during a crash.
8. Make sure that the shoulder-belt restraints fit over the shoulders, see figure 4.

Belt restraints must not be held away from the body by wheelchair components such as armrests or wheels.

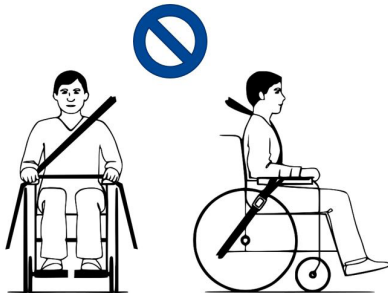
Figure 3

Belt restraints make full contact with shoulder, chest and pelvis. Pelvic belt low on the pelvis near the thigh-abdominal junction

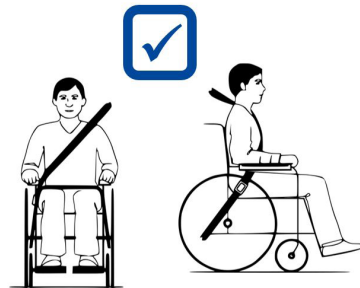
Figure 4

3. Using your wheelchair

⚠ WARNING

Risk of injuries

- First read previous chapters and inform yourself about the intended use. Do NOT use your wheelchair unless you have read and fully understood all instructions.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you with this.


3.1. Operating the brakes

3.1.1. Parking brakes (only for 22"/24" wheels)

⚠ CAUTION

Risk of injuries

- The brakes are not used to slow down the wheelchair during movements. Use the brake only to prevent the wheelchair from unintended movements.
- Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...). Check the condition of the tires before each use.
- The brakes are adjustable and can wear. Check the operation of the brakes before each use.
- Make sure the wheelchair is on a flat horizontal surface before releasing the brakes. Never release both brakes simultaneously.

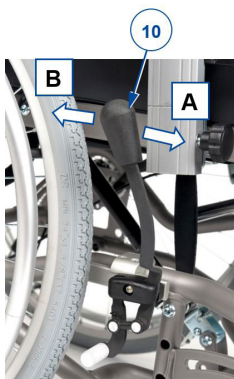3.1.1.1. To apply the brakes:

1. With one hand, hold one hand rim tight (or your attendant holds the push bar with one hand).
2. Push the brake lever **(10)** of the OTHER wheel to the FRONT until you feel a distinctive lock [A].
3. Repeat this for the second wheel and parking brake.

3.1.1.2. To release the brakes:

1. With one hand, hold one hand rim tight (or your attendant holds the push bar with one hand).
2. Pull the brake lever **(10)** of the OTHER wheel to the REAR [B].
3. Repeat this for the second wheel and parking brake.

3.1.2. Attendant brakes (only for 16" fixed wheels)

CAUTION

Risk of injuries

- Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...). Check the condition of the tires before each use.
- To park or stop on a slope, use the parking brakes after coming to a halt.
- The brakes are adjustable and can wear. Check the operation of the brakes before each use.
- Make sure the wheelchair is on a flat horizontal surface before releasing the brakes. Never release both brakes simultaneously.

3.1.2.1. To apply the brakes:

1. Pull the brake levers **(1)** in a squeezing motion.
2. Each brake lever **(1)** can be blocked by pushing the lock lever **(2)** down with your finger.
3. Release the brake lever **(1)**.

3.1.2.2. To release the brakes:

1. Pull the brake lever **(1)**. The lock lever **(2)** will be released by this motion.
2. Hold the handgrips firmly, while releasing the brake levers **(1)**.

3.2. Moving to/from the wheelchair

CAUTION

Risk of injuries or damage

- In case you cannot perform the transfer in a safe manner, ask someone to assist you.
- Do not stand on the footplates.

3.2.1. Transfer

1. Position the wheelchair as close as possible to the chair, couch or bed to/from which you wish to transfer.
2. Apply both parking brakes, see **§3.1.**
3. Move the backrest and seat to sitting position, see **§3.3.1.**
4. If needed, remove the footplate to prevent standing on it, see **§3.3.6.**
5. Transfer to/from the wheelchair using the strength of your arms or with the help of attendant(s) or lifting equipment.

3.2.2. Sitting in the wheelchair

1. Sit down on the seat with your lower back against the backrest.
2. Make sure that your upper legs are horizontal and that your feet are in a comfortable position. Adjust if necessary, see **§3.3.**

3.3. Comfort adjustments

CAUTION

Risk of injuries or damage

- Always keep the swing range of the footplate in mind to prevent bystanders from being injured or objects from being damaged.
- Make sure that your fingers, clothes, buckles don't get trapped during adjustment.

3.3.1. Backrest and seat inclination

CAUTION

Risk of injuries or damage

- Be aware that the stability decreases when the wheelchair is adjusted from sitting-straight to inclined position.
- Apply the parking brakes before adjusting the backrest and seat inclination.

The seat inclination and backrest inclination can be adjusted simultaneously or separately by an attendant. Make sure that the patient sits in the chair when the attendant adjusts the backrest inclination and that the wheelchair does not tip over.


3.3.1.1. Seat inclination

1. Squeeze the lever **(1)** on the left handle (see label) towards the handle to adjust the seat inclination.
2. Pull/Push the seat in the desired position (0° to $+21^{\circ}$), the gas pressure spring shall be pressed in/out.
3. Loosen the lever **(1)** to block the gas pressure spring.

3.3.1.2. Backrest inclination

1. Squeeze the lever **(1)** on the right handle (see label) towards the handle to adjust the angle of the backrest.
2. Pull/Push the backrest in the desired position (0° to $+54^{\circ}$), the gas pressure spring shall be pressed in/out.
3. Loosen the lever **(1)** to block the gas pressure spring.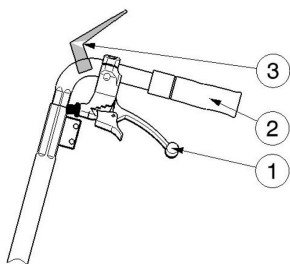

If your wheelchair features drum brakes, the levers **(1)** are used as brakes; additional levers **(3)** have been mounted to adjust the backrest and seat inclination.

3.3.2. Armrest height

The arm pads **(3)** of the wheelchair can be adjusted in 6 different height positions:

1. Turn the star knob **(45)** anticlockwise to unlock.
2. Pull and hold the star knob to release the inner tube of the armrest **(9)**.
3. Slide the inner tube of the armrest **(9)** up/down in the support **(46)** until the desired height. Make sure that the holes on the inner/outer tube align.
4. Retighten the star knob **(45)** by turning it clockwise.
5. Repeat this for the second armrest. Make sure that the armrests are firmly secured.

3.3.3. Push handle height

⚠ CAUTION

Risk of injuries

- Always use the available adjustment holes on the push handle. Never exceed the maximum height adjustment.

The push handles **(16)** and push bar **(23)** are connected and can be adjusted in 7 different height positions:

1. Loosen both star knobs **(31)** on the handle supports and remove them.
2. Adjust the height of both push handles simultaneously by sliding them up/down in the supports. Make sure that the holes on the inner/outer tube align.
3. Insert the two star knobs **(31)** in the supports and the adjustment holes.
4. Turn the star knobs clockwise until they are finger-tight.
5. Make sure that the push handles are firmly secured.

3.3.4. Leg rest inclination

The leg rest is adjustable in 6 positions.

1. Slightly loosen the lever **(4)** on both sides of the leg rest.
2. Lift/Lower the leg rest until the desired angle is reached.
3. Tighten the lever again firmly.

3.3.5. Footplate angle

The footplate is adjustable in 3 positions.

1. Slightly loosen the lever **(5)** on both sides of the footplate.
2. Turn the footplate up/down until the desired angle is reached.
3. Tighten the lever again firmly.

3.3.6. Removable footplate

It is possible to remove the footplate to facilitate transfer in/out of the wheelchair.

3.3.6.1. To remove the footplate:

1. Unzip the upholstery and cushions of the footplate from the upholstery and cushions of the leg rest.
2. On the bottom of either side of the footplate, pull the protruding screws down to unlock the footplate from the frame.
3. Gently pull the footplate forward off the leg rest tubes.

3.3.6.2. To attach the footplate:

1. Gently place the footplate on the leg rest tubes.
2. Push the protruding screws upward to lock the footplate to the frame.
3. Close the zippers of the upholstery and cushions.

3.3.7. Headrest

The headrest is fully adjustable in height and angle. To alter the position of the headrest: loosen any lever/star knob as specified below, adjust the headrest to the desired position, and retighten firmly.


- [6] To adjust the headrest sideways.
- [7] To rotate the headrest (360°).
- [8] To adjust the headrest forward/backward.
- [9] To adjust the height of the headrest.

To remove the headrest, loosen lever (9) and lift the headrest out of the support.

3.3.8. Tipping aid height

1. Press and hold the spring button (11).
2. Slide the tipping aid tube (64) in/out of the frame tube.
3. Release the spring button.
4. Repeat this for the second tipping aid. Make sure that both tipping aids are firmly secured.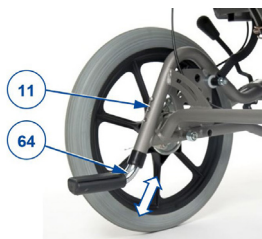

3.4. Riding the wheelchair

-  Depending on your medical condition and type of wheelchair, you can ride the wheelchair yourself or have it pushed by an attendant.

CAUTION

Risk of entrapment

- When driving with an attendant, keep your arms away from the wheels and keep your feet on the footplates.

CAUTION

Risk of injuries or damage

- Do NOT drive on slopes, obstacles, steps or kerbs larger than specified in § 6..
- Do not proceed with your wheelchair in traffic. Always stay on the pavement.
- Pay attention if the road has holes or gaps that may cause entrapment of the wheels.
- Avoid stones and other objects that may block the wheels.
- Always keep the swing range of the footplate in mind to prevent bystanders from being injured or objects from being damaged.
- Before every usage, make sure that:
 - all adjustments are firmly secured, see §3.3..
 - the parking brakes are working properly.
 - the tires are in good condition, see §5.2.1..

3.4.1. Riding with an attendant


- Release the brakes or have them released by the attendant, while he/she is holding the wheelchair to prevent movement, see §3.1..
- The attendant grasps the handgrips or push bar to push the wheelchair in the desired direction.
- After stopping, apply the brakes while holding the wheelchair in place, see §3.1..

3.4.2. Riding on slopes

CAUTION

Risk of injuries

- If the attendant has too little strength to control the wheelchair, stop riding and use the brakes immediately.
- When you stop on a (small) slope, use your brakes.

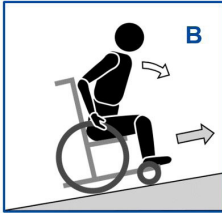1. Secure yourself in the wheelchair with a safety belt.
2. Ask an attendant or a bystander to help you.
3. Drive slowly and in a straight line. Lean slightly forward (A/B) when going up the slope. Lean backwards, against the backrest, when going down the slope.
4. Never reverse on a slope.

3.4.3. Using ramps

CAUTION

Risk of tipping over

- Only use ramps approved by Vermeiren and do not exceed their maximum load.
 - Make sure that the wheelchair does not touch the ground or ramp due to the inclination of the wheelchair.
 - Only negotiate ramps with the help of one or two attendants.
1. Adjust the position of backrest, seat and footrest to decrease the volume taken by the wheelchair and to increase the stability during inclination.
 2. Follow the instructions in **§3.4.2.**

3.4.4. Using stairs

CAUTION

Risk of tipping over

- Stairs should always be negotiated with the help of two attendants.
- Never try to use stairs that are not appropriate for wheelchairs.

3. One attendant tips the wheelchair slightly backwards by the handles.
4. The second attendant grabs the wheelchair at the front of the frame, on both sides of the wheelchair.
5. Stay calm, avoid sudden movements and keep your arms inside the wheelchair.
6. Both attendants lift and pull the wheelchair from step to step while the rear wheels roll over the steps.

3.4.5. Coping with obstacles


CAUTION

Risk of tipping over

- Always ask assistance of one or two attendants.
- Make sure that the footplates do not touch ground while taking an obstacle.
- Do not use your wheelchair on an escalator.
- If available, wear your safety belt.

3.4.5.1. Small kerbs (up or down)

These can be taken forward (E / F) with one attendant:

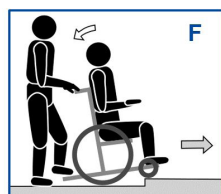1. The attendant moves the wheelchair forward approaching the kerb. Make sure the footplates won't touch the kerbs.
2. Lean backwards to reduce the pressure on the front wheels.
3. The attendant holds the handles firmly while moving forward. If necessary he uses the tipping aid to keep the front wheels lifted until they have passed the kerb rim.
4. The attendant releases the pressure on the handles and tipping aid to gently put the front wheels on the ground.
5. Next, he holds the handles firmly while pushing the wheelchair with the rear wheels up/down the kerb.

3.4.5.2. Medium kerbs (up or down)

These should be taken backwards with one attendant:

1. The attendant reverses the wheelchair so that rear wheels approach the kerb first (G / H).
2. Going down: Lean forward (**G**) to move your centre of gravity to the front.
Going up: Lean backward (**H**) to move your centre of gravity to the back.
3. The attendant pulls the wheelchair gently from/on the kerb.

3.4.5.3. Higher kerbs

Higher kerbs, but below maximum size, see §6., should be taken with help of two attendants.

Going down:

1. Remove the footrests.
2. The attendant moves the wheelchair forward approaching the kerb.
3. Lean backwards to reduce the pressure on the front wheels.
4. The attendant holds the handles firmly and uses the tipping aid to keep the front wheels lifted until they have passed the kerb rim.
5. The second attendant grabs the front of the frame and moves backwards while pulling the rear wheels down the kerb.
6. The first attendant releases the pressure on the handles and tipping aid to put the front wheels on the ground.

Going up:

1. Remove the footrests.
2. The first attendant reverses the wheelchair so that rear wheels approach the kerb first.
3. Lean backward to move your centre of gravity to the back.
4. The second attendant grabs the front of the frame and moves forward while pulling/pushing the rear wheels on the kerb. The first attendant lifts and holds the handles firmly to prevent the wheelchair from tipping over.

4. Installation and adjustment

CAUTION

Risk of injuries or damage

- The wheelchair needs to be (dis)assembled and adjusted by a specialist dealer according to the instructions in this chapter.
- Only use parts and tools described in this manual.
- While (dis)assembling and operating the wheelchair, make sure no objects or body parts get caught between moving parts.
- Make sure all screws are firmly secured before driving.

 Read the user manual of this wheelchair. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

4.1. Tools

To (dis)assemble and adjust the wheelchair, the following tools are needed:

- Allen key n° 4

4.2. Scope of delivery

The following items are part of the delivery.

- Framework with seat, backrest, wheels, armrests, leg rest, and L70 granule seat system
- Headrest
- User manual


Before use, check if everything is included and that no products are damaged.

Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

4.3. Assembly

The wheelchair is delivered fully assembled except for the headrest, the side support pillows of the backrest and seat, and the optional arm pads.

- The headrest can be mounted and adjusted according to the instructions in **§3.3.7**.
- The pillows are easy to attach by means of the zippers. The label on each pillow indicates the specific place and direction of the pillow (see examples below).

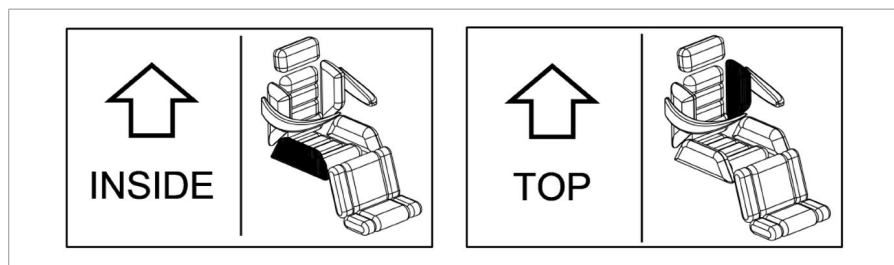4.4. Adjustments

4.4.1. Leg rest length

The length of the leg rest can be adjusted in 4 positions.

1. Take off the L70 granule cushions **(1)** from the leg rest frame.
2. Remove the lower foam protection **(2)** from the leg rest tubes.
3. Loosen and remove the screw-nut connection **(3)** on both sides of the leg rest.
4. Move the inner tubes with foot plate up/down in the outer frame tubes until the desired length is reached.
5. Replace the screws **(3)** and tighten firmly by hand.
6. Replace the foam protection **(2)**.
7. Reattach the granule cushions **(1)** to the leg rest frame.

5. Maintenance

- i** Regular care ensures that your wheelchair is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: www.vermeiren.com.

5.1. Points of maintenance

⚠ CAUTION

Risk of injuries and damage

- Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.
- i** The last page of this manual contains a registration form for the specialist dealer to record each service.
The service frequency depends on the frequency and intensity of use. Contact your dealer to agree to a common timetable for inspection/maintenance/repair.

5.1.1. Before each use

Inspect the following points:

- All parts: Present and undamaged or unworn.
- All parts: Clean, see **§ 5.2.2.**
- Wheels, seat, leg rests, arm rests, foot rests, footplates and headrest (if applicable): Well secured.
- All zippers: closed.
- Condition of wheels/tyres, see **§ 5.2.1.**
- Condition of brakes, see **5.2.1.**
- Condition of frame parts: No deformation, instability, weakness or loose connections.
- Seat, backrest, arm pads, calf pads and head rest (if applied): No excessive wear (like dented spots, damage or tears).

Contact your specialist dealer for possible repairs or part replacements.

5.1.2. Yearly or more often

Have your wheelchair inspected and serviced by your specialist dealer, at least once a year, or more often. The minimum maintenance frequency depends on use and should therefore be commonly agreed upon with your specialist dealer.

5.1.3. If stored

Make sure that your wheelchair is stored in a dry environment to prevent mould from growing and the upholstery from being damaged, see also chapter 6..

5.2. Maintenance instructions

5.2.1. Wheels and tyres

-  Proper working of the brakes depends on the state of the tyres, and can change due to wear and contamination (water, oil, mud, ...).

Keep your wheels free of wires, hair, sand and fibres.

Check the profile of the tyres. If the tread depth is less than 1 mm, the tyres need to be replaced. Contact your specialist dealer for this matter.

5.2.2. Cleaning

CAUTION

Risk of damage by moisture

- Never use a hose or high-pressure cleaner to clean the wheelchair.

Wipe all rigid parts with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

The granule seat cushions can be cleaned with a damp cloth (not drenched), and mild household soap if necessary. The teddy covers can be machine washed at 60° C. Do not use abrasive cleaning agents.

5.2.3. Disinfection

CAUTION

Risk of damage

- Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

5.3. Troubleshooting

Even if you use your wheelchair properly, a technical problem may occur. In this case, contact your local specialist dealer.

WARNING

Risk of injuries and damage

- NEVER attempt to repair your wheelchair yourself.

The following symptoms may indicate a serious problem. Therefore always contact your specialist dealer if you spot any of the following deviations:

- Strange sound;
- Uneven tread wear on one of the tyres;
- Jerky movements;
- Wheelchair deflects to one side;
- Damaged or broken wheel assemblies.

5.4. Expected lifespan

The wheelchair is designed to have an average lifespan of 5 years. Depending on the frequency of use, driving circumstances and maintenance, the lifespan of your wheelchair will increase or decrease.

5.5. Reuse

Before each reuse, have the wheelchair disinfected, inspected and serviced according to the instructions in § 5.1. and § 5.2..

5.6. End of use

At end of life, you need to dispose your wheelchair according to the local environmental legislation. The best way to do so, is to disassemble the wheelchair to facilitate the transport of recyclable parts.

5.7. Warranty

The warranty on this product is subject to the general terms and conditions of each country.

6. Technical specifications

The technical details below are only valid for this wheelchair, at standard settings and optimal ambient conditions. Take these details into account during use. The values are no longer applicable if your chair has been modified, damaged, or is severely worn.

Brand	Vermeiren
Product group	Manual wheelchair
Type	Inovys II with L70 granule seat system
Description	Dimensions
Maximum user weight	135 kg
Overall length	1280 mm
Overall width	680 mm 730 mm
Overall height (without headrest)	990 - 1060 mm
Total mass	38 kg
Mass of heaviest part	33 kg
Mass of removable parts	
Armrests	3,2 kg
Headrest	1,85 kg
Rated slope	10°
Seat plane angle	0° - 30°
We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°	

Brand	Vermeiren
Product group	Manual wheelchair
Type	Inovys II with L70 granule seat system
Description	Dimensions
Effective seat depth	460 - 520 mm
Effective seat width	450 mm 500 mm
Seat surface height at front edge	510 - 580 mm
Backrest angle	0° - 45°
Backrest height	550 - 600 mm
Footplate to seat distance	360 - 385 - 410 - 435 mm
Leg to seat surface angle	17° - 85°
Footplate angle	73° - 107°
Distance between arm pad and seat	220 - 320 mm
Front location of arm pad structure	460 - 490 mm
Horizontal location of axle (deflection)	-8 mm - 41 mm
Minimum turning radius	1500 mm
Diameter of rear wheels	16" (T30) with drum brakes
Diameter of front wheels	200 mm
Use temperature	+5°C - +41°C
Storage and use humidity	30% - 70%
We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°	

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was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:

VERMEIREN GROUP

Vermeirenplein 1/15

2920 Kalmthout

Belgium

www.vermeiren.com

Version: E, 2025-02

Basic UDI-DI: 5415174122103InovysIIN9

All rights reserved, including translation.

VERMEIREN
we care **for you**