


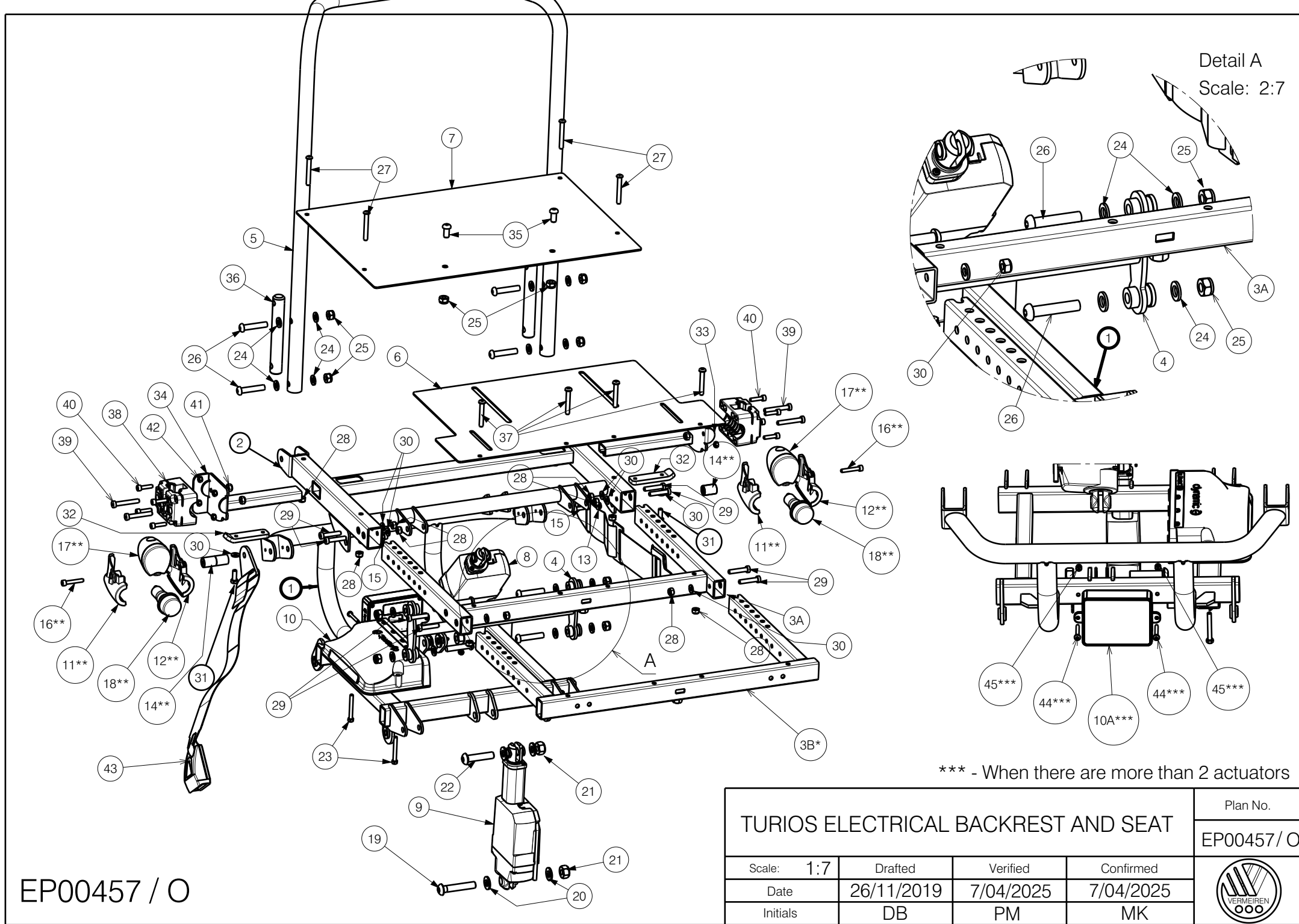

EP00457 / O

TURIOS ELECTRICAL BACKREST AND SEAT				Plan No.
				EP00457 / O
Scale: 1:7	Drafted	Verified	Confirmed	
Date	26/11/2019	7/04/2025	7/04/2025	
Initials	DB	PM	MK	

Material			
No.	Plan	Article	Quantity
1	YP07411 (S)	2776954	1
	YP07412 (M)	2777054	1
	YP07413 (L)	2777154	1
2	YP07414 (S)	2777254	1
	YP07415 (M)	2777354	1
	YP07416 (L)	2777454	1
3A	YP07417 (S)	2777554	1
	YP07418 (M)	2777654	1
	YP07419 (L)	2777754	1
3B*	YP08368 (S)	2614954	1
	YP08369 (M)	2615054	1
	YP08370 (L)	2615154	1
4	YP07421 (M)	2777954	1
5	YP07682 (S)	2600854	1
	YP07683 (M)	2600954	1
	YP07684 (L)	2601054	1
6	YP07706 (S)	2779754	1
	YP07707 (M)	2779854	1
	YP07708 (L)	2779954	1
7	YP07709 (S)	2780054	1
	YP07710 (M)	2780154	1
	YP07711 (L)	2780254	1
8	LA23	1906790	1
9	LINAKMOTOR	1908165	1
10	LINX 60A	1912294	1
#	LINX 70A	1912302	1
	LINX120A	1912415	1
10A***	2 ACT MODULE	1912304	1
11**	X004942	1907377	2
12**	X004943	1907376	2
13**	X004960	1907430	2
14**	---	1910745	2

* - use with BZ7/8 and BZE |

** - Lights SE19 - option

*** - When there are more than 2 actuators - option

- option

Material			
No.	Plan	Article	Quantity
15**	1007198	1007198	2
16**	M5x30 DN912	1903983	2
17**	Led Head Light	1907340	2
18**	Indicator Lamp	1991679	2
19	M10x55 ISO 7380	1998518	1
20	M10 DIN125	1001558	9
21	M10 DIN985	1001683	4
22	M10x45	1908848	3
23	M5x40 DIN933	1001427	2
24	M8 DIN125 BLACK	2994415	16
25	M8 DIN985	1001682	10
26	M8x40	1008739	8
27	M6x45	1991017	4
28	M6 DIN985	2994063	15
29	M6X35BLACK	1915416	8
30	M6 DIN125	1001564	10
31	M6x16 DIN933	1001429	2
32	Y012123	1675800	2
33	YP08993	2631054	1
34	YP08994	2631154	1
35	M8x16	1991858	2
36	XP02856	GP21400	2
37	6x40	1006111	3
38	X005565	1912287	2
39	M6x40	1006850	4
40	M5x20 DIN912	1915452	8
41	M6 DIN985	2994063	4
42	M5 DIN985	2994006	8
43	BELT	2993352	1
44***	M4x30	1001487	2
45***	M4 DIN985	1001678	2

TURIOS ELECTRICAL BACKREST AND SEAT

Plan No.

EP00457 / O

Scale: 1:7

Drafted

Verified

Confirmed

Date

26/11/2019

7/04/2025

7/04/2025

Initials

DB

PM

MK



EP00457 / O