
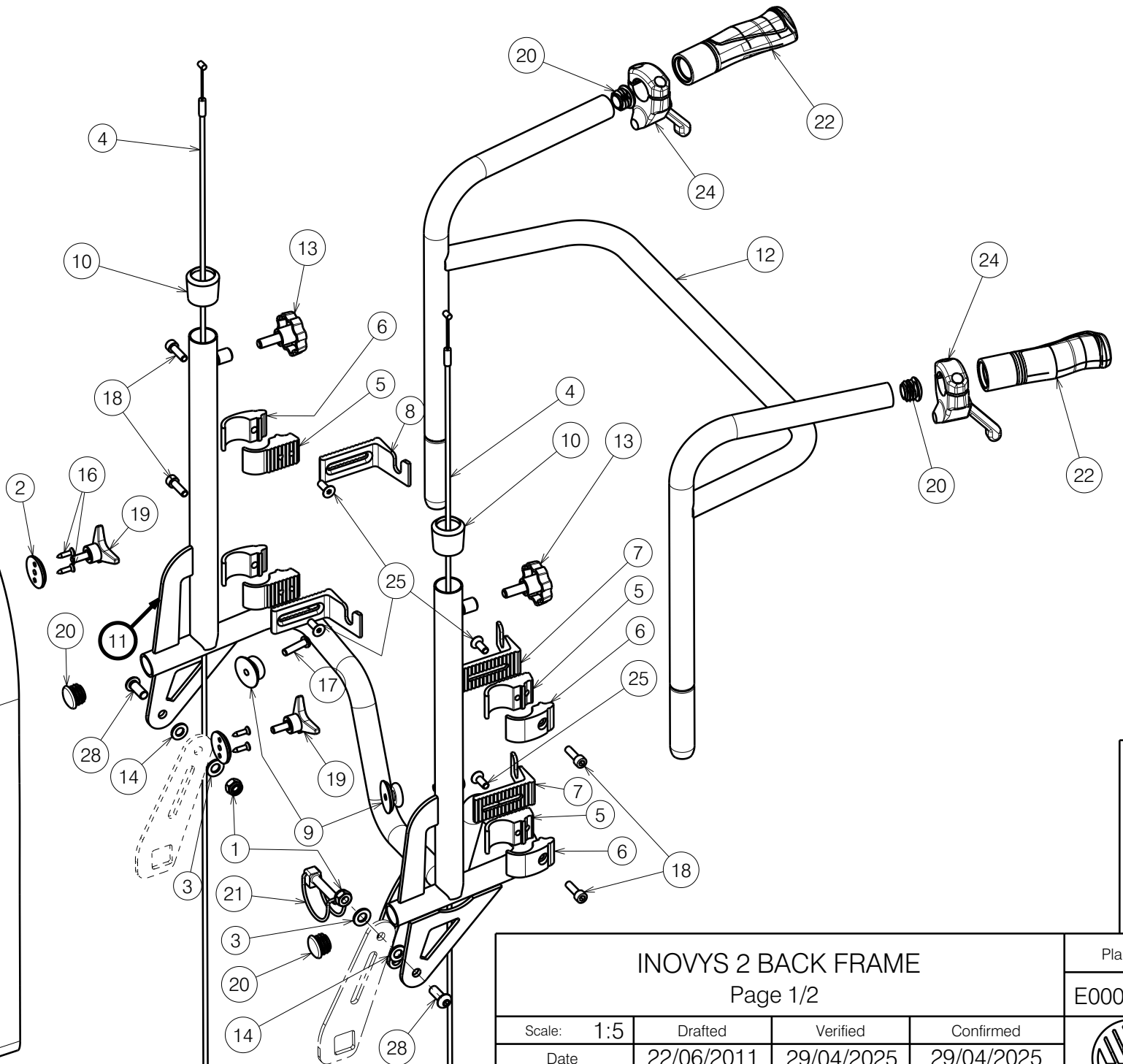


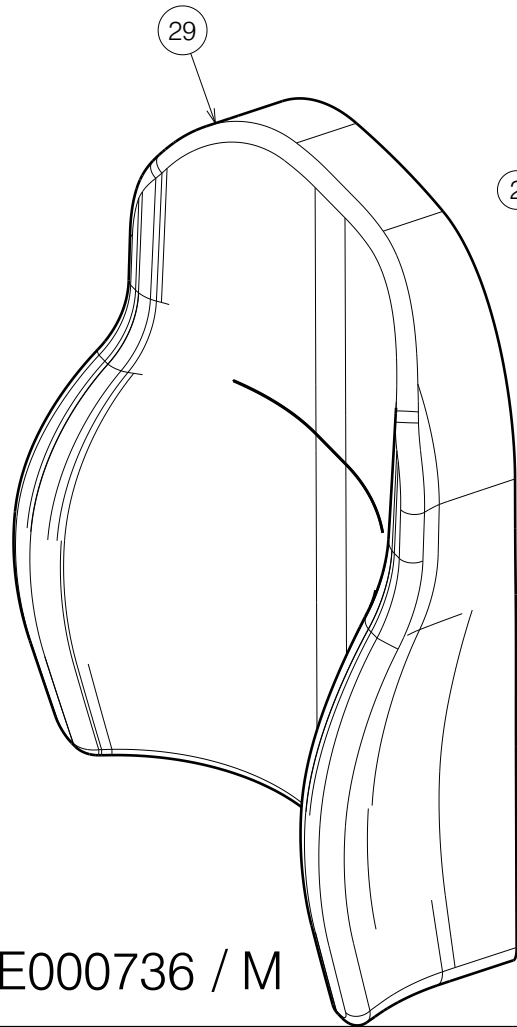

2991382

INOVYS 2 BACK FRAME

Page 1/2

Plan No.

E000736 / M

Scale: 1:5

Drafted

Verified

Confirmed

Date

22/06/2011

29/04/2025

29/04/2025

Initials

SD

DR

AS



E000736 / M

Material			
No.	Plan	Article	Quantity
1	X001016	1001682	2
2	X001052	G066600	2
3	X001119	1001554	2
4	X003646	1999963	2
5	X004833	1745032	4
*	X004833	1745099	4
6	X004832	1744932	4
*	X004832	1744999	4
7	X004822	1744732	2
*	X004822	1744799	2
8	X004823	1744832	2
*	X004823	1744899	2
9	X004861	1906454	2
10	Y004353	F000411	2
11	Y017806	17562.. (40)	1
	Y017651	17560.. (45)	1
	Y017807	17563.. (50)	1
	YP06467	27347.. (35)	1
12	Y018198	1765462 (40)	1
	Y018199	1765562 (45)	1
	Y018200	1765662 (50)	1
	YP06468	2734862 (35)	1
13	M8x20	1001580	2
14	M8 DIN125	1003199	2
16	3,9x16 DIN7982	1005767	4
17	M6x25 ISO7380	1006998	2
18	M6x25 DIN912	1001494	4
*	M6x25 DIN912 Black	2996887	4
19	M6x20	1009932	2
20	Ø22	1990648	4
21	D8x40	1992510	1
22	X005513	1992600	2
24	---	1996806	2

Material			
No.	Plan	Article	Quantity
25	M6x16 DIN7991	1994752	4
*	M6x16 DIN7991 Black	2994064	4
28	M8x20 ISO7380	1992349	2
29	L34	1906938 (40)	1
	L34	1906940 (45)	1
	L34	1906942 (50)	1
	L34	2996478 (35)	1

* - Use for Inovys Junior (35)

INOVYS 2 BACK FRAME				Plan No.
Page 2/2				E000736/ M
Scale: 1:5	Drafted	Verified	Confirmed	
Date	22/06/2011	29/04/2025	29/04/2025	
Initials	SD	DR	AS	

E000736 / M

(40 - 45 - 50 - 35 = Seat width)

2991382