


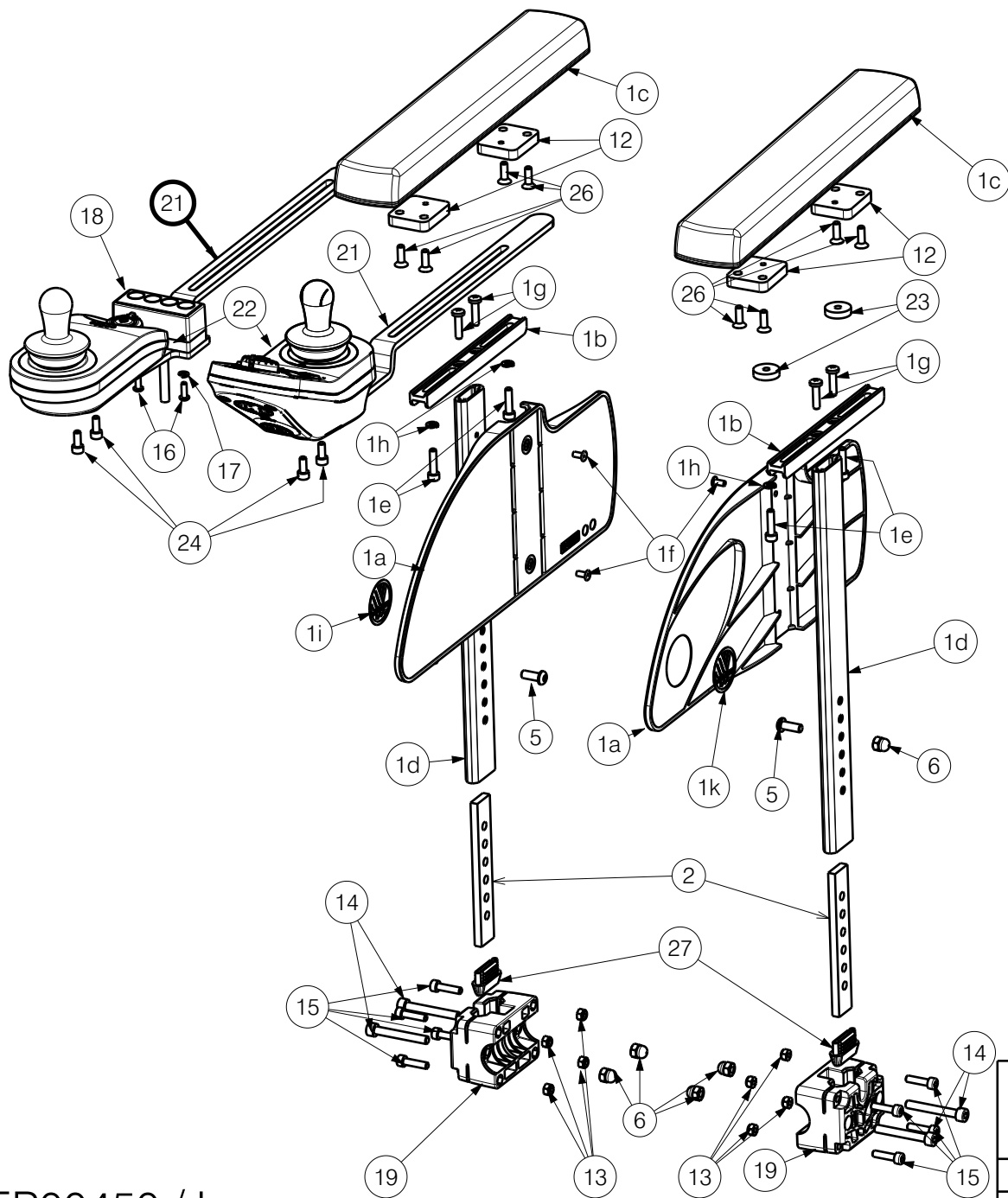

*Used with light controller

Material			
No.	Plan	Article	Quantity
1	Armrest right	1913465	1
	Armrest left	1913464	1
2	YP09265	2642654	2
3	-	-	-
4	-	-	-
5	M6x16 ISO7380	2994067	2
6	M6 DIN986	2994059	6
7	-	-	-
8	-	-	-
9	-	-	-
10	-	-	-
11	-	-	-
12	YP07839	2783154	4
13	M5 DIN985	2994006	8
14	M6x45 DIN912	2994058	4
15	M5x20 DIN912	1915452	8
16*	M4x12	2994126	2
17*	Washer M4 DIN127B	1998515	2
18*	Light controller	1921014	1
19	X005565	1912287	2
20	-	-	-
21	YP07841	2783254	1
22	YP07833	2782754	1
	Linx DLX REM 050	1909020	1
	Linx DLX REM 060	1910700	1
	Linx SE19 SE39	1912296	1
	Linx SE39	1912301	1
	Linx SE19 ACT5	1912305	1
	Linx LCD + JOYST	1913523	1
23	YP07842	2783354	2
24	M5x12 DIN912/8.8	1001489	2
25	-	-	-
26	M5x16	1907493	8
27	XP03080	2996656	2

EP00450 / L

BI5 TYPE SQUOD/ TURIOS

1/2

Plan No.

EP00450/ L

Scale: 1:5

Drafted

Verified

Confirmed

Date

13/09/2019

27/03/2025

27/03/2025

Initials

DB

DR

MK



EP00450 / L

*Used with light controller

BI5 TYPE SQUOD/ TURIOS2/2				Plan No.
				EP00450/ L
Scale: 1:5	Drafted	Verified	Confirmed	
Date	13/09/2019	27/03/2025	27/03/2025	
Initials	DB	DR	MK	

Bill of Material 1913465/1913464			
No.	Plan	Article	Quantity
1a	X005567	1912286 (R)	1
	X005566	1912285 (L)	1
1b	X005517	2014999	1
1c	X005519	1911778	1
1d	X005561	2021699	1
1e	M5x16	1007198	2
1f	Rivet Ø4	1992805	2
1g	M5x16	3991136	2
1h	Washer M5	1906561	2
1i	Sticker	T001002	1