

Use WEICONLOCK AN 302-43.

Use WEICONLOCK AN 302-43.

Degrease the screws before using the adhesive./ Odtlusc sruby przed uzyciem kleju

Detail B
Scale: 3:5

Drawing shows only right side

BACKREST ADJUSTABLE ANGLE AND ADJUSTABLE HIGHT WITH FOLDING HANDLE TRIGO				Plan No. EP00374/ F
Scale: 3:10	Drafted	Verified	Confirmed	
Date	13/02/2019	14/10/2021	14/10/2021	
Initials	AS	MK	TK	

EP00374 / F

Material (35-40)			
No.	Plan	Article	Quantity
1	YP07083	2745665	1
** 2	YP08603	2622665	1


Material (40-45)			
No.	Plan	Article	Quantity
1	YP07083	2745665	1
** 2	YP08604	2622765	1

Material (45-50)			
No.	Plan	Article	Quantity
1	YP07084	2643065	1
** 2	YP08605	2744665	1

Material			
No.	Plan	Article	Quantity
3	XP03203	GP25700	1
4	XP01862	GP13200	1
5	M4x12	2994126	1
6	XP01632	GP09399	2
7	XP01861	GP12900	1
8	M5x12 DIN 7991	2994076	1
9	M6x30	1992721	2
10	M6	2994059	1
11	WASHERM6	2994068	1
12	Fold. push handle	EP00246	1
13	Y004353	F000411	1
14	M6x10 DIN916	2994284	2
15	M6X10	2994127	2
*16	Cap D22	1990648	1

* When unused folding handle / Gdy nie używana rączka składana.

** When the folding handle is not used, the backrest is 4cm lower /
Gdy nie używana rączka składana oparcie jest o 4 cm niższe

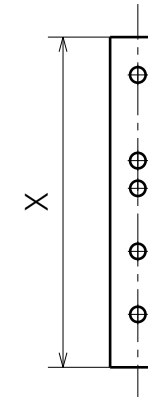

Plan	X
YP08603(35-40)	144mm
YP08604(40-45)	193,5mm
YP08605 (45-50)	243,5mm

EP00374 / F

BACKREST ADJUSTABLE ANGLE AND ADJUSTABLE HIGHT WITH FOLDING HANDLE TRIGOPage 2/2				Plan No.
				EP00374/ F
Scale: 3:10	Drafted	Verified	Confirmed	
Date	13/02/2019	14/10/2021	14/10/2021	
Initials	AS	MK	TK	