


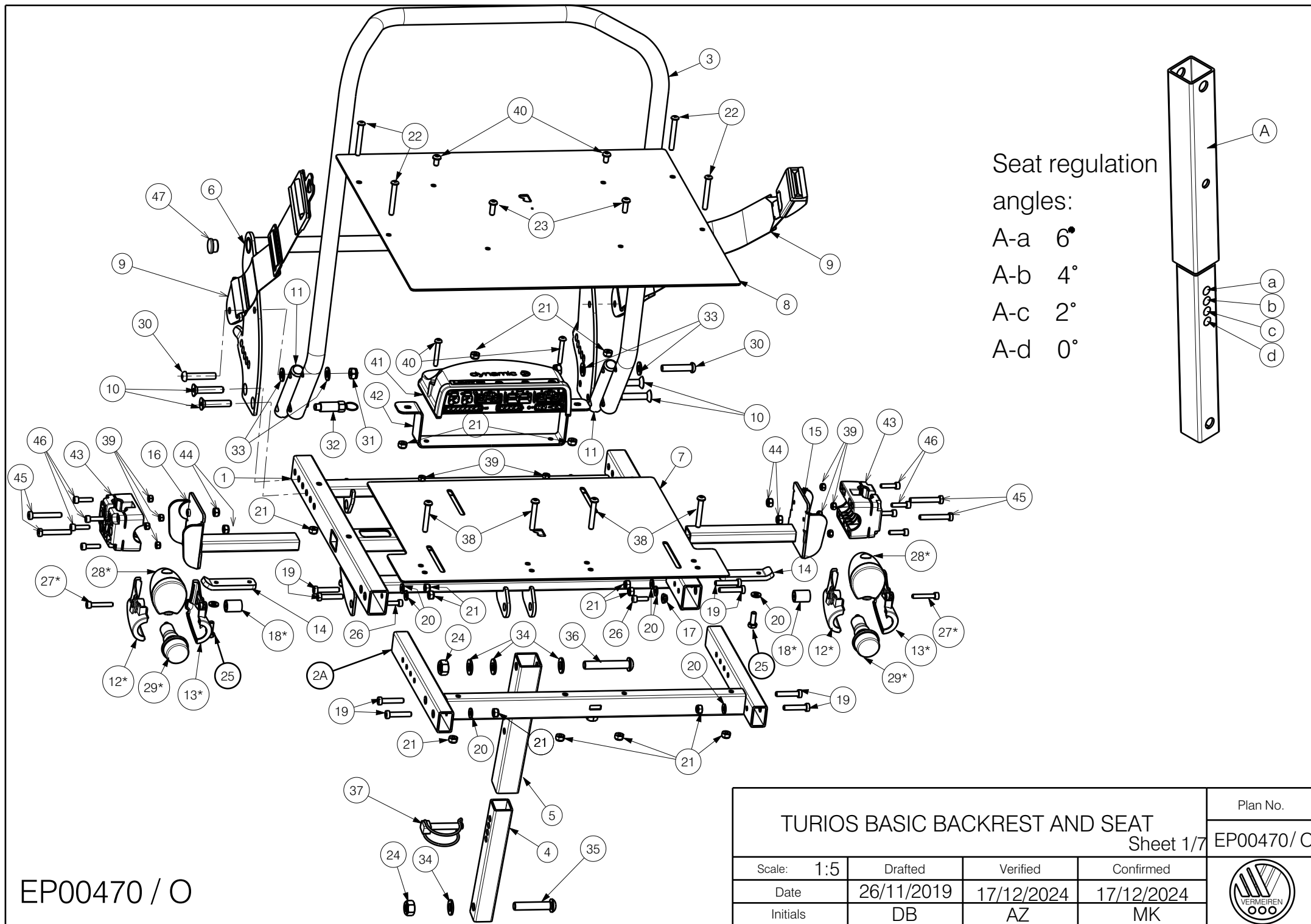

Material			
No.	Plan	Article	Quantity
1	YP07428 (S)	2778654	1
	YP07429 (M)	2778754	1
	YP07430 (L)	2778854	1
2A	YP07431 (S)	2778954	1
	YP07432 (M)	2779054	1
	YP07433 (L)	2779154	1
2B**	P00087337(S)	2692554	1
	P00087866(M)	2692854	1
	P00088204(L)	2692954	1
3	YP07434 (S)	2779254	1
	YP07435 (M)	2779354	1
	YP07436 (L)	2779454	1
4	YP07736	2781754	1
5	YP07737	2781854	1
6	YP07750 (S)	2780854	1
	YP07751 (M)	2780954	1
	YP07752 (L)	2781054	1
7	YP07753 (S)	2781154	1
	YP07754 (M)	2781254	1
	YP07755 (L)	2781354	1
8	YP07756 (S)	2781454	1
	YP07757 (M)	2781554	1
	YP07758 (L)	2781654	1
9	SAFETY BELT	2993352	1
10	M8x40	2994122	4
11	XP02887	GP21700	2
12*	X004942	1907377	2
13*	X004943	1907376	2
14	Y012123	1675800	2
15	YP08993	2631054	1
16	YP08994	2631154	1

* - Lights SE19 - option

** - Not depicted |

2A - Use with B06 |

2B - Use with BZ7/8 and BZE |

Material			
No.	Plan	Article	Quantity
17*	X004960	1907430	2
18*	---	1910745	2
19	M6x30 Black	1915416	8
20	M6 DIN125A	1001564	10
21	M6 DIN985	2994063	20
22	M6x45	1991017	4
23	M6x16	1008459	2
24	M10 DIN985	1001683	2
25	M6x16 DIN933	1001429	2
26*	M5x16 DIN912	1007198	2
27*	M5x30 DIN912	1903983	2
28*	LedHeadLight	1907340	2
29*	IndicatorLamp	1991679	2
30	M8x40	1008739	2
31	M8 DIN985	1001682	6
32	GN 717-8-AK	2990455	2
33	M8 DIN125 BLACK	2994415	4
34	M10 DIN125A	1001558	4
35	M10x50	1009335	1
36	M10x60	1907128	1
37	D8x45	1993594	1
38	6x40	1006111	4
39	M5 DIN 985	2994006	10
40	M5x35	2990115	2
41	LE 50A (6km/h)	1910896	1
	LINX 60A (10km/h)	1912294	1
42	YP07971	2603654	1
43	X005565	1912287	2
44	M6 DIN985	2994063	4
45	M6x40	1006850	4
46	M5x20 DIN912	1915452	8
47	Plastic Plug	1001602	2

TURIOS BASIC BACKREST AND SEAT


Sheet 2/7 EP00470/ O

Scale: 1:5	Drafted	Verified	Confirmed
Date	26/11/2019	17/12/2024	17/12/2024
Initials	DB	AZ	MK

Plan No.




EP00470 / O

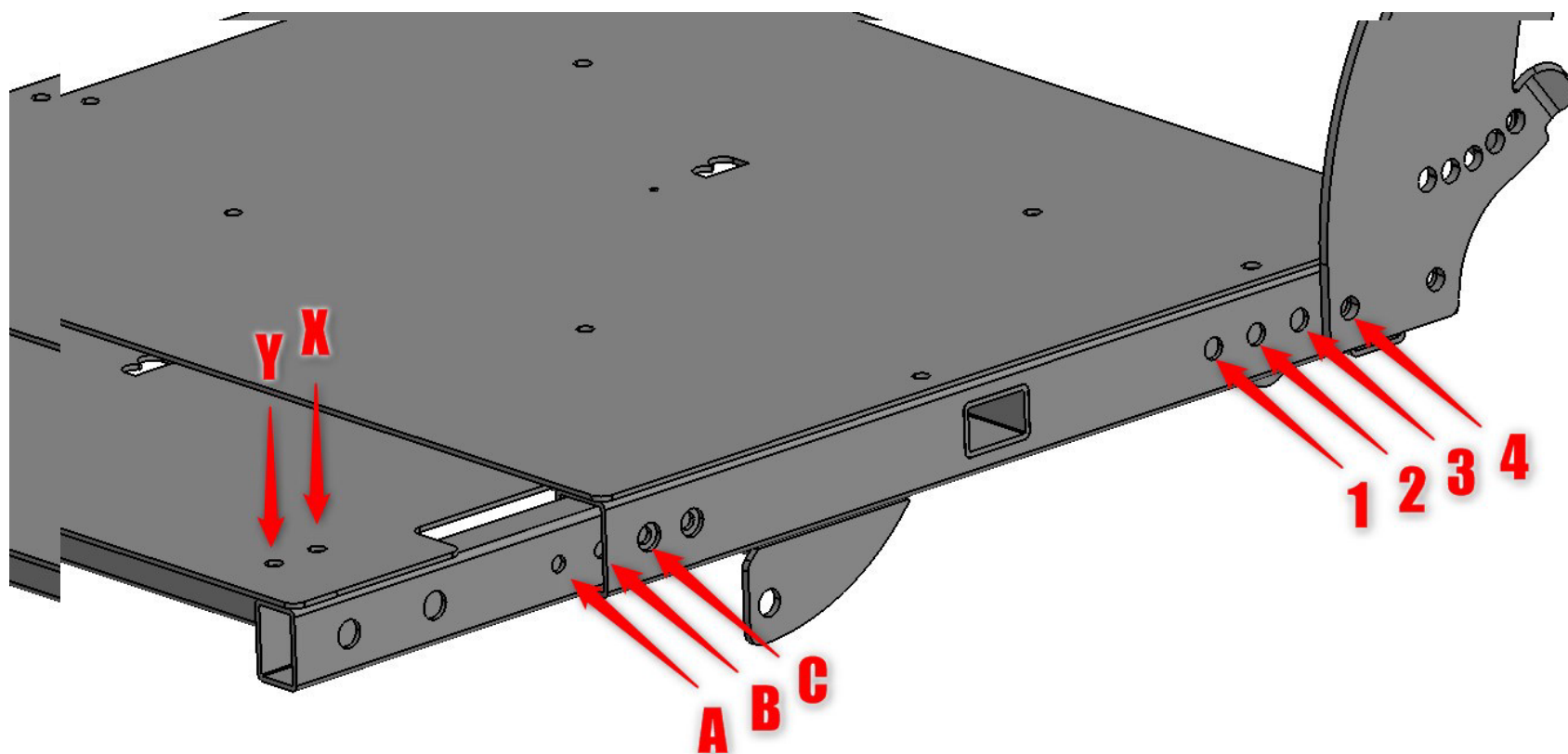


EP00470 / O

TURIOS BASIC BACKREST AND SEAT				Plan No.
Sheet 3/7				EP00470 / O
Scale: 1:5	Drafted	Verified	Confirmed	
Date	26/11/2019	17/12/2024	17/12/2024	
Initials	DB	AZ	MK	

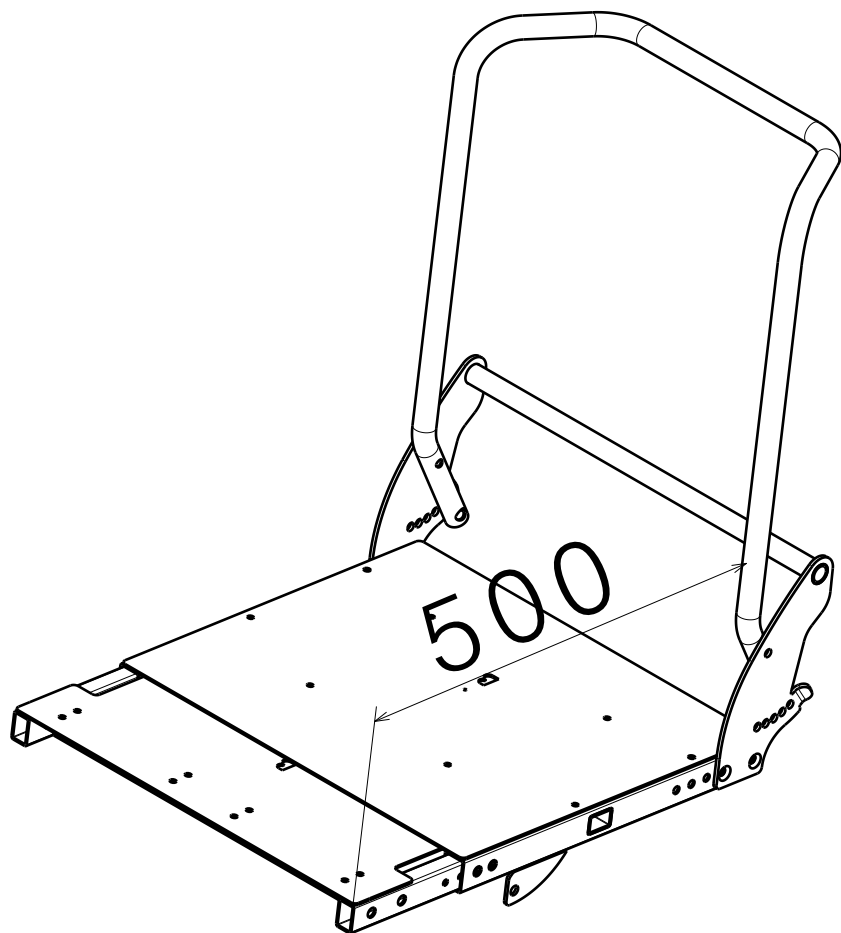

Size 500 / Rozmiar 500
XC4

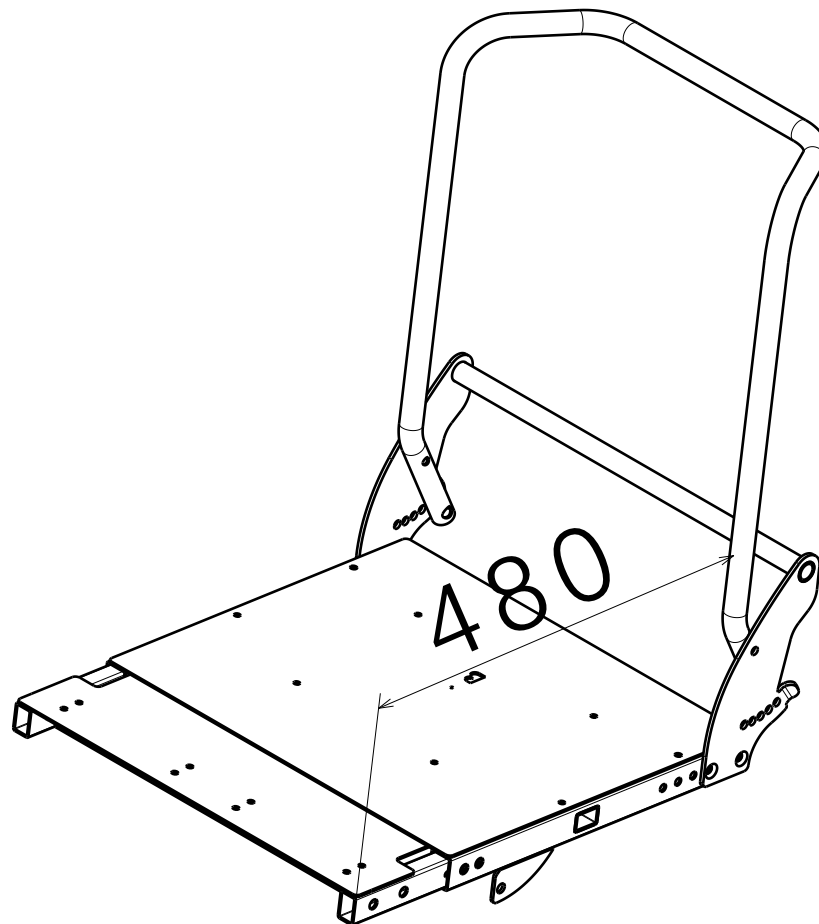


Size 480 / Rozmiar 480
XB4

EP00470 / O

TURIOS BASIC BACKREST AND SEAT				Plan No.
Sheet 4/7				EP00470/ O
Scale: 1:5	Drafted	Verified	Confirmed	
Date	26/11/2019	17/12/2024	17/12/2024	
Initials	DB	AZ	MK	




Size 460 / Rozmiar 460
XA4

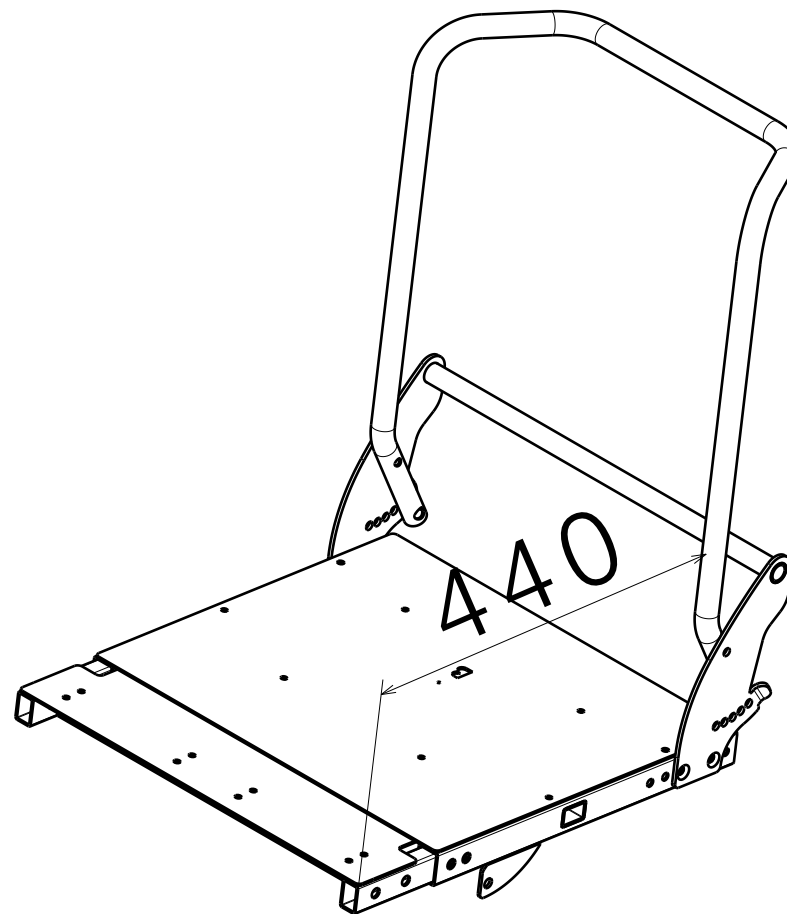


Size 440 / Rozmiar 440
XA3

EP00470 / O

TURIOS BASIC BACKREST AND SEAT				Plan No.
Sheet 5/7				EP00470/ O
Scale: 1:5	Drafted	Verified	Confirmed	
Date	26/11/2019	17/12/2024	17/12/2024	
Initials	DB	AZ	MK	

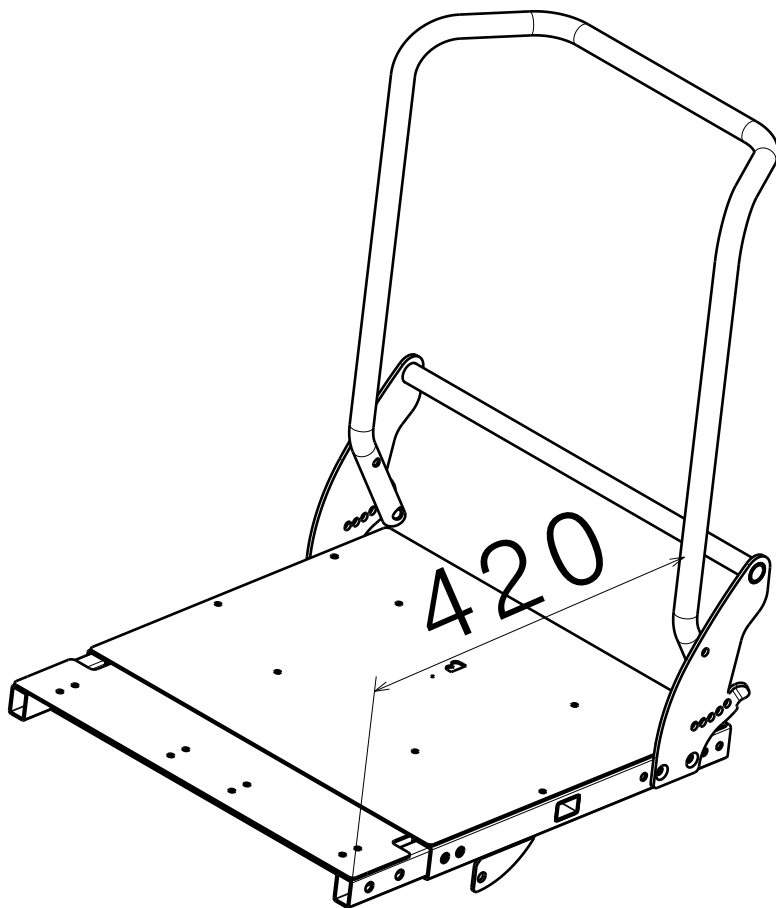

Size 420 / Rozmiar 420
XA2

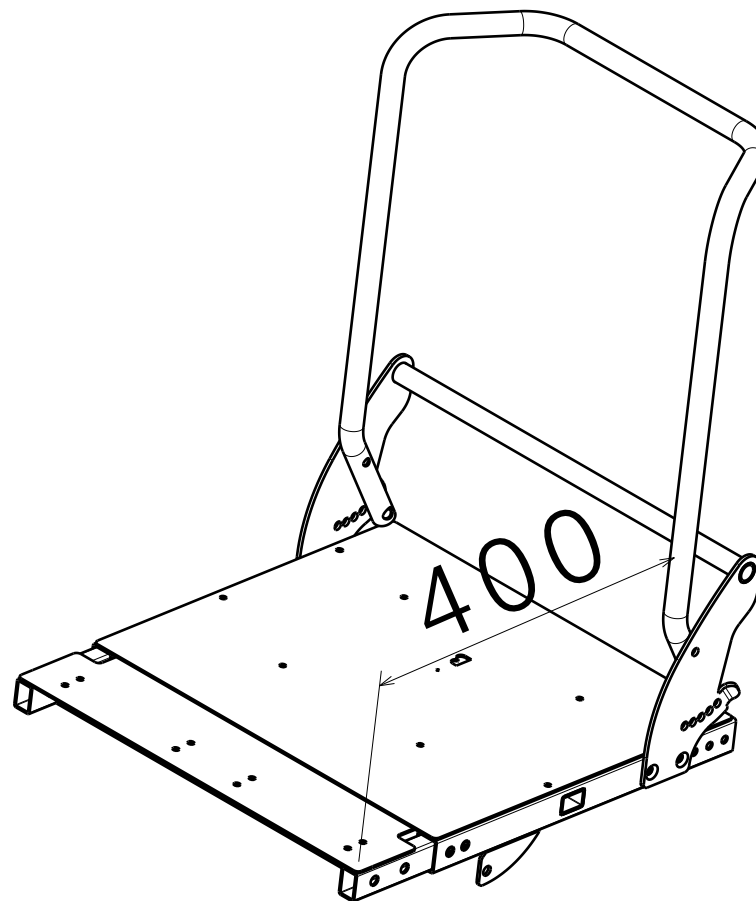


Size 400 / Rozmiar 400
XA1

EP00470 / O

TURIOS BASIC BACKREST AND SEAT				Plan No.
				EP00470/O
Scale:	1:5	Drafted	Verified	Confirmed
Date	26/11/2019	17/12/2024	17/12/2024	
Initials	DB	AZ	MK	



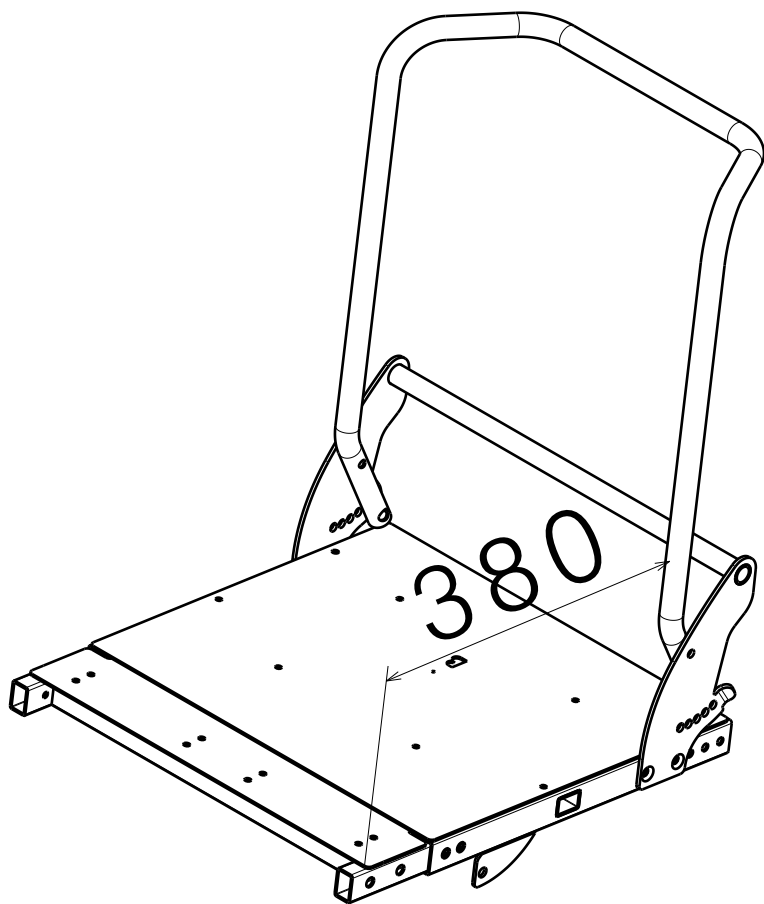

Size 380 / Rozmiar 380
YA1

No.	Size	Configutation
1	380	YA1
2	400	XA1
3	420	XA2
4	440	XA3
5	460	XA4
6	480	XB4
7	500	XC4

TURIOS BASIC BACKREST AND SEAT

Sheet 7/7

Plan No.

EP00470/O

Scale: 1:5

Drafted

Verified

Confirmed

Date

26/11/2019

17/12/2024

17/12/2024

Initials

DB

AZ

MK



EP00470 / O