

Use WEICONLOCK AN 302-43.

Degrease the screws before using the adhesive.


Detail B
Scale: 3:5

Drawing shows only right side

EP00374 / E

BACKREST ADJUSTABLE ANGLE AND ADJUSTABLE HIGHT WITH FOLDING HANDLE TRIGO				Plan No.
Page 1/2				EP00374 / E
Scale: 3:10	Drafted	Verified	Confirmed	
Date	13/02/2019	31/03/2021	31/03/2021	
Initials	AS	MK	TK	

Material			
No.	Plan	Article	Quantity
1	YP07083(35-45)	2745665	1
	YP07084(40-50)	2745765	1
**2	YP08603(35-40)	2622665	1
	YP08604 (40-45)	2622765	1
	YP08605(45-50)	2622865	1
3	XP03203	GP25700	1
4	XP01862	GP13200	1
5	M4x12	2994126	1
6	XP01632	GP09399	2
7	XP01861	GP12900	1
8	M5x12 DIN 7991	2994076	1
9	M6x30	1992721	2
10	M6	2994059	1
11	WASHERM6	2994068	1
12	Fold. push handle	EP00246	1
13	Y004353	F000411	1
14	M6x10 DIN916	2994284	2
15	M6X10	2994127	2
* 16	Cap D22	1990648	1

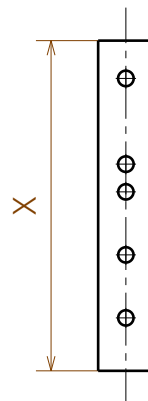

Plan	X
YP08603(35-40)	144mm
YP08604(40-45)	193,5mm
YP08605 (45-50)	243,5mm

* When unused folding handle .

** When the folding handle is not used, the backrest is 4cm lower /

EP00374 / E

BACKREST ADJUSTABLE ANGLE AND ADJUSTABLE HIGHT WITH FOLDING HANDLE TRIGOPage 2/2				Plan No. EP00374/ E
Scale: 3:10	Drafted	Verified	Confirmed	
Date	13/02/2019	31/03/2021	31/03/2021	
Initials	AS	MK	TK	