


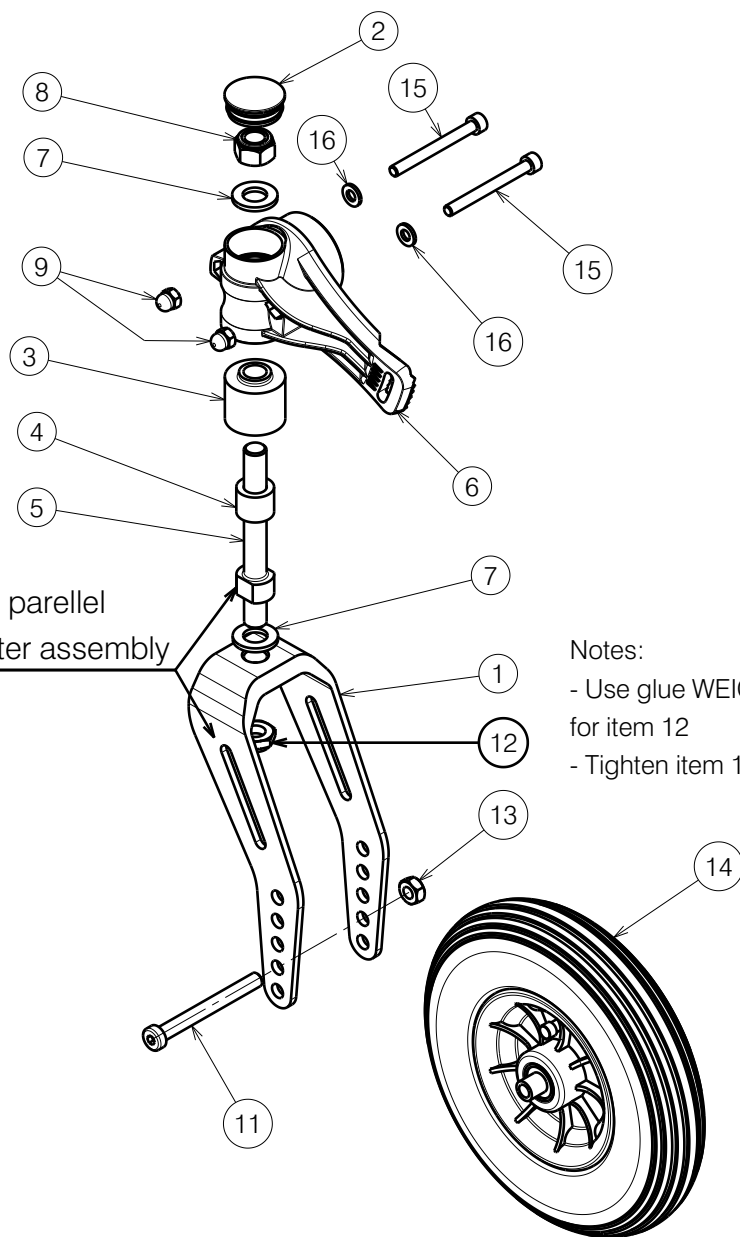

- Notes:
- Use glue WEICON AN 302-43 (H900009) for item 12
 - Tighten item 12 with 35 Nm Torque

BOM			
No.	Plan	Article	Qty
1	P00056854	2086465 (std.)	1
		2086481 (LB)	1
		20864C5 (LG)	1
		20864A3 (OR)	1
		2086471 (WH)	1
2	X004436	1903928	1
3	X004444	1722354	1
4	X004445	1904571	1
5	X004466	1730700	1
6	X005865	1915642 (L)	1
		1915643 (R)	1
7	M12 DIN125A	1001561	2
8	M12 DIN986	1001684	1
9	M6 DIN986	2994059	2
11	M8x75 DIN7984	2994912	1
12	M12x1.25	1994605	1
13	M8	1915942	1
14	200x50 PU	1916559	1
		1915907	1
15	M6x60 DIN912	2995262	2
16	M6 DIN125	2994414	2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

2995845

VERSO FRONT FORK

Plan No.

E00054057/H1.1

Scale:	1:4	Drafted	Verified	Confirmed
Date	14/03/2023	30/06/2025	30/06/2025	
Initials	SD	SD	TS	

