
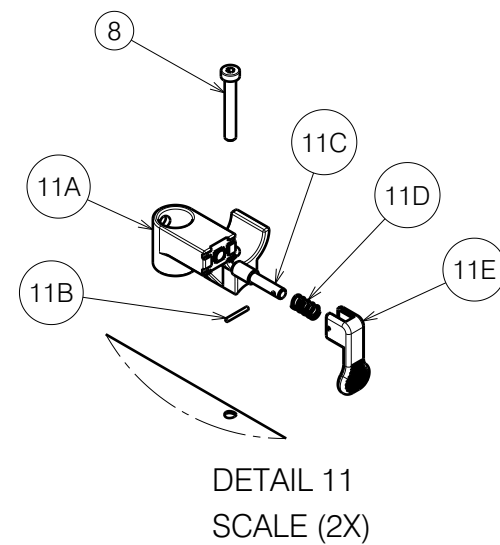


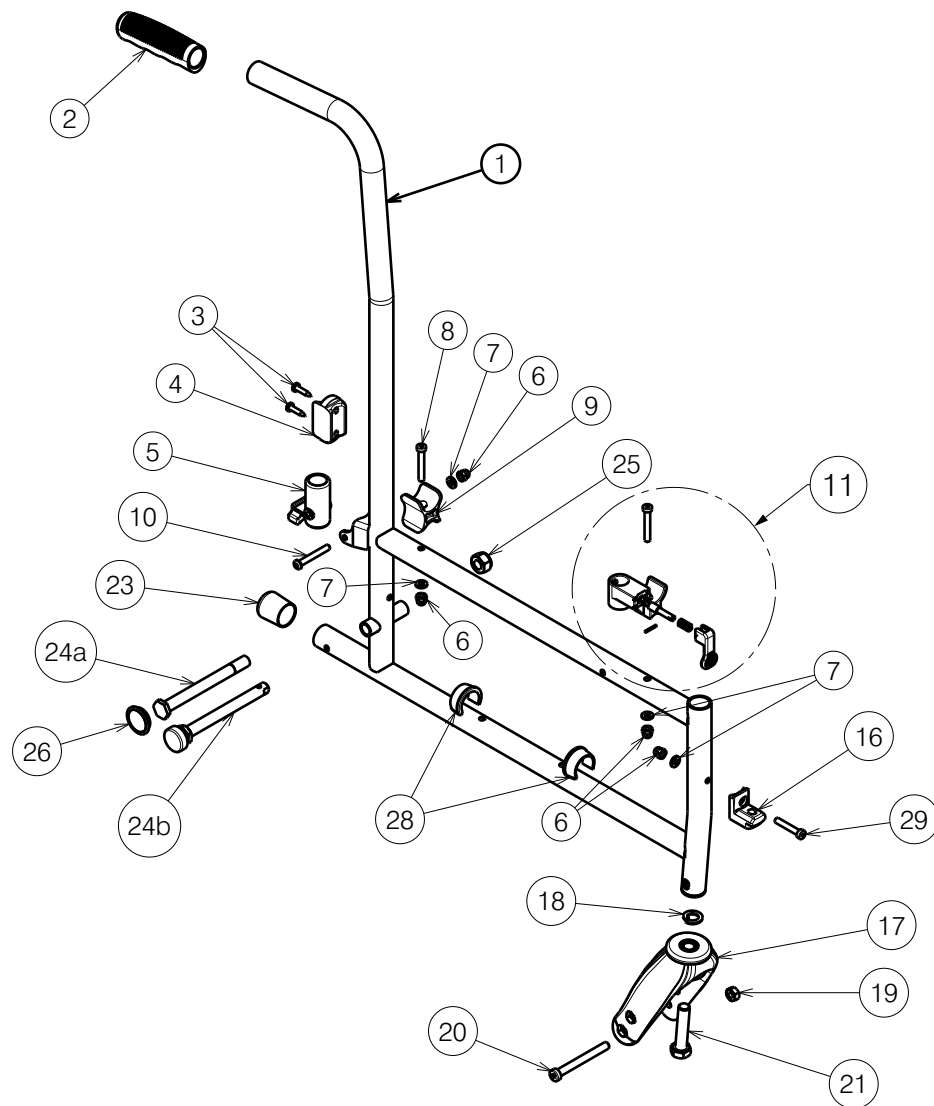

* Item 21 add torque 60Nm.

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

SIDE FRAME TYPE JAZZ S50

Scale: 1:8	Drafted	Verified	Confirmed
Date	03/11/2017	09/07/2025	09/07/2025
Initials	SL	AZ	TK

Plan No.

EC00066/N



Material			
No.	Plan	Article	Number
1	YC00358(L)	1C158	1
	YC00359(R)	1C159	1
2	---	1908236	2
3	---	2990070	4
4	X002679	1994452	2
5	X000582	1005740	2
6	---	1008492	8
7	---	1001564	8
8	---	1006850	4
9	X004309	1904133	2
10	---	1991017	2
11	(L)	3990524	1
	(R)	3990525	1
11A	XC00119(R)	3990382	1
	XC00120(L)	3990383	1
11B	---	1992597	2
11C	X002163	1488100	2
11D	X002169	1992791	2
11E	X002132	1992423	2
16	X004415	17193	2
17	---	1991447	2
18	---	1001572	2

Material			
No.	Plan	Article	Number
19	---	1998994	2
20	---	1998993	2
21	---	3994158	2
22			
23	X003293	1001626	2
24a	XC00122	3990395*	2
24b	XC00037	3990106*	2
25	---	1001684*	2
26	X003801	1900365*	2
27			
28	X004308	1904129	4
29	---	1903903	2

*If assembly rear wheel fixed axle, take out 24b;
 If assembly rear wheel quick released axle,
 take out 24a&25&26, only use 24b.

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

SIDE FRAME TYPE JAZZ S50				---
				Plan No.
				EC00066/N
Scale: ---	Drafted	Verified	Confirmed	
Date	03/11/2017	09/07/2025	09/07/2025	
Initials	SL	AZ	TK	