

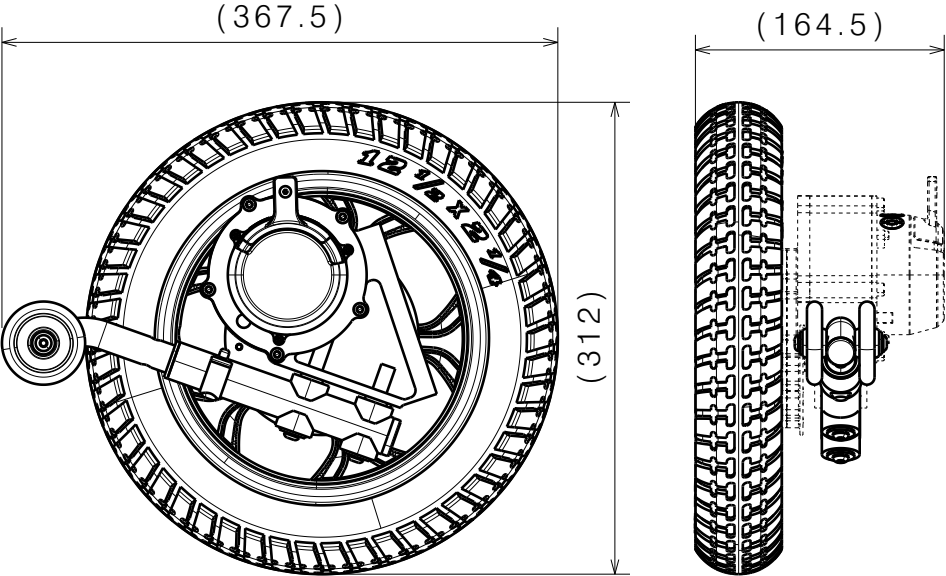


No.	Plan	Article	Qty
1	-	-	-
2	3994671	3994671	1
3	DIN7991 M6X12	3994672	6
4	E00053020	---	1
5	X000106	G003218	4
6	M6X50	2994965	2
7	---	1995235	1
8	2994908	2994908	2

Electrical components are presented in 2998597

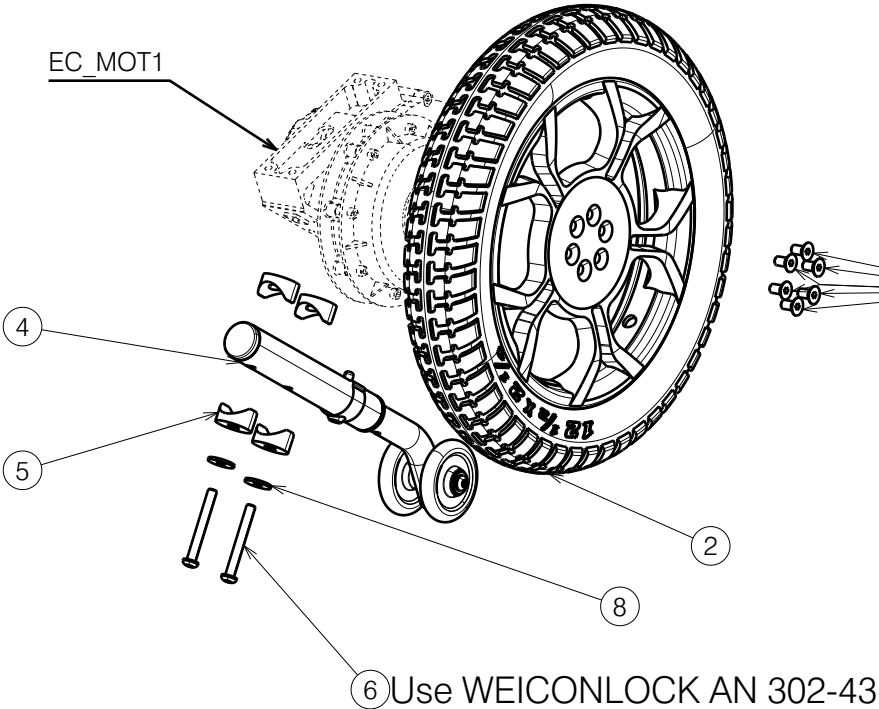

3 10 Nm
Use WEICONLOCK AN 302-43

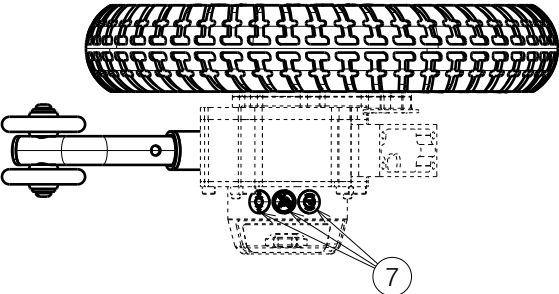

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

LEFT REAR WHEEL PLEGO			
Scale:	1:5	Drafted	Verified
Date	29/06/2023	20/03/2025	20/03/2025
Initials	BK	DR	MK

Plan No.
E00062350/D.1
