


No.	Plan	Article	Qty
1	P00033167	2995472	12
2	P00033418 DP	2650237	1
3	P00033422	2650137	1
4	YP08733	2626300	2
5	X001016	1001682	2
6	X004967	1908262	4
7	XP01421	2993116	6
8	---	1001637	2
9	D8,4 DIN125	1003199	4
10	Cap 15x15	1005899	2
11*	Arrow sticker	1903223	2
12	Motor LA24	1918548	1
13	Control Box	1918550	1
14	Remote Control	1919111	1
15	M8x40 ISO 7380	1008739	4
16	M8x45 DIN931	1005901	2
17	Pen D10x50	1998885	2
18	Panel Cable	1921012	2
19	Headrest Cable	1919185	1
20	D3,9x16 DIN7982	1005767	2
21	Control Box Holder	1919522	1
22	Footrest Cable	1919533	1
23	P00058901	2997834	2

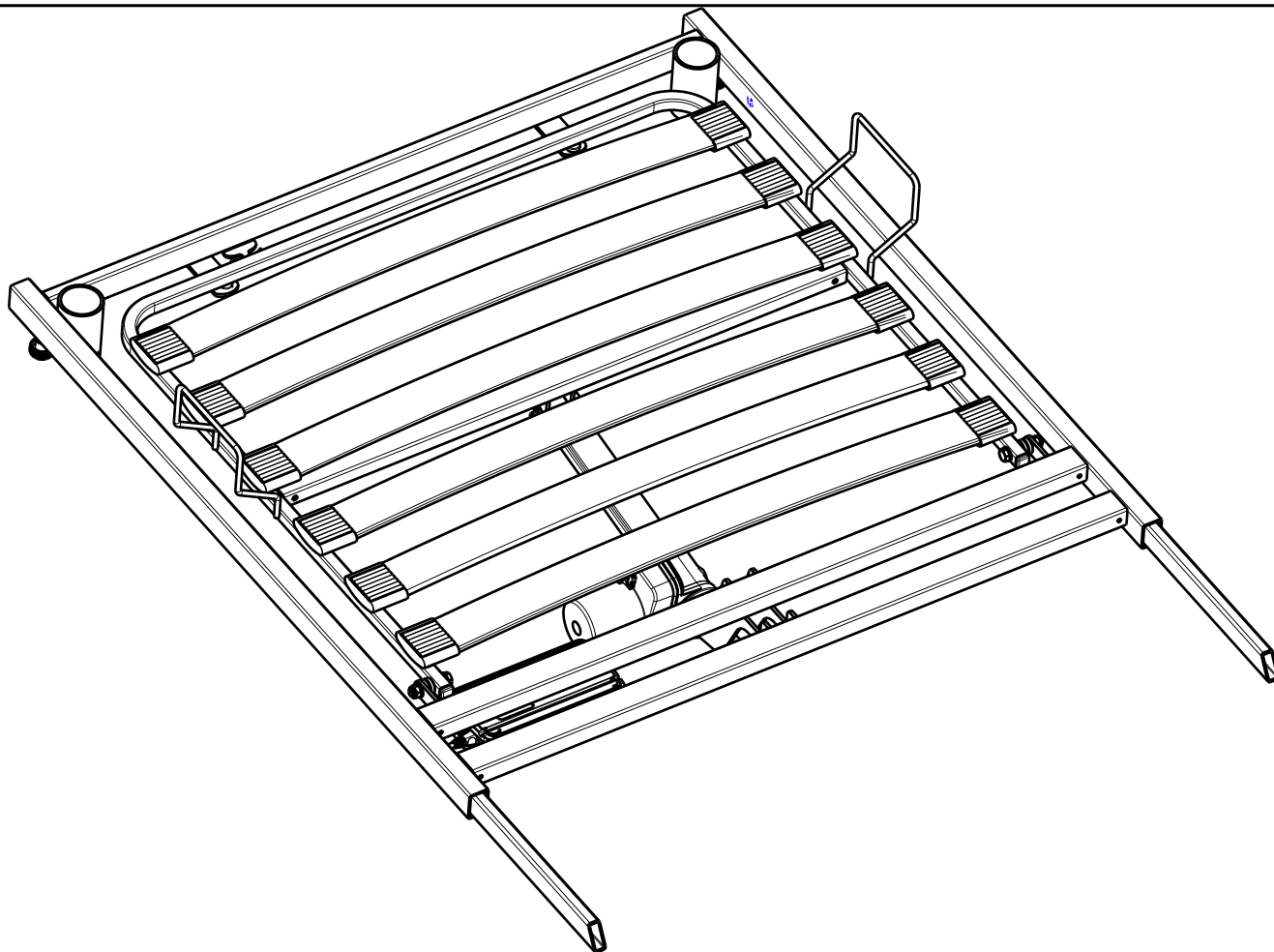

* - Use only with metal side rails (E000037, E000038).
Stickers positioning: use S-PL.70.04.

HEAD SIDE WITH WOODEN LATHS / CLUB ^{2/2}				---
				Plan No. E00044910/G1.1
Scale: 1:8	Drafted	Verified	Confirmed	
Date	2022-10-27	14/02/2025	14/02/2025	
Initials	OY	AZ	AZ	