
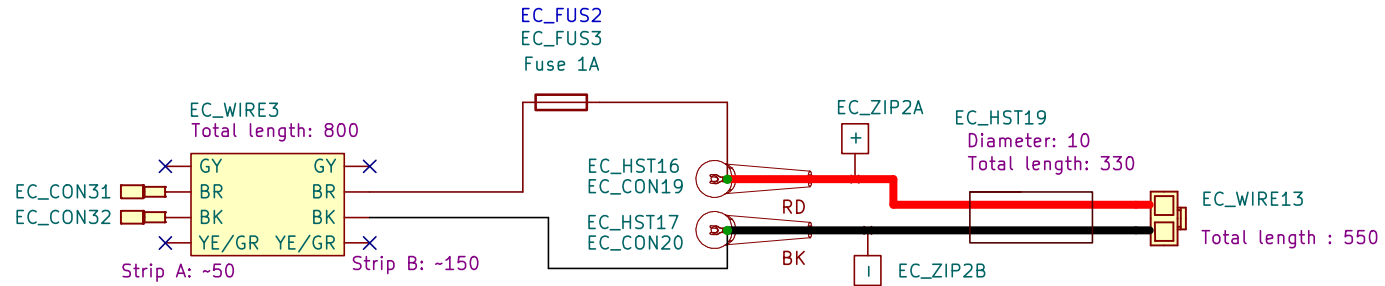


BK BLACK
RD RED
LBL LIGHT BLUE
BL BLUE
LGR LIGHT GREEN
GR GREEN
VI VILLET
OR ORANGE
YE YELLOW
WH WHITE
LRD LIGHT RED
PK PINK
GY GRAY
/ AND
I OR

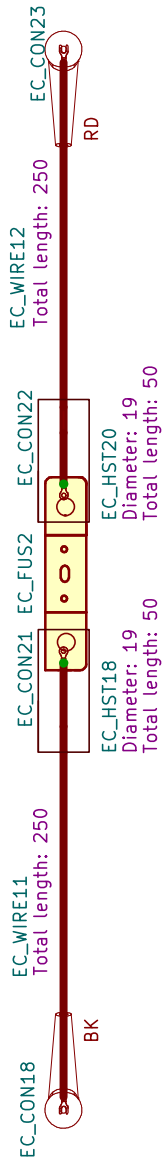

#	Reference	Art number	Description	Qty
1	EC_WIRE11, EC_WIRE12	L000068	EC WIRE 6mm2 BR	0,5M
2	EC_FUS2	1910488	EC FUS Fuse 150A	1
3	EC_HST15, EC_HST17	1994180	EC HST PLASTIC CAP FOR BATTERY NEGATIVE BLACK	2
4	EC_HST16, EC_HST21	1901731	EC HST PLASTIC CAP FOR BATTERY POSITIVE RED	2
5	UNDEPICTED	1007000	HEX SOCKET BUTTON HEAD SCREW M6X12	2
6	UNDEPICTED	1004414	LOCKWASHER DIN127 M6	2
7	UNDEPICTED	1001680	SELFLOCKING NUT DIN985 M6 ZINC	2
8	EC_CON21, EC_CON22	L000007	EC CON CABLE SHOE M6 4-6MM ² HOLE 6MM	2
9	EC_CON18-EC_CON20, EC_CON23	L000001	EC CON CABLE SHOE M6 4-6MM ²	4
10	EC_HST18, EC_HST20	T001009	EC HST Heat Shrink Tubing D19	0,1M
11	EC_WIRE13	1994573	EC WIRE CONNECTOR KIT POWER MODULE+LIGHTS DX2	1
12	EC_ZIP2	1901755	EC ZIP STICKER POLARITEIT + EN -	1
13	EC_HST19	H005006	EC HST ISO ISOLATE COVER D10 PVC BLACK	0,33M
14	EC_WIRE3	1907749	EC WIRE CABLE 4X0,5 BLACK	0,8M
15	EC_CON31, EC_CON32	2996202	EC CON CABLE SHOE 68/76 FEMALE	2
16	EC_FUS3	2998575	EC FUS FUSE HOLDER 5X20	1
17	EC_FUS2	2998574	EC FUS FUSE 1A QUICK BLOW 5X20	1

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORIZATION FROM VERMEIREN GROUP

2995297/B

Plan No.

2995297/B

EC A TURIOS BATTERY HARNESS SET 2

Size: A4

Drafted

Verified

Comformed

Date:

2025-03-11

2025-03-11

2025-03-11

Initials:

FM

FM

TK