



REMOVABLE SIDE GUARD NOT POSSIBLE WITH CAMBER 6

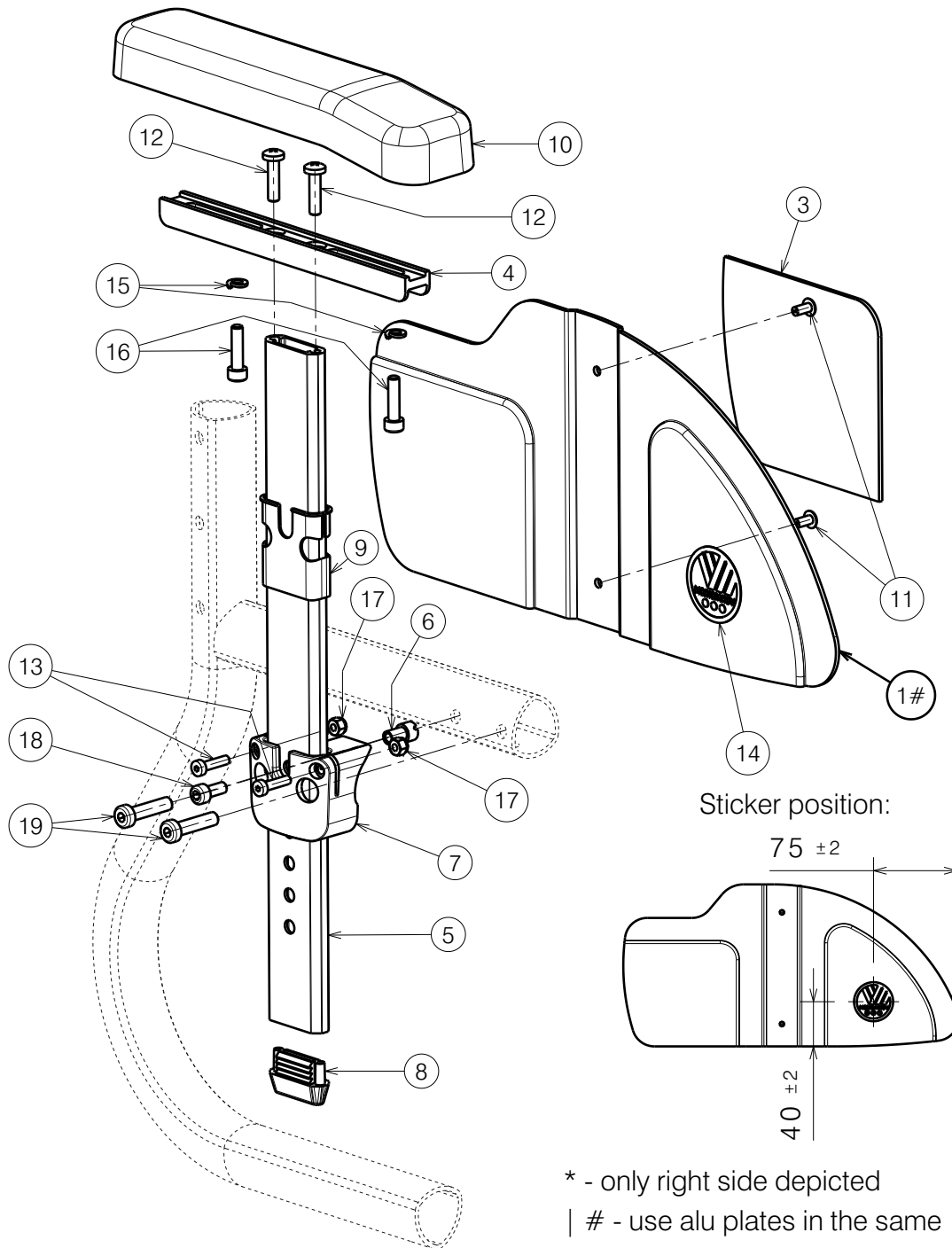

No.	Plan	Article	Qty
1#	P00058348 (R)	26764...	1
	P00058840 (L)	26765...	1
2	P00058354	2997835	1
3	P00058355	2997836	1
4	X005517	2014999	1
5	X005561	2021699	1
6	XP02501	GP18200	1
7	XP03056	GP23226	1
8	XP03080	2996656	1
9	XP03083	2996657	1
10	XP03265	2996861	1
11	Rivet 4x8 DIN7337A	1992805	2
12	M5x16 DIN7985	3991136	2
13	M4x16 DIN7984	2994070	2
14	Sticker Vermeiren	T001002	1
15	M5 DIN127B	1906561	2
16	M5x20 DIN912	1003907	2
17	M4 DIN985	1001678	2
18	M5x12 DIN912	2994072	1
19	M6x25 DIN7984	2994897	2

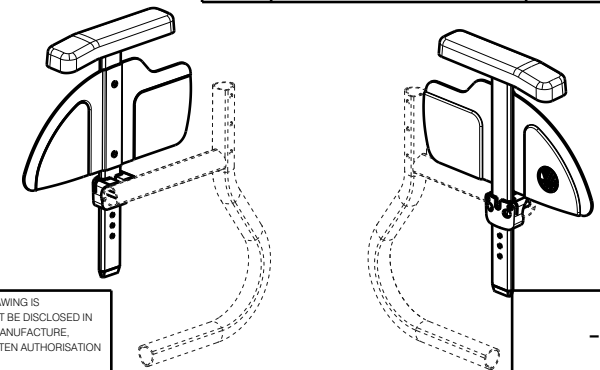

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

ALU BI5 SMALL (BI5A) / TRIGO 30-36

Scale:	1:3	Drafted	Verified	Confirmed
Date	25.04.2023	19/02/2025	19/02/2025	
Initials	AZ	AZ	AS	

Plan No.
E00058843/C1.1