


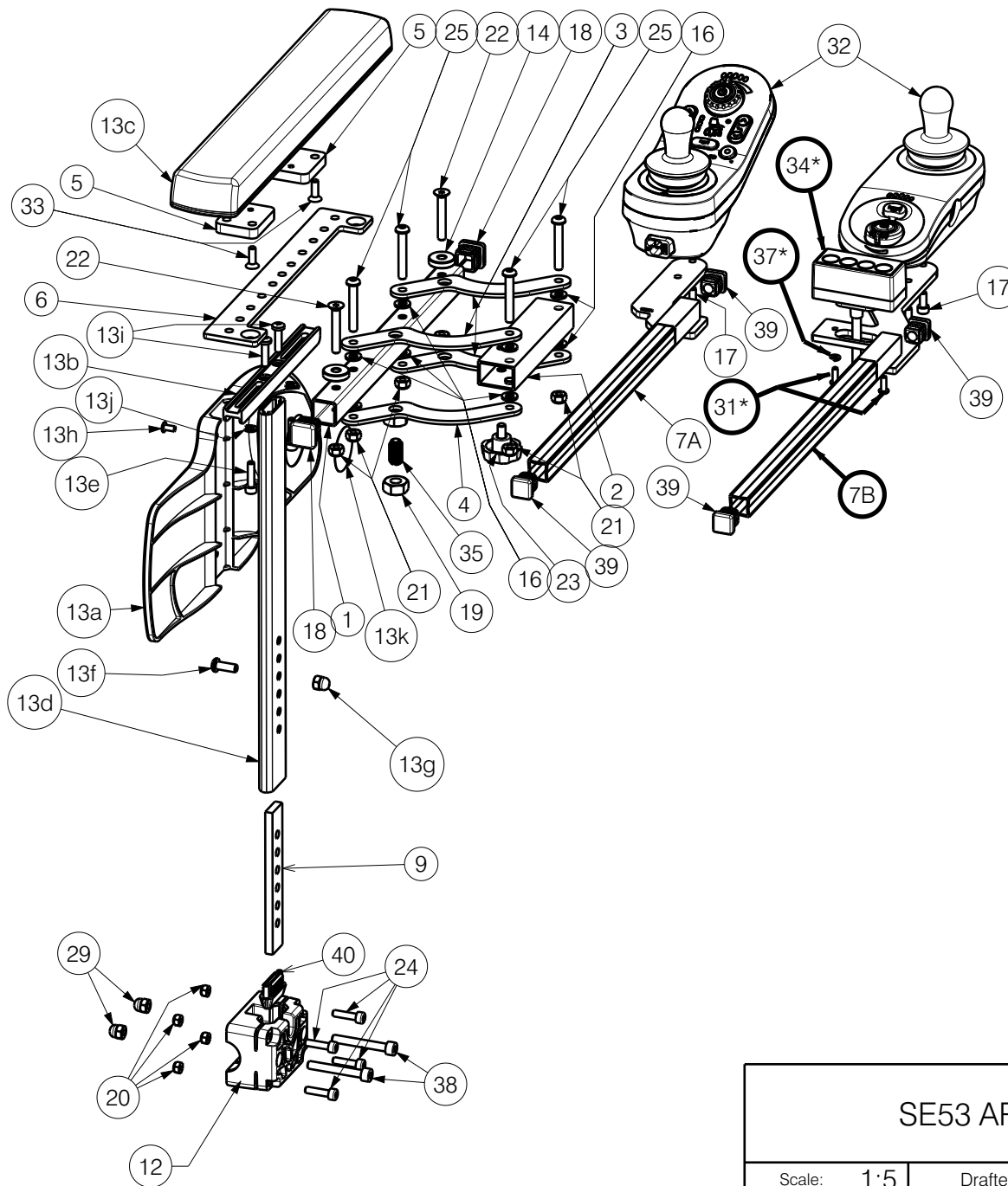

* - used only with SE19

EP00488 / H

SE53 ARMREST / TURIOS				Plan No.
				EP00488 / H
Scale: 1:5	Drafted	Verified	Confirmed	
Date	12/02/2020	13/03/2025	13/03/2025	
Initials	DR	PM	MK	

Material			
No.	Plan	Article	Quantity
1	-	2606454	1
2	Y003626	1476554	1
3	Y009175	1641654	3
4	Y009176	1641754	1
5	YP07839	2783154	2
6	YP08051	2605154	1
7A	YP08058	2605454 (R)	1
	YP08059	2605554 (L)	1
7B*	YP08060	2605654 (R)	1
	YP08061	2605754 (L)	1
8	-	-	-
9	YP09265	2642654	1
10	-	-	-
11	-	-	-
12	X005565	1912287	1
13	Armrest right	1913465	1
	Armrest left	1313464	1
14	XP00227	2990147	2
15	-	-	-
16	Washer M6	1000973	8
17*	M5x12	1001489	2
18	Tube Cap 20x20	1001612	2
19	Nut M10	1001672	1
20	Nut M5	1001679	4
21	Nut M6	1001680	6
22	M6x40	1003111	2
23	Star Knob	1003658	1
24	M5x20	1003907	4
25	M6x40	1006111	4
26	-	-	-
27	-	-	-
28	-	-	-
29	Nut M6	1008492	2
30	-	-	-

Material			
No.	Plan	Article	Quantity
31*	M4X12	2994126	2
32	Linx DLX REM 050	1909020	1
	Linx DLX REM 060	1910700	1
	Linx SE19 SE39	1912296	1
	Linx SE39	1912301	1
	Linx SE19 ACT5	1912305	1
	Linx LCD +JOYST	1913523	1
33	M5x16	1907493	2
34*	Light controller	1921014	1
35	Spring plunger	1991432	1
36	-	-	-
37*	Washer M4 DIN127B	1998515	2
38	M6x45	2994058	2
39	Tube Cap 16x16	2994195	2
40	XP03080	2996656	1

Bill of Material 1913465/1913464			
No.	Plan	Article	Quantity
13a	X005567	1912286 (R)	1
	X005566	1912285 (L)	1
13b	X005517	2014999	1
13c	X005519	1911778	1
13d	X005561	2021699	1
13e	M5x20	1915452	2
13f	M6x16	2994067	1
13g	Nut M6	2994059	1
13h	Rivet Ø4	2995285	2
13i	M5x16	3992507	2
13j	Washer M5	2995288	2
13k	Sticker	T001002	1

* - used only with SE19

EP00488 / H

SE53 ARMREST / TURIOS				Plan No.
				EP00488/ H
Scale: 1:5	Drafted	Verified	Confirmed	
Date	12/02/2020	13/03/2025	13/03/2025	
Initials	DR	PM	MK	