


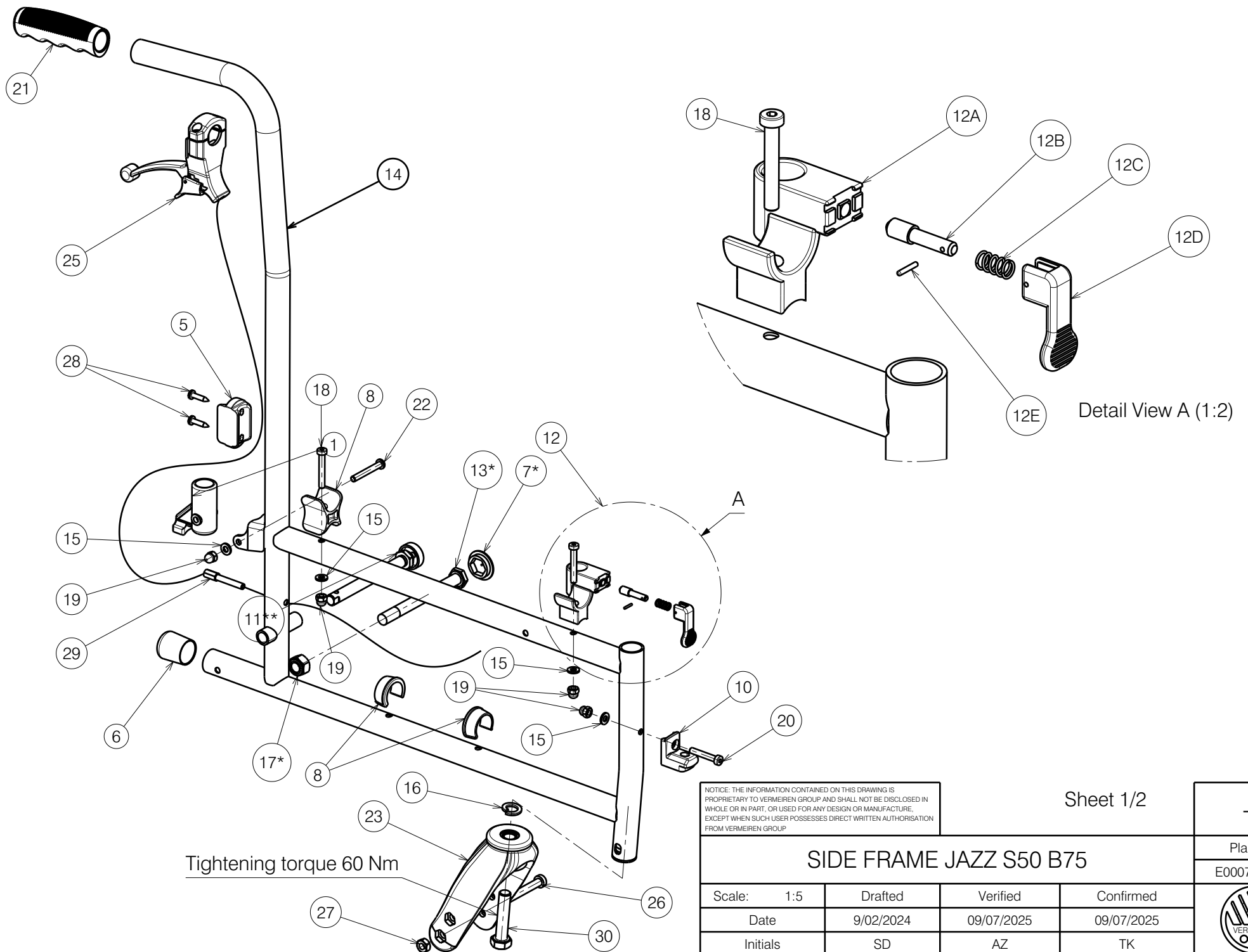

Sheet 1/2

SIDE FRAME JAZZ S50 B75

Scale: 1:5	Drafted	Verified	Confirmed
Date	9/02/2024	09/07/2025	09/07/2025
Initials	SD	AZ	TK

Plan No.

E00075971/C



Material			
N°	Plan	Article	Qty
1	X000582	1005740	2
5	X002679	1994452	2
6	X003293	1001626	2
7*	X003801	1900365	2
8	X004308	1904129	4
9	X004309	1904133	2
10	X004415	17193	2
11**	XC00037	3990106	2
12	---	3990524 (L)	1
	---	3990525 (R)	1
12A	XC00119	3990382 (L)	1
	XC00120	3990383 (R)	1
12B	X002163	1488100	2
12C	X002169	1992791	2
12D	X002132	1992423	2
12E	D2x12 DIN1475	1992597	2
13*	XC00122	3990395	2
14	YC00358	1C158.. (L)	1
	YC00359	1C159.. (R)	1

Material			
N°	Plan	Article	Qty
15	M6 DIN125	1001564	8
16	M12 DIN127	1001572	2
17*	M12 DIN985	1001684	2
18	M6x40 DIN7984	1006850	4
19	M6 DIN986	1008492	8
20	M6x35 DIN7984	1903903	2
21	---	1908236	2
22	M6x45 ISO7380	1991017	2
23	P00006412	1991447	2
25	--	3991154	2
26	M8x76 DIN7984	1998993	2
27	M8	1998994	2
28	4.8x22 DIN7981	2990070	4
29	M6x50	3990117	2
30	M12x55 DIN931	3994158	2

* For fixed rear wheel

** For quick release rear wheel

Left frame only depicted
Quantities are for one wheelchair

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.				Sheet 2/2		---
SIDE FRAME JAZZ S50 B75						Plan No.
						E00075971/C
Scale:	---	Drafted	Verified	Confirmed		
Date		13/02/2024	09/07/2025	09/07/2025		
Initials		SD	AZ	TK		