
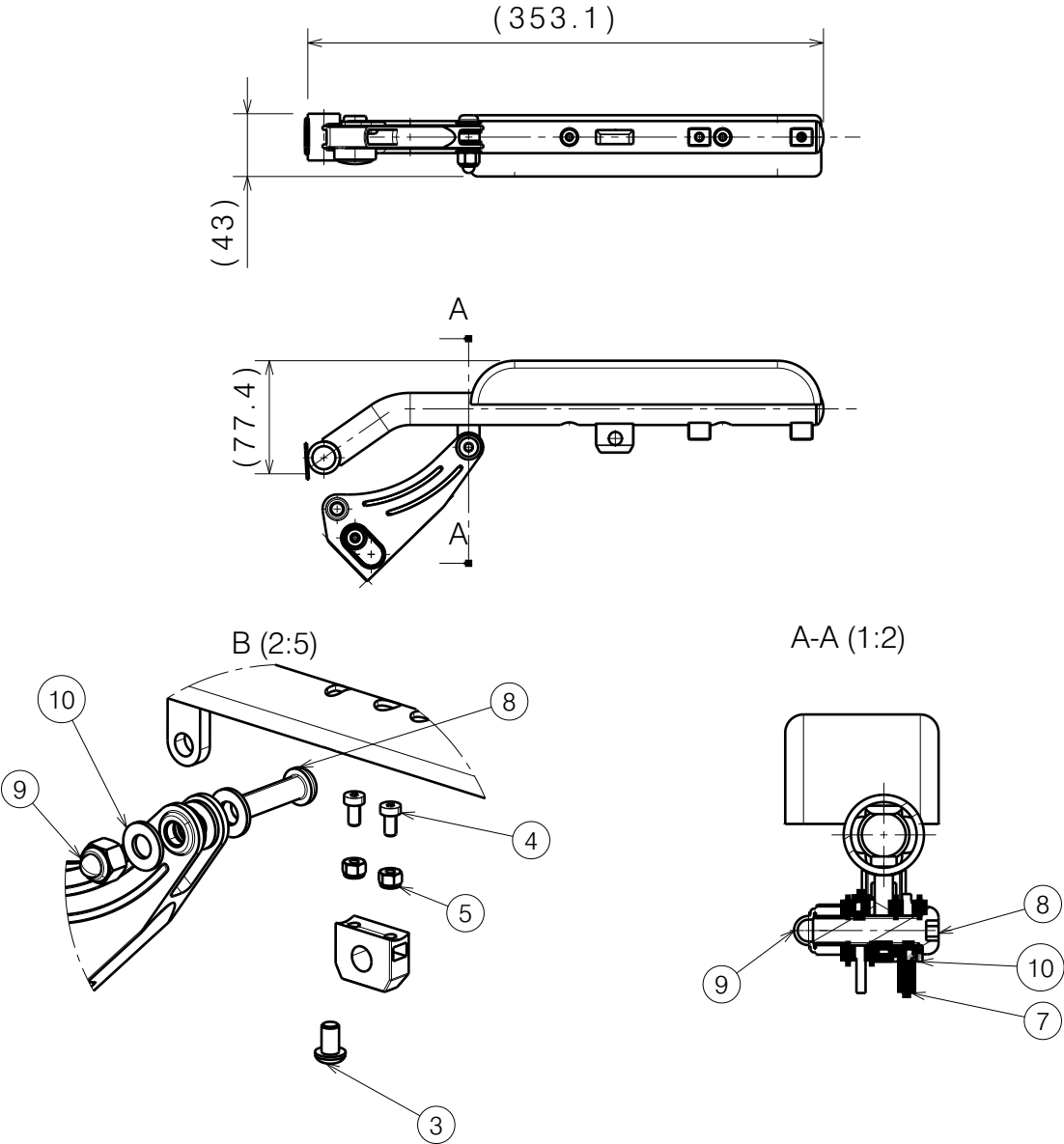


| Bill of Material |                      |         |          |
|------------------|----------------------|---------|----------|
| No.              | Plan                 | Article | Quantity |
| 1                | P00052610            | 2677565 | 1        |
| 2                | X001993              | 1006045 | 1        |
| 3                | M6x10                | 2994127 | 2        |
| 4                | M4x8 DIN7984         | 1996530 | 2        |
| 5                | Nut M4               | 1907202 | 2        |
| 6                | Cap D22              | 1990648 | 1        |
| 7                | ---                  | 3994687 | 1        |
| 8                | M8x30 ISO7380        | 2995396 | 1        |
| 9                | Closed nut M8        | 2995398 | 1        |
| 10               | Washer PA6 M8 DIN125 | 2994415 | 4        |
| 11               | XC00529              | 3992256 | 1        |

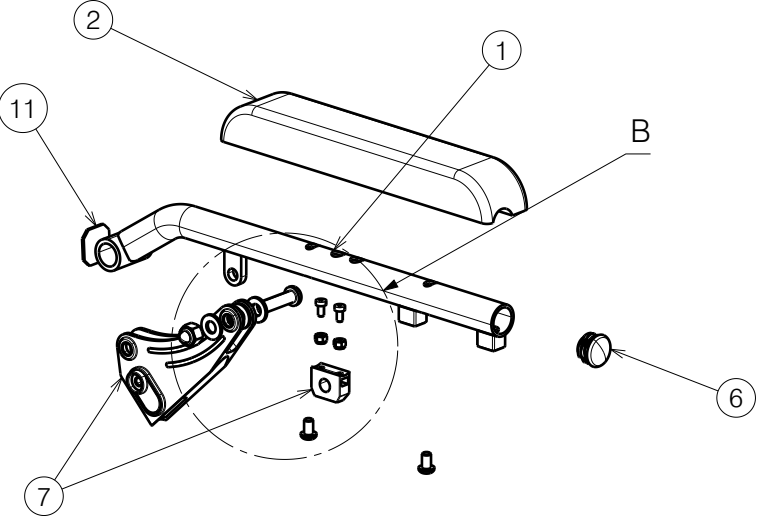

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS  
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN  
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,  
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION  
 FROM VERMEIREN GROUP.

| LEFT ARMREST PLEGO |            |            |            |
|--------------------|------------|------------|------------|
| Scale: 1:5         | Drafted    | Verified   | Confirmed  |
| Date               | 29/06/2023 | 17/02/2025 | 17/02/2025 |
| Initials           | BK         | WM         | MK         |

|                                                                                       |
|---------------------------------------------------------------------------------------|
| ---                                                                                   |
| Plan No.<br>E00062355/D.1                                                             |
|  |