


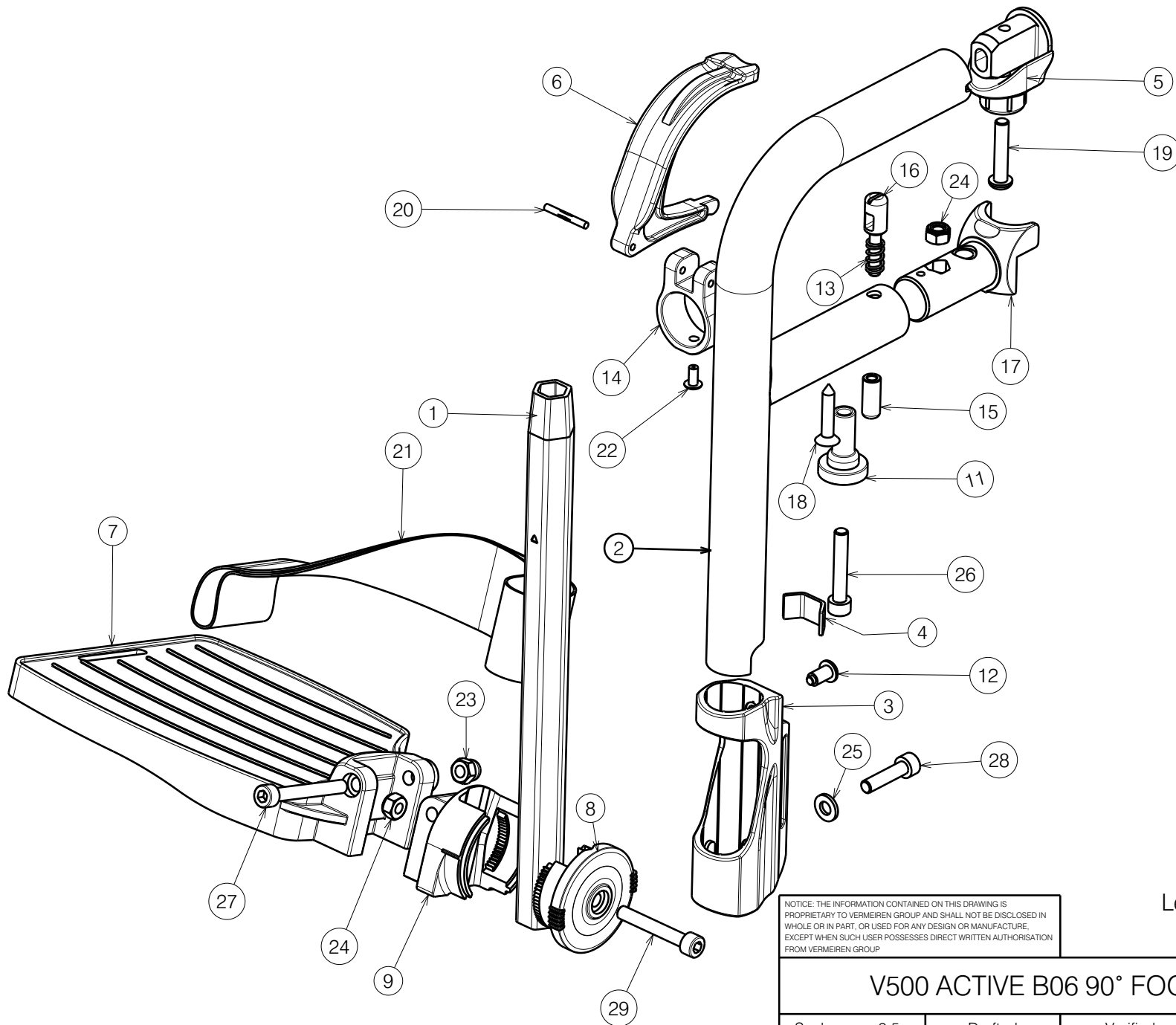

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS  
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN  
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,  
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION  
 FROM VERMEIREN GROUP.


Left Side Depicted  
 Sheet 1/2

|                              |            |            |            |                                                                                       |
|------------------------------|------------|------------|------------|---------------------------------------------------------------------------------------|
| V500 ACTIVE B06 90° FOOTREST |            |            |            | Plan No.                                                                              |
|                              |            |            |            | E00087278/B.1                                                                         |
| Scale: 2:5                   | Drafted    | Verified   | Confirmed  |  |
| Date                         | 12/08/2024 | 25/06/2025 | 25/06/2025 |                                                                                       |
| Initials                     | AS         | AZ         | AS         |                                                                                       |

| Material |           |                     |     |
|----------|-----------|---------------------|-----|
| No.      | Plan      | Article             | Qty |
| 1        | YP06404   | 27301               | 1   |
| 2        | P00074521 | 26850..             | 1   |
| 3        | P00082368 | 3994782             | 1   |
| 4        | Y016501   | 1733500             | 1   |
| 5        | X004414   | 1904130             | 1   |
| 6        | X004418   | 1904207             | 1   |
| 7        | X004570   | 1904646 (L) (39-42) | 1   |
|          | X004571   | 1904647 (R) (39-42) | 1   |
|          | X004457   | 1904475 (L) (44-46) | 1   |
|          | X004453   | 1904210 (R) (44-46) | 1   |
|          | X004572   | 1904648 (L) (48-60) | 1   |
|          | X004573   | 1904649 (R) (48-60) | 1   |
| 8        | X004464   | 1904212             | 1   |
| 9        | X004465   | 1904213             | 1   |
| 11       | X004660   | 1905195             | 1   |
| 12       | X004687   | 1905639             | 1   |
| 13       | XC00142   | 3990576             | 1   |
| 14       | XC00300   | 3991091             | 1   |

| Material |               |         |     |
|----------|---------------|---------|-----|
| No.      | Plan          | Article | Qty |
| 15       | XC00304       | 3991093 | 1   |
| 16       | XC00305       | 3991094 | 1   |
| 17       | XC00381       | 3991544 | 1   |
| 18       | S-Point 5x25  | H016007 | 1   |
| 19       | M6x30 ISO7380 | 1009588 | 1   |
| 20       | D3x24 DIN1475 | 1905640 | 1   |
| 21       | ---           | 1905642 | 1   |
| 22       | D4x8 DIN7337A | 1992805 | 1   |
| 23       | M6 DIN986     | 2994059 | 1   |
| 24       | M6 DIN985     | 1001680 | 2   |
| 25       | M6 DIN125A    | 1001564 | 1   |
| 26       | M6x35 DIN912  | 2994634 | 1   |
| 27       | M6x50 DIN912  | 2995258 | 1   |
| 28       | M6x25 DIN912  | 1001494 | 1   |
| 29       | M6x40 DIN912  | 1003909 | 1   |

Quantities are for one side

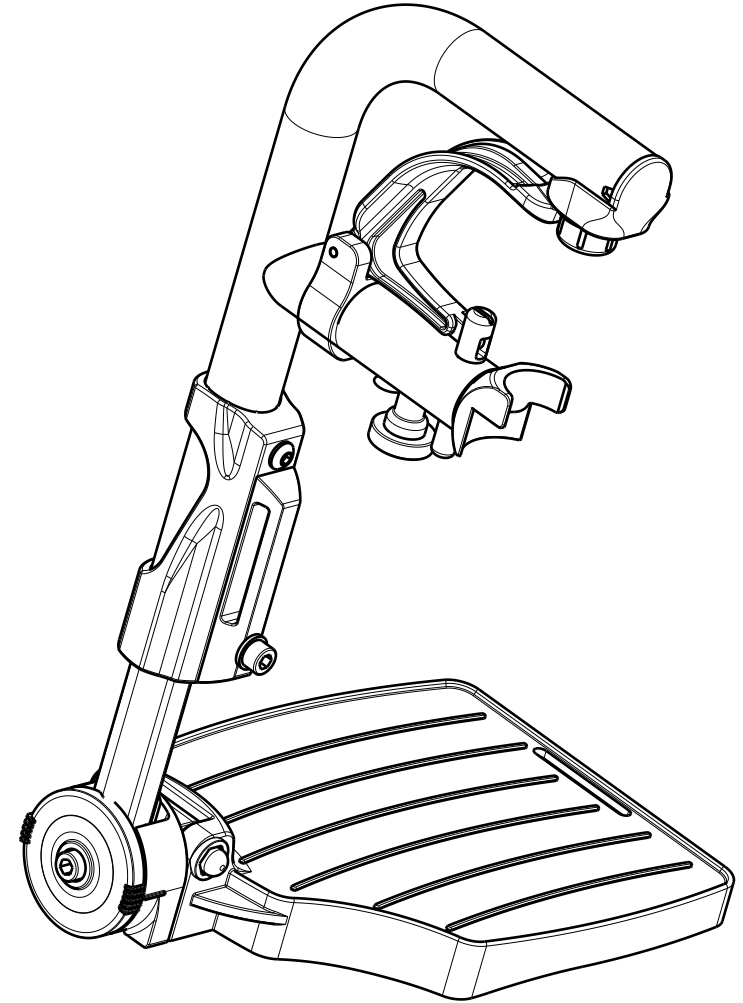

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS  
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN  
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,  
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION  
 FROM VERMEIREN GROUP.

Sheet 2/2

---

| V500 ACTIVE B06 90° FOOTREST |            |            |            | Plan No.      |
|------------------------------|------------|------------|------------|---------------|
|                              |            |            |            | E00087278/B.1 |
| Scale:                       | 2:5        | Drafted    | Verified   | Confirmed     |
| Date                         | 12/08/2024 | 25/06/2025 | 25/06/2025 |               |
| Initials                     | AS         | AZ         | AS         |               |

