
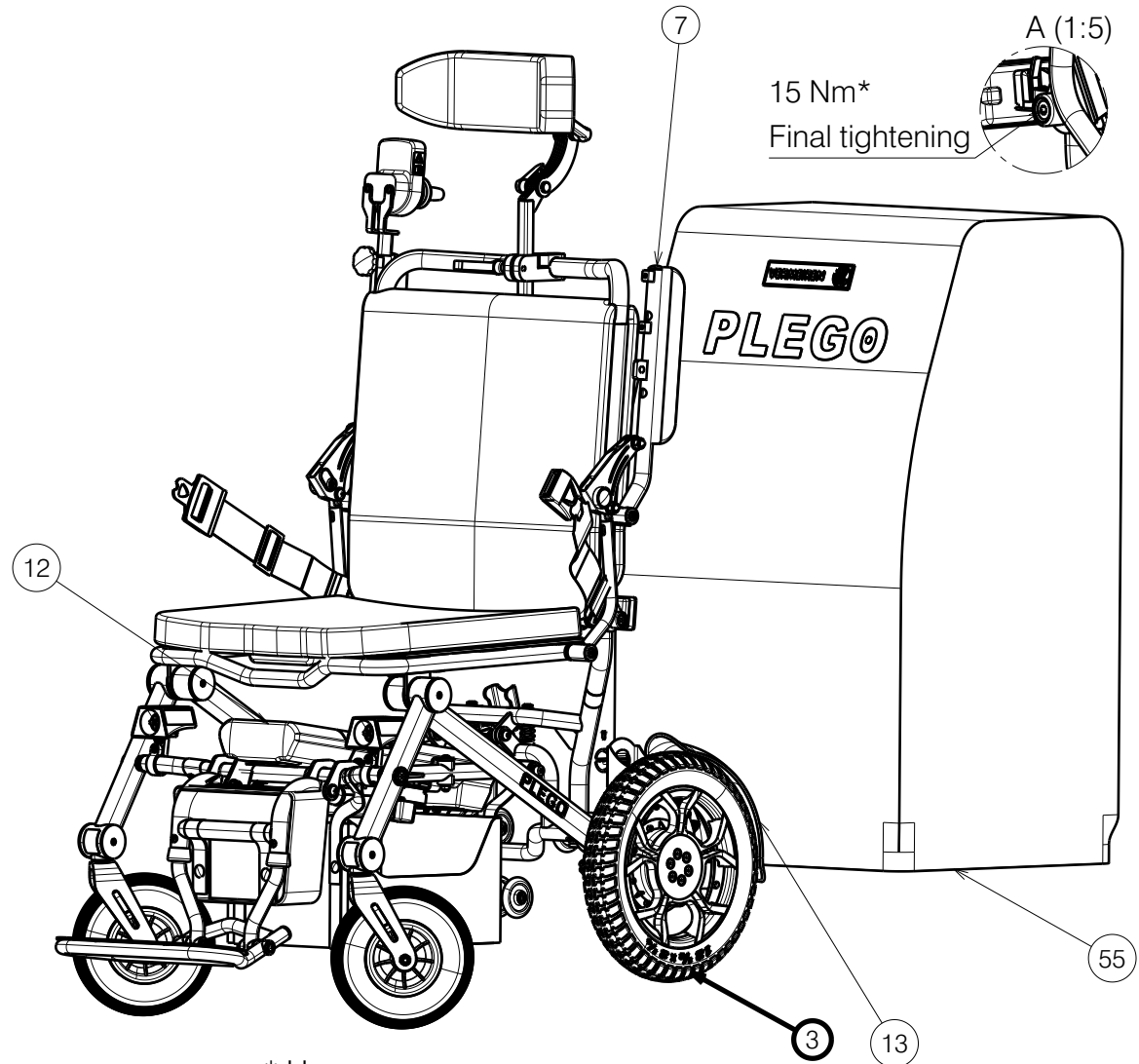


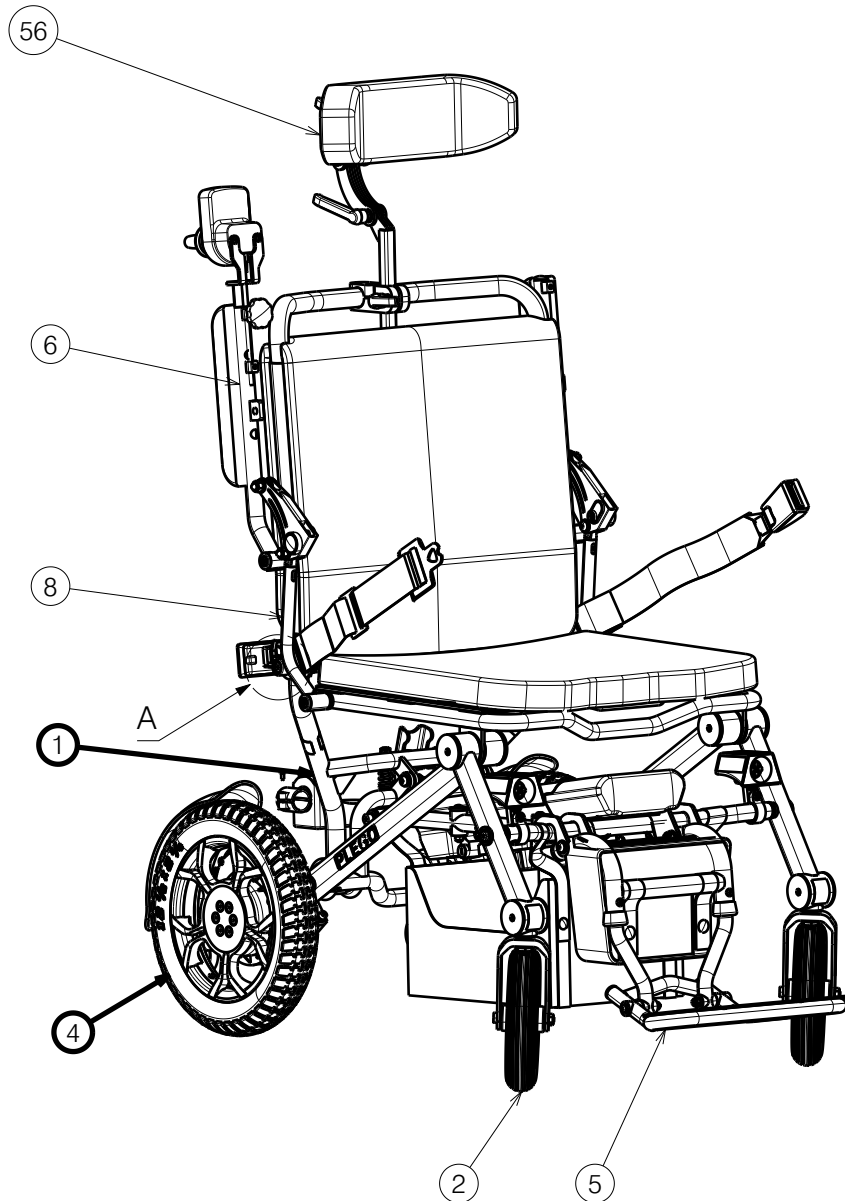


* Use:
WEICONLOCK AN 302-43

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.


Sheet 1/6

PLEGO			
Scale:	1:10	Drafted	Verified
Date	27/09/2023	24/03/2025	24/03/2025
Initials	BK	WM	MK

Plan No. E00052670/F.1

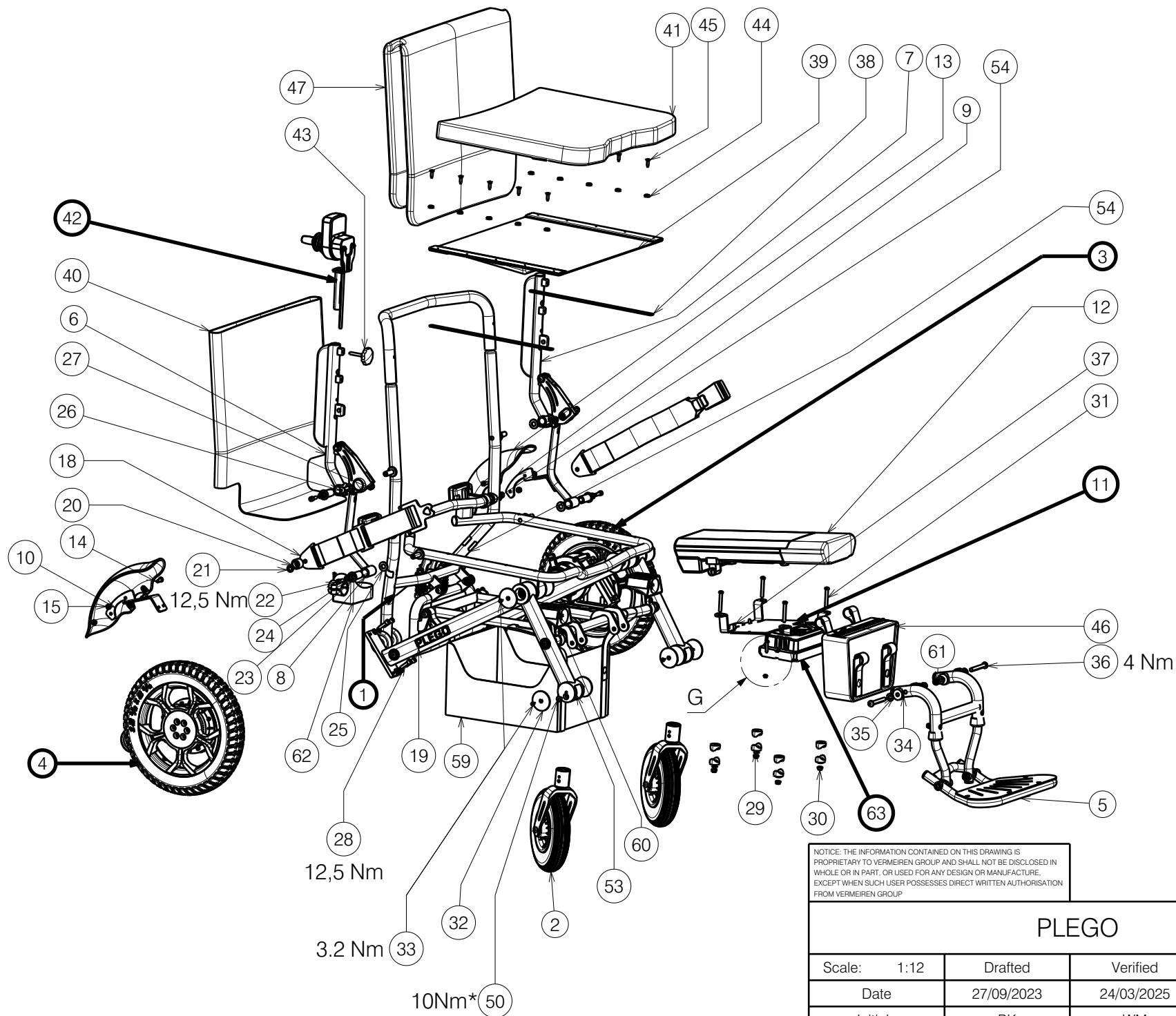G (1:5)


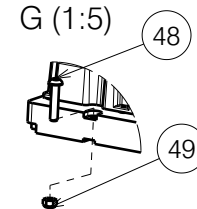


* Use:
WEICONLOCK AN 302-43

Sheet 2/6

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

PLEGO				Plan No.
				E00052670/F.1
Scale: 1:12	Drafted	Verified	Confirmed	
Date	27/09/2023	24/03/2025	24/03/2025	
Initials	BK	WM	MK	


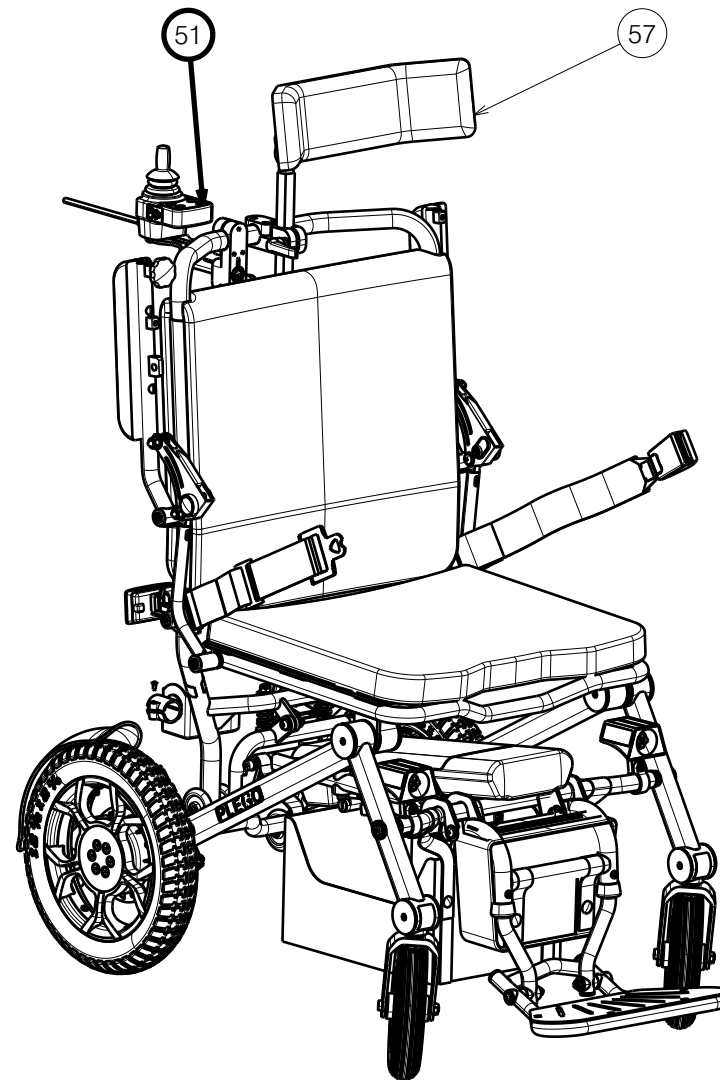
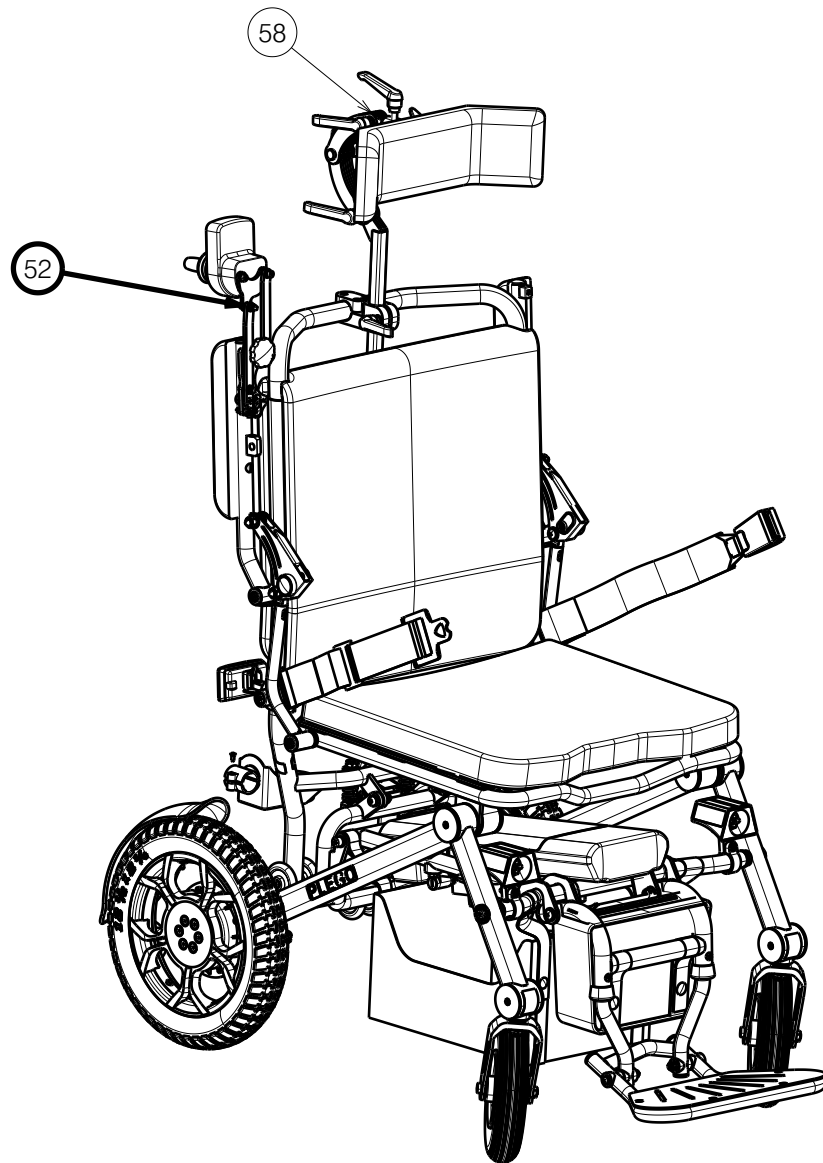


NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

Sheet 3/6

PLEGO			
Scale: 1:10	Drafted	Verified	Confirmed
Date	27/09/2023	24/03/2025	24/03/2025
Initials	BK	WM	MK

Plan No.
E00052670/F.1

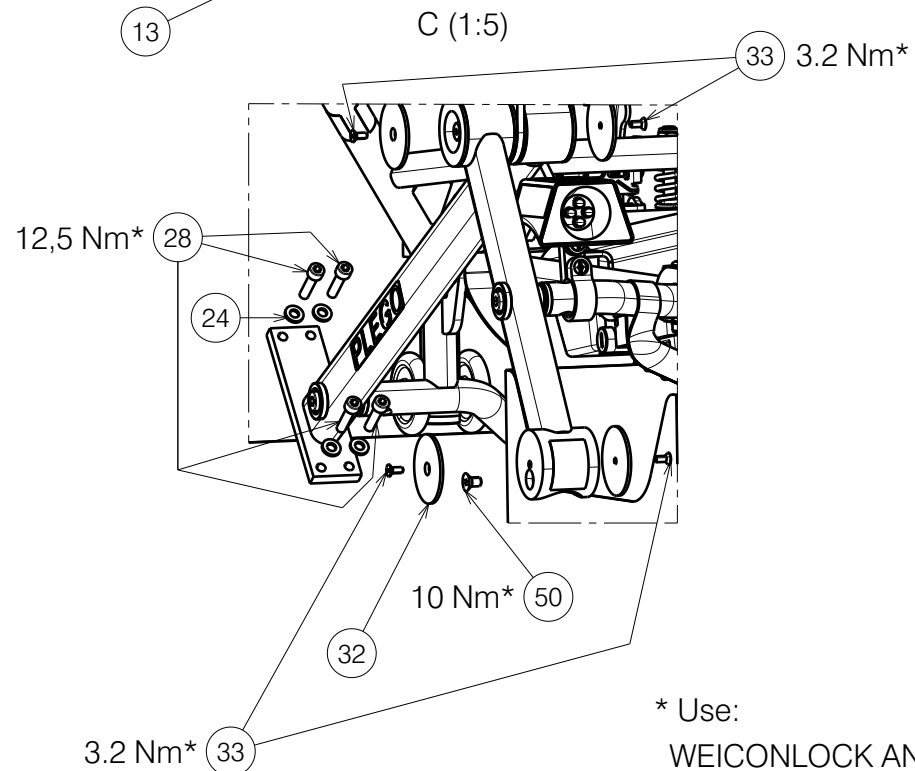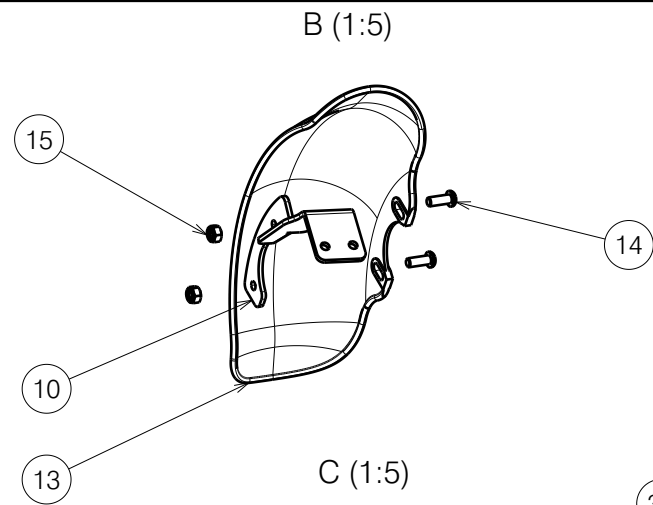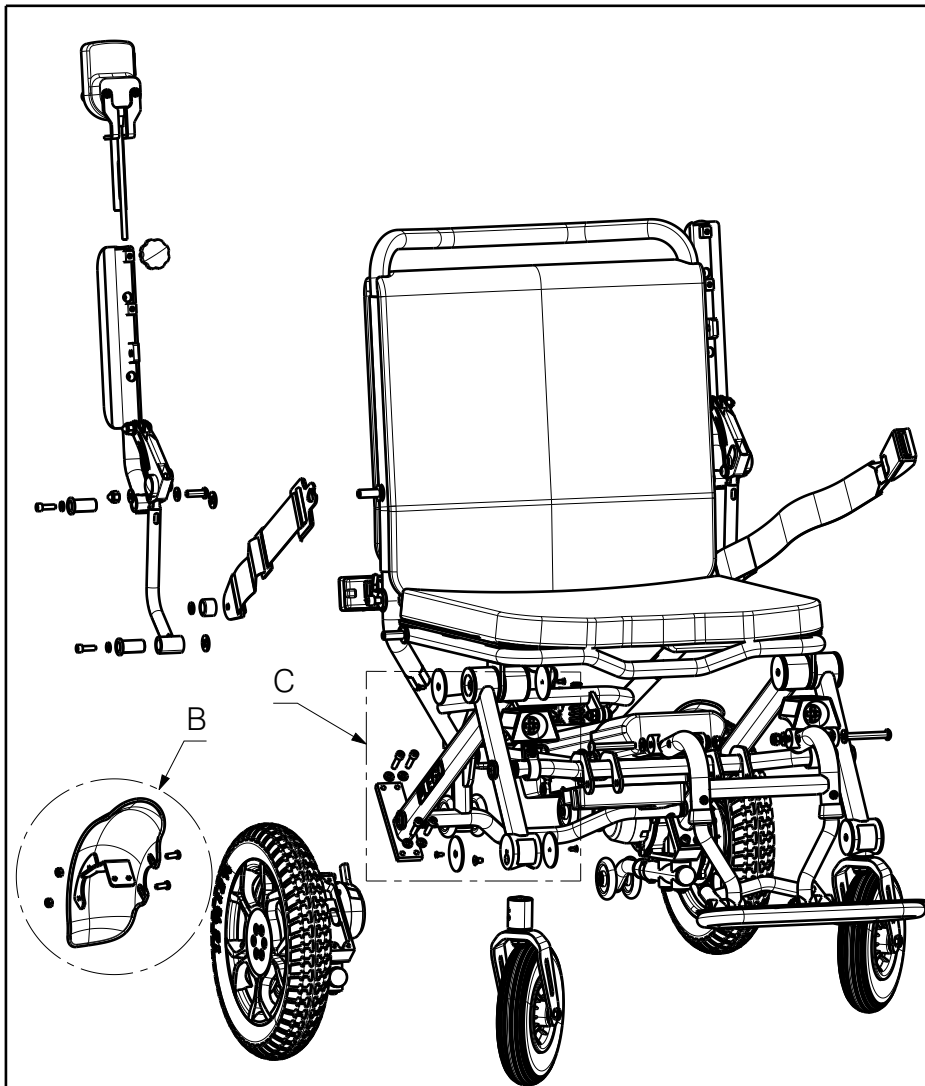



* Use:
WEICONLOCK AN 302-43

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

Sheet 4/6

PLEGO			
Scale:	1:12	Drafted	Verified
Date	27/09/2023	24/03/2025	24/03/2025
Initials	BK	WM	MK

Plan No.
E00052670/F.1

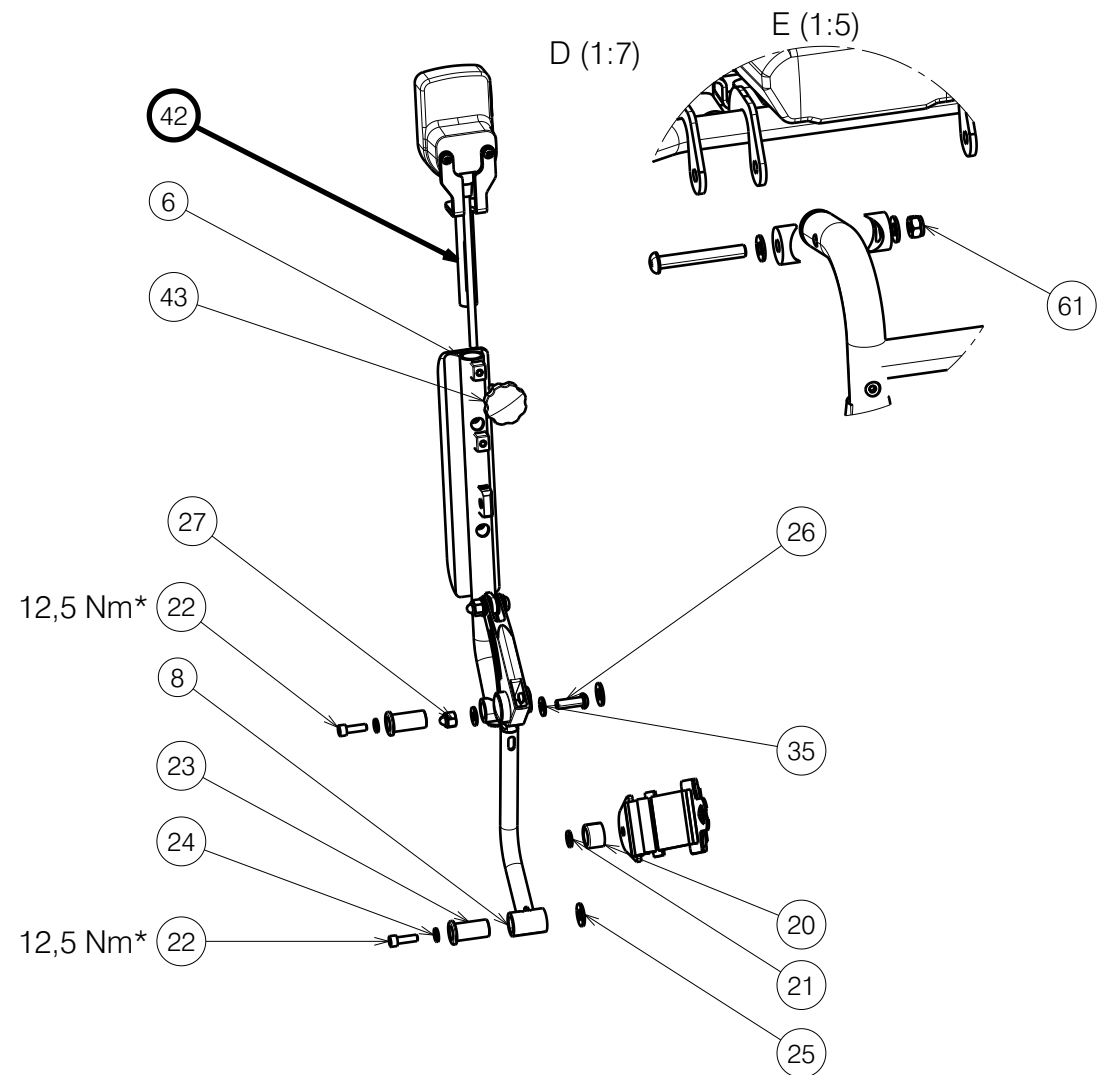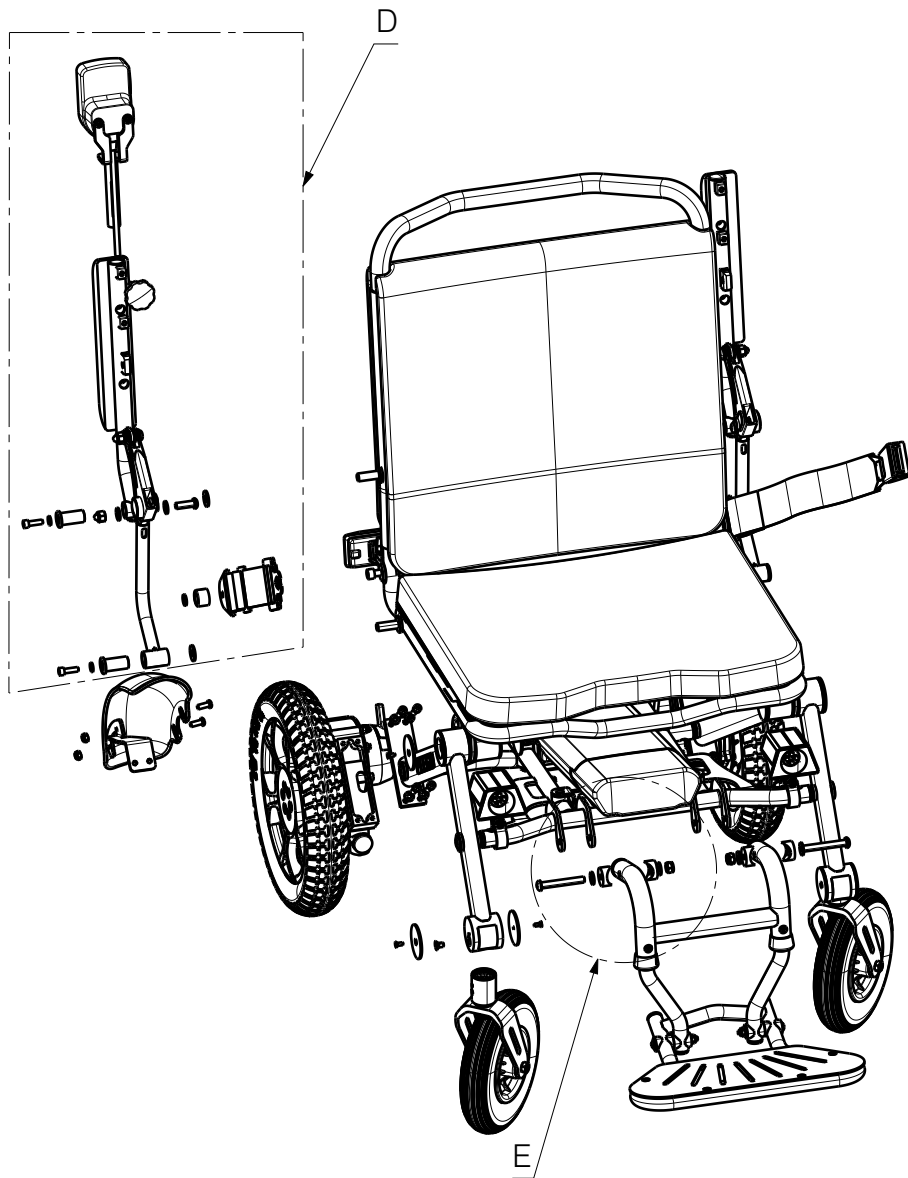

* Use:
WEICONLOCK AN 302-43

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

Sheet 5/6

PLEGO			
Scale:	1:12	Drafted	Verified
Date	27/09/2023	24/03/2025	24/03/2025
Initials	BK	WM	MK

Plan No.
E00052670/F.1

No.	Plan	Article	Qty	24	WASHER M6	2994414	16	45	SCREW PARKER 4.8X19 DIN7983	1001534	10						
1	E00063648	---	1	25	WASHER M10 DIN125	2998106	4					46	A00056460	2997888**	1		
2	E00062349	---	1*					26	M8x25	1990501	2					47	A00056459
3	E00062350	---	1	27	NUT M8	2995398	2					48	M5x25	2994083	2		
4	E00062352	---	1					28	M6X25 DIN912	2996887	8					49	NUT M5
5	E00056068	---	1	29	X000862	1006463	8					50	M6X10	1904683	4		
6	E00063644	---	1					30	NUT DIN985 M6	2994063	4					51	E00054750
7	E00062355	---	1	31	M6x45	2994086	4					52	E00065217/ E00066678	---**/ ---**	1		
8	P00053716	26771	2					32	P00053220	26742B8 26742A9 26742B2 26742A7 26742D5 2674221	8					53	P00066352
9	P00062160	2677665	1	54	P00066353	T001060	2										
10	P00061760	2677765	1									55	A00066354	1920500**	1		
11	P00053715	2678965	1	56	E00066356	---**	1										
12	Battery 24V 12Ah	1918834	1									57	E00066564	---**	1		
	Battery 24V 17Ah	1919283**	1	58	E00066475	---**	1										
13	XC01046	1919584	2									33	M4X10 DIN7991	2998006	8	59	A00066482
14	M6x16	2994067	4	34	X000106	G003218	4	60	P00066485	1920046**	2						
15	NUT M6	2994063	4									35	M8 DIN125	2994415	8	61	NUT M8
16	-	-	-	36	M8X60 ISO7380	2995491	2	62**	B31 CRUTCH HOLDER 25mm	1906616	1						
17	1919240	1919240	1									37	M5x12	2994074	4	63	Drive train standard Drive train + SE19 lights**
18	2993352 ELEMENT 1 /2993352 ELEMENT 2	2993352	1	38	P00054831	26779	2	63	Drive train standard Drive train + SE19 lights**	2998597 2998582	1 1						
	39											A00056456	2997887	1			
				40	A00056458	2997886	1										
41	A00056457	2997884	1														
				42	E00052668	---	1										
43	STAR KNOB M6x35 VGF	2990215	1														
				44	X001473	1002994	10										
20	P00053212	2997848	2	* 1 pair contains 2 pcs - check AS system ** Optional equipment				NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.				Sheet 6/6		---			
21	M8 DIN125B	1001556	2					PLEGO								Plan No.	
22	M6x20	2994004	4													E00052670/F.1	
23	P00053209	2997847	4														
								Scale: 1:10	Drafted	Verified	Confirmed						
								Date	27/09/2023	24/03/2025	24/03/2025						
								Initials	BK	WM	MK						