


A (1:3)

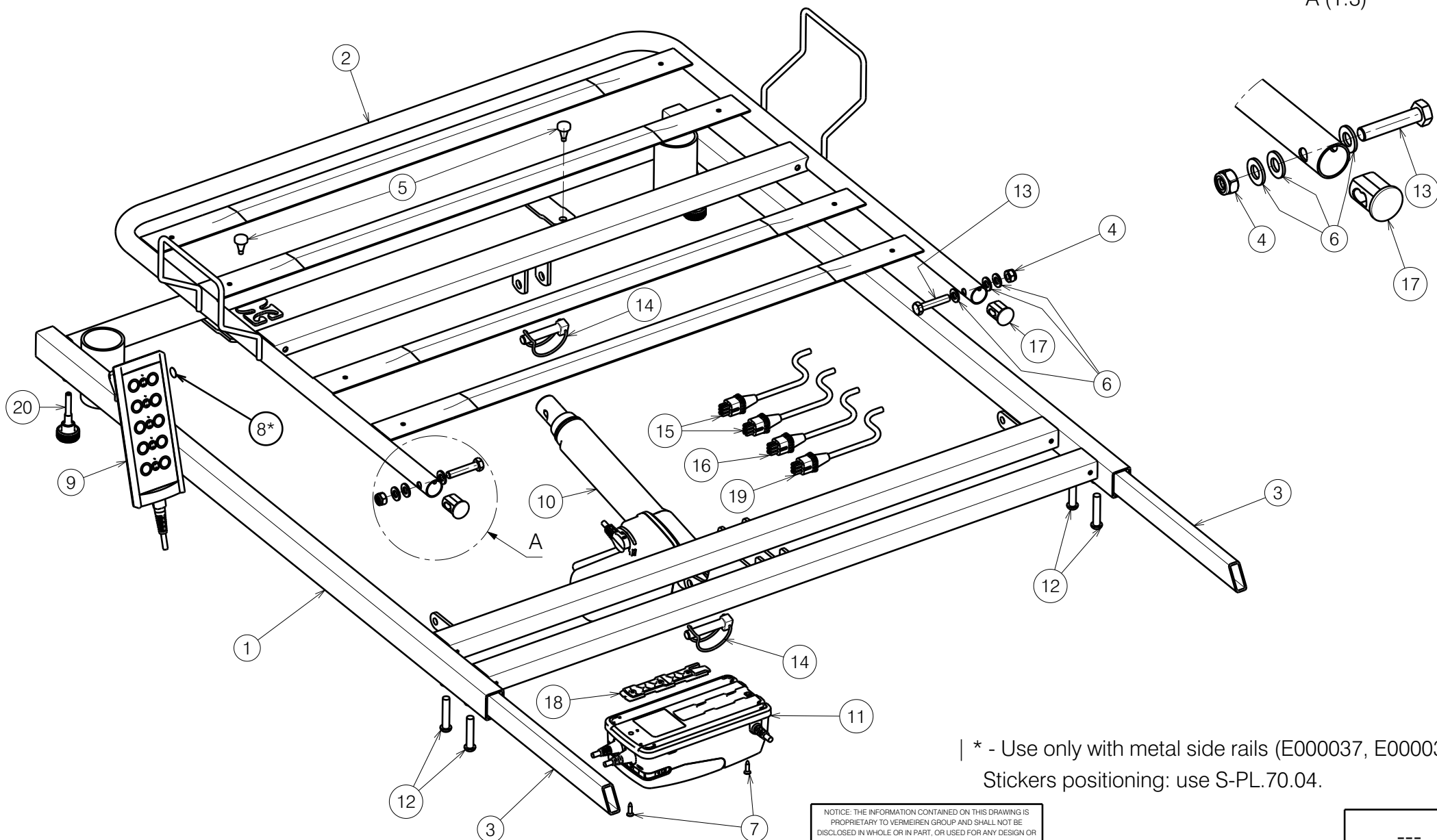

| * - Use only with metal side rails (E000037, E000038).
Stickers positioning: use S-PL.70.04.

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE
DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR
MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT
WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

HEAD SIDE WITH STEEL LATHS / CLUB


1/2

Scale: 1:6	Drafted	Verified	Confirmed
Date	2022-10-27	2024-10-30	2024-10-30
Initials	OY	MK	AZ

Plan No.

E00046581/F1.1



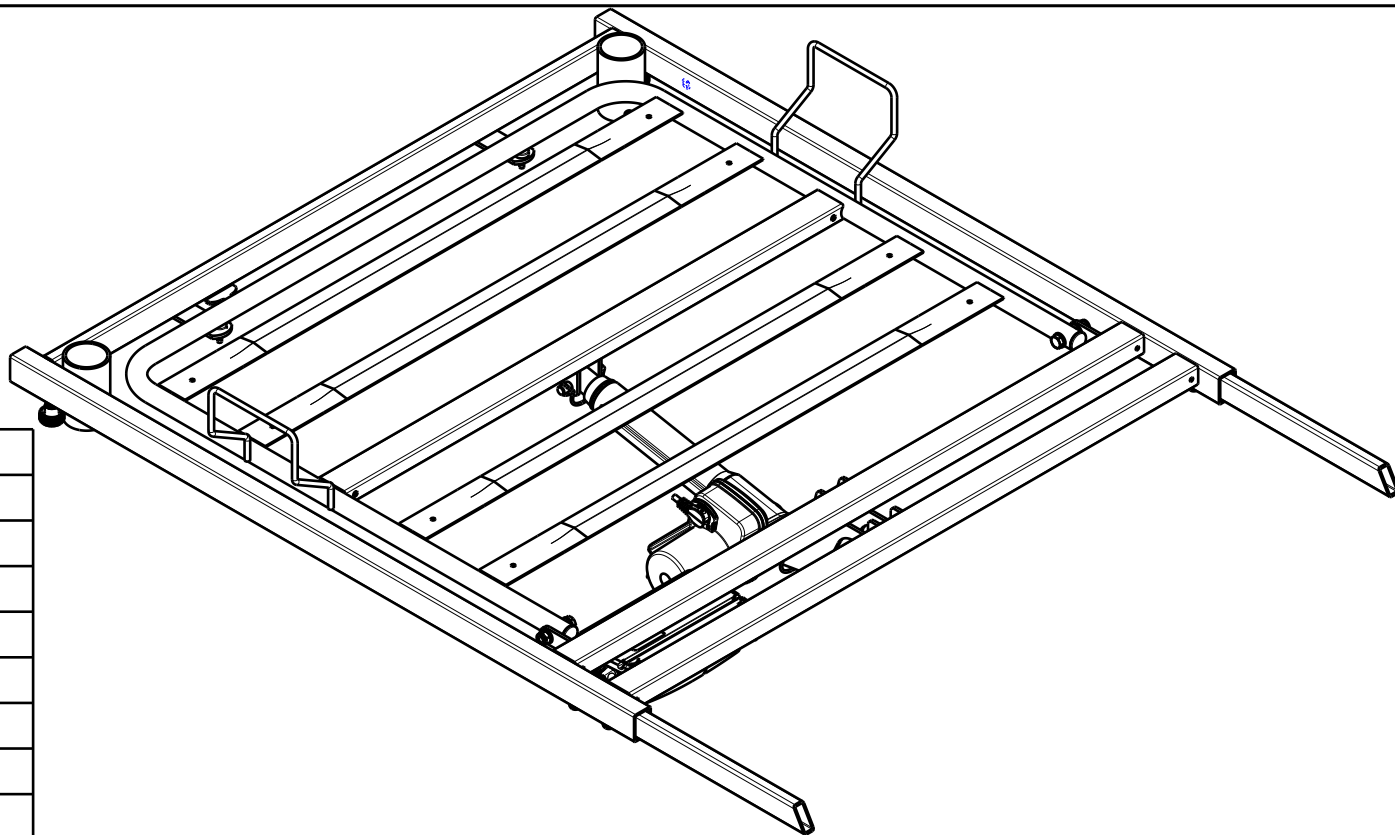

No.	Plan	Article	Qty
1	P00033418 DP	2650237	1
2	YP05099DP	2899537	1
3	YP08733	2626300	2
4	X001016	1001682	2
5	---	1001637	2
6	D8,4 DIN125	1003199	6
7	D3,9x16 DIN7982	1005767	2
8*	Arrow Sticker	1903223	2
9	Remote Control	1919111	1
10	Motor LA24	1918548	1
11	Control Box	1918550	1
12	M8x40 ISO7380	1008739	4
13	M8x40 DIN 931	2993705	2
14	Pen D10x50	1998885	2
15	Panel Cable	1921012	2
16	Headrest Cable	1919185	1
17	XP02064	2993688	2
18	Control Box Holder	1919522	1
19	Footrest Cable	1919533	1
20	P00058901	2997834	2

| * - Use only with metal side rails (E000037, E000038).
Stickers positioning: use S-PL.70.04.

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP				---
HEAD SIDE WITH STEEL LATHS / CLUB				Plan No.
2/2				E00046581/F1.1
Scale: 1:8	Drafted	Verified	Confirmed	
Date	2022-10-27	2024-10-30	2024-10-30	
Initials	OY	MK	AZ	